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«I certify»

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M. Gobet
(seen exclusively to certify the above signature)

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Current operational documentation was prepared for provision of information with the purpose of the medical device registration and use at the territory of the Russian Federation.

Information is presented in the volume required by the Guidance for Medical devices registration (Governmental order #1416 dated from 27.12.2012).

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Medical compression hosiery SMARTLEG® (here-and-after – medical device, compression hosiery).

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Limited liability company (under the laws of the Russian Federation) “INNOTECH”
(LLC “INNOTECH”)
71, Sadovnicheskaya naberezhnaya str., Moscow, 115035, Russia
Phone: **8 800 250 1738**.

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For low extremities venous blood flow and microcirculation amelioration with the purpose of treatment and prevention of legs edema, heaviness and pain in cases of functional venous insufficiency (phlebopathy), telangiectasia (starburst veins or vascular spider), varicose veins, and thrombosis prevention during the pregnancy and in post-delivery period. The medical device is suitable for multiple use.

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Medical compression hosiery is an important component of low extremities veins diseases prophylaxis and treatment. It differs from standard compression clothes in compression, which is different at each separate part of the device.

Medical compression hosiery SMARTLEG® is presented in a wide range of models, most requested from which are tights, stockings and knee-socks.

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It's a medical device, representing stocking-socks device, fully covering the lower part of the body and legs, including each foot separately.

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The product is manufactured by means of circle knitting technology with subsequent seaming and gusset insertion. Sutures are plane.

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- tight torso in the area of thigh upper part and perineum is presented in a form of tight panties with oval gusset;
- transition zone (with a drawing) between the thigh and the panty
- the leg knitted with the knitting yarn and a lay-in yarn is smooth and without drawings
- tread is closed, produced from smooth knitting with strengthened bottom;
- classic heel, knitted with reinforced lower part, height of at least 30 mm and not more than 60 mm;
- classic form reinforced toe closed with a stitch suture

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It is a medical device in form of stocking-socks device, fully covering leg and foot.

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The product is produced by means of circle knitting technology.

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- buffet (edge of the stocking in the thigh upper part) in a form of lace ribbon of not more than 60 mm height with attached silicon band of not more than 40 mm height, providing reliable stockings fixation on the leg;
- calf (leg of stocking), smooth, without paintings, knitted with the knitting yarn and a lay-in yarn;
- tread is closed, produced by smooth knitting with strengthened bottom;
- classic heel, knitted with reinforced lower part, height of at least 30 mm and not more than 60 mm;

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It is a medical device in form of stocking-socks device, covering foot and the lower part of the leg up to the knee.

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Following elements of the product are determined:

- buffet (edge of the sock in the calf upper part) in a form of smoothly knitted solid, double ribbon of not more than 45 mm height;

- calf (leg of the sock), smooth, without paintings, knitted with the knitting yarn and a lay-in yarn;

- tread is closed, produced by smooth knitting with strengthened bottom;

- classic heel, knitted with reinforced lower part, height of at least 30 mm and not more than 60 mm;

- classic form reinforced toe closed with a stitch suture.

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Document #	Date	Title
Board Directive EC 93/42	June, 14, 1993	Board Directive EC 93/42 on Medical devices
Board Directive EC 2007/47	September, 05, 2007	Board Directive EC 2007/47/EC on Medical devices
ISO 9001:2015	2015	Quality management systems. Requirements.
NF-S97-114	November, 01, 2011	Medical sock-stocking devices
ISO 15223-1:2016	November, 01, 2016	Medical devices. Symbols for use at medical devices labels, labelling and presented information. Part 1. General
ISO 3758:2012	April, 01, 2012	Textile. Care rules labelling symbols at labels.
NF G30-102	1986	Determination of compression pressure
ISO 14971	2012	Application of risk management to medical devices Risk management
NF EN 1041+A1	Novembre 2013	Information provided by the manufacturer

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Main parameters and characteristics are provided in tables ## 2-6.

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* The medical device is delivered to the consumer through sale in specialised shops (salons). Individual product selection is carried out with the participation of consultants authorized by the manufacturer.
Operational documentation containing up-to-date and sufficient data on medical devices is provided to consultants as an information materials, as well as catalogues.

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Table 3. Recommended maximum shoe size (in accordance with the Russian scale of sizes)

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Compression devices sizes	T0	T1	T1+	T2	T2+	T3	T3+	T4
Recommended maximal shoes size	40	40	40	41	41	42	42	43

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Table 4.1

№		Total length Court/ Normal/ Long (mm) ± 50	Foot length (mm) ± 10	Width of the thigh (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression tights opaque size T0	858/902/940	155,0	165,0	116,0	69,0	114
2	Compression tights opaque size T1	843/924/950	155,0	165,0	119,0	75,0	114
3	Compression tights opaque size T1+	897/897/946	160,0	183,0	123,0	75,0	122
4	Compression tights opaque size T2	855/938/945	160,0	182,0	122,0	78,0	127
5	Compression tights opaque size T2+	893/917/947	160,0	198,0	125,0	78,0	136
6	Compression tights opaque size T3	930/941/1007	160,0	190,0	124,0	86,5	149
7	Compression tights opaque size T3+	892/904/976	160,0	198,0	130,0	88,0	180
8	Compression tights opaque size T4	876/938/987	160,0	195,0	126,0	90,0	185

Table 4.2

№		Total length Court/ Normal/ Long (mm) ± 50	Foot length (mm) ± 10	Width of the thigh (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression tights semi- transparent size T0	689/718/760	135,0	143,0	105,0	75,0	71
2	Compression tights semi- transparent size T1	684/688/742	135,0	142,0	106,0	80,0	69
3	Compression tights semi- transparent size T1+	700/720/738	135,0	151,0	110,0	80,0	74
4	Compression tights semi- transparent size T2	685/691/739	135,0	157,0	108,0	85,0	77
5	Compression tights semi- transparent size T2+	718/736/770	145,0	163,0	120,0	86,0	84
6	Compression tights semi- transparent size T3	718/742/747	150,	168,0	111,0	93,0	86
7	Compression tights semi- transparent size T3+	765/784/835	150,0	178,0	126,0	93,0	92
8	Compression tights semi- transparent size T4	750/790/795	150,0	178,0	125,0	94,0	95

Table 4.3

№ п.п.		Total length Court/ Normal/ Long (mm) ± 50	Foot length (mm) ± 10	Width of the thigh (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression tights transparent size T0	695/735/800	130,0	147,0	101,0	69,0	62
2	Compression tights transparent size T1	685/720/760	130,0	145,0	102,0	75,0	62

3	Compression tights transparent size T1+	730/800/800	130,0	152,0	113,0	74,0	66
4	Compression tights transparent size T2	690/710/755	130,0	153,0	106,0	78,0	72
5	Compression tights transparent size T2+	755/760/790	130,0	159,0	114,0	78,0	76
6	Compression tights transparent size T3	670/705/710	130,0	160,0	113,0	87,0	80
7	Compression tights transparent size T3+	760/760/820	135,0	180,0	124,0	87,0	87
8	Compression tights transparent size T4	730/775/795	140,0	172,0	121,0	93,0	90

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Table 4.4

№		Total length Court/ Normal (mm) ± 40	Foot length (mm) ± 10	Width of the thigh (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression stockings semi-transparent size T0	422/443	130,0	140,0	96,0	73,0	53
2	Compression stockings semi-transparent size T1	405/455	130,0	144,0	99,0	79,0	58
3	Compression stockings semi-transparent size T1+	430/446	130,0	160,0	106,0	80,0	62
4	Compression stockings semi-transparent size T2	423/474	135,0	150,0	107,0	85,0	67
5	Compression stockings semi-transparent size T2+	420/450	135,0	168,0	111,0	86,0	71
6	Compression stockings semi-transparent size T3	440/480	140,0	168,0	118,0	93,0	73
7	Compression stockings semi-transparent size T3+	424/465	140,0	181,0	124,0	93,0	76
8	Compression stockings semi-transparent size T4	446/457	140,0	178,0	129,0	104,0	79

Table 4.5

№		Total length Court/ Normal (mm) ± 40	Foot length (mm) ± 10	Width of the thigh (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression stockings transparent size T0	455/475	130,0	146,0	102,0	67,0	50
2	Compression stockings transparent size T1	433/463	130,0	140,0	99,0	76,0	51
3	Compression stockings transparent size T1+	427/450	130,0	146,0	104,0	74,0	53
4	Compression stockings transparent size T2	447/468	135,0	154,0	109,0	78,0	56
5	Compression stockings transparent size T2+	455/474	135,0	158,0	113,0	78,0	59
6	Compression stockings transparent size T3	449/482	140,0	160,0	117,0	86,0	61
7	Compression stockings transparent size T3+	455/475	140,0	180,0	122,0	87,0	65
8	Compression stockings transparent size T4	465/475	140,0	172,0	121,0	93,0	68

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Table 4.6

№		Total length Court/Normal (mm) ± 30	Foot length (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression knee-socks semi-transparent size T0	289/300	130,0	102,0	71,0	30
2	Compression knee-socks semi-transparent size T1	295/306	130,0	102,0	80,0	31
3	Compression knee-socks semi-transparent size T1+	318/341	130,0	113,0	80,0	34
4	Compression knee-socks semi-transparent size T2	300/320	135,0	111,0	86,0	36
5	Compression knee-socks semi-transparent size T2+	302/335	135,0	117,0	86,0	36

6	Compression knee-socks semi-transparent size T3	300/320	140,0	116,0	93,0	38
7	Compression knee-socks semi-transparent size T3+	325/340	140,0	126,0	93,0	42
8	Compression knee-socks semi-transparent size T4	310/343	140,0	127,0	96,0	44

Table 4.7

№		Total length Court/ Normal (mm) ± 30	Foot length (mm) ± 10	width of the calf (mm) ± 8	width of the ankle (mm) ± 5	Mass* (weight) (g) ± 10%
1	Compression knee-socks transparent size T0	297/302	125,0	103,Ж0	68,0	25
2	Compression knee-socks transparent size T1	296/313	130,0	99,0	75,0	26
3	Compression knee-socks transparent size T1+	305/310	130,0	111,0	74,0	27
4	Compression knee-socks transparent size T2	320/333	135,0	106,0	78,0	31
5	Compression knee-socks transparent size T2+	319/325	135,0	115,0	78,0	32
6	Compression knee-socks transparent size T3	310/334	140,0	113,0	85,0	35
7	Compression knee-socks transparent size T3+	320/325	140,0	122,0	87,0	35
8	Compression knee-socks transparent size T4	320/335	140,0	125,0	93,0	35

* Weight is specified for products with length - Normal

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Name	Shrinkage after washing (%)	Elongation in length before and after washing (%)	Working tensile width before and after washing (%)	Surface density (g/m ²)	Breaking load before and after washing H (kg)	Breaking elongation before washing (after washing) (%)	Residual deformation at elongation before and after the washing (%)	The method of fixation on patients body / hold reliability
1. Opaque compression tights SMARTLEG® Color: Splendide, Prodigieuse, Reveuse, Captivante Size: T0, T1, T1+, T2, T2+, T3, T3+, T4, Length: Court, Normal, Long	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of dense elastic band / the product is securely held on the patient's body
2. Semi-transparent compression tights SMARTLEG®:	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from	Not more than 10	The edge is in the form of dense elastic band / the product is

Color Mysterieuse, Naturelle, Lumineuse, Superbe, Delicieuse, Imperiale, Audacieuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal, Long				more than 300		Breaking elongation before washing)		securely held on the patient's body
3. Transparent compression tights SMARTLEG [®] : Color: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal, Long	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of dense elastic band / the product is securely held on the patient's body
4. Semi-transparent compression stockings SMARTLEG [®] : Color: Mysterieuse, Ardoise, Saphir, Ambre, Opale, Poudre Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of a lace elastic band with a silicone strip / the product is securely held on the patient's body
5. Transparent compression stockings SMARTLEG [®] : Colors: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Sizes: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of a lace elastic band with a silicone strip / the product is securely held on the patient's body
6. Semi-transparent compression knee-socks SMARTLEG [®] : Colors: Mysterieuse, Ardoise, Saphir, Ambre, Opale, Poudre Sizes: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of dense elastic band / the product is securely held on the patient's body
7. Transparent compression knee-socks SMARTLEG [®] : Colors: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Sizes: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal	Not more than 20	Not less than 120	Not less than 30	Not less than 120, but not more than 300	Not less than 68,6 (7)	Not less than 120 (not more than 20 % from Breaking elongation before washing)	Not more than 10	The edge is in the form of dense elastic band / the product is securely held on the patient's body

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Table 6 Technical characteristics of knitted fabric threads

Name of the type of the medical device	Characteristics of the thread	
	Knitted thread	Weft yarn
<p>1. Compression tights opaque SMARTLEG® Color: Splendide, Prodigieuse, Reveuse, Captivante Size: T0, T1, T1+, T2, T2+, T3, T3+, T4, Length: Court, Normal, Long</p>	<p>Elastane 22 dtex, coated with a double layer of polyamide 6.6 1/60/68 dtex, uncolored</p>	<p>Elastane 330 dtex, coated with a double layer of polyamide 6.6 1/22/7 dtex, uncolored</p>
<p>2. Compression tights semi-transparent SMARTLEG®: Color: Mystérieuse, Naturelle, Lumineuse, Superbe, Delicieuse, Imperiale, Audacieuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal, Long</p>	<p>Elastane 17 dtex, coated with a single layer of polyamide 6.6 1/42/46 dtex, uncolored</p>	<p>Elastane 330 dtex, coated with a double layer of polyamide 6.6 1/22/7 dtex, uncolored</p>

Name of the type of the medical device	Characteristics of the thread	
Compression stockings semi-transparent SMARTLEG®, Color: Mysterieuse, Ardoise, Saphir, Ambre, Opale, Indre Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal Compression knee-socks semi-transparent SMARTLEG®, Color: Mysterieuse, Ardoise, Saphir, Ambre, Opale, Indre Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal		
Compression tights transparent SMARTLEG®, Color: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal, Long Compression stockings transparent SMARTLEG®, Color: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal Compression knee-socks transparent SMARTLEG®, Color: Irrésistible, Delicate, Ravissante, Radieuse, Rayonnante, Fabuleuse Size: T0, T1, T1+, T2, T2+, T3, T3+, T4 Length: Court, Normal, Long	<i>Elastane 22 dtex, coated with polyamide 6.1/22/34 dtex and polyamide 6.6 1/16/10 dtex, uncolored</i>	<i>Elastane 285, coated with a double layer of elastane 22 dtex, uncolored</i>

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2.1.1 Dependently on the potential risk of the medical application, in accordance with Annex 9 of Directive about medical devices 93/42/EEC, medical device “Compression medical stockings SMARTLEG®” is associated with Class 1 potential risk.

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2.1.2 The medical device is certified according to ASQUAL and belongs to the II compression class.

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2.1.3 According to the classification of the standard RAL-GZ 387, the product belongs to compression class I.

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Supply package of compression tights includes:

1. Compression tights of one design style, one color, one size – 1 item.
2. Individual (customer) package – 1 item.

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Supply package of compression stockings includes:

1. Compression stockings of one design style, one color, one size – 1 pair.
2. Individual (customer) package – 1 item.

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Supply package of compression knee-socks includes:

1. Compression knee-socks of same design style, same color, same size – 1 pair.
2. Individual (customer) package – 1 item.

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Laboratories INNOTHERA guarantees Medical compression hosiery SMARTLEG® quality, safety and efficacy in conditions of use in accordance with instruction for use and storage and transportation conditions following, but it doesn't responsible for the device application different from its intended use.

The warranty period for preserving the compression properties, according the recommendations for use, is 6 months from the date of sale.

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Medical device labelling is performed by means of putting of the main data to the device, customer and group (transportation) packaging. Symbols of ISO 15223-1:2016 standard ensuring data uniqueness and symbols of textile devices caring ISO 3758:2012 standard are used for the labelling.

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Word «SMARTLEG» is knitted immediately at the medical device foot under the toes.

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Following information should be stated on the customer package label:

- medical device name – «Medical compression hosiery»;
- trademark symbol;

- manufacturer's address and name;
- production facilities address;
- medical device indications;
- medical device contraindications;
- medical device composition;
- medical device action principle;
- precautions;
- link for getting information on size range and recommendations for selection of the proper size of the medical device;
- short instructions on the method of putting on medical compression hosiery;
- application conditions;
- short instructions for the medical device care;
- symbol «Maximal washing temperature 40 °C, soft regimen»;
- symbol «Bleaching with any oxidizing or bleaching substances is prohibited»;
- symbol «Don't use tumble dry»;
- symbol «Ironing is prohibited»;
- symbol «Cleaning is prohibited».

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Additional label, containing following information, should be glued to the customer's packaging:

- medical device type name;
- compression class;
- size and length;
- medical device color;
- symbol «Batch code» and number assigning;
- symbol of conformity with EU Directive;
- symbol for article (nomenclature code) and the code;
- symbol «production date» and data in format DD.MM.YYYY;
- EAN-13 system bar-code;
- QR- code.

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Label with following information should be glued to the box:

- medical device name;
- medical device design style name;
- manufacturer's logotype;
- symbol «legal Manufacturer» and manufacturer's name;
- production facilities address;
- symbol «Production date» and production date in DD.MM.YY format;
- symbol «Batch code» and number assigning;
- storage and transportation conditions;
- quantity of supplied units in a package;
- medical device color;
- size and length;
- registration certificate number and date;
- packaging gross weight;
- symbol of conformity with EU Directive;
- symbol «Keep dry»
- symbol «Keep far from the sunlight»

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The artworks for consumer packaging with labelling, additional label and the label for grouping (transportation) package are presented in the Annex A to the document.

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Table 7 – Symbols used for the medical device labelling

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Symbol	Description
	Batch code
	EU label – symbol of conformity to Directive 93/42/EEC requirements medical device class risk 1
	Production date
	Legal Manufacturer, according the European rule (Responsible of putting the product on the market)
	Catalog number
	Keep far from the sunlight

Symbol	Description
	Keep dry
	Maximal washing temperature 40 °C, soft regimen
	Don't use tumble dry
	Ironing is prohibited
	Cleaning is prohibited
	Bleaching with any oxidizing or bleaching substances is prohibited

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Compression tights, stockings and knee-socks are folded several times and packed into a heat-sealed transparent low density 30 µm polyethylene bag FPBP-30, France, with opened upper edge and overall dimensions, namely “length x width (mm)”, (195 x 150) ± 5.

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The medical device in the individual package is placed into a customer's package – carton box, assuring the medical device protection from mechanical and climate factors during its transportation and storage.

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Carton box overall dimensions, namely “length x width x height” (mm), are:

- for compression tights, stockings and socks (180 x 132 x 45) ± 5;

Mass of the product in consumer pack is not more than 250 g.

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Additional flap, opening along the perforation line, is considered at the front side of the medical device customer's package construction. Additional information on the medical device application is shown at the reverse side of this flap.

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Devices are grouped into a transport package based on the batch numbers (devices of one type, size, length, color).

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After this, medical devices inside of the customer's package are distributed into a group package: 24 units. It is a differently designed corrugated cardboard box, assuring the medical device protection from mechanical and climate factors during its transportation and storage and loading-and-unloading comfort.

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One unit of the operational documentation should be added to the box.

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Box flaps are fixed with a polyethylene tape with adhered layer to avoid opening without integrity disturbance.

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Transportation package dimensions namely "length x width x height" (mm), are: 404 x 371 x 182. The gross weight of the transport packaging is not more than 5.5 kg.

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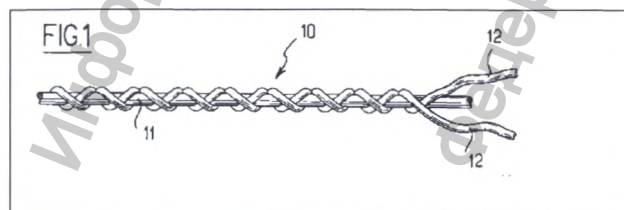
Medical compression hosiery SMARTLEG® is produced from the textile yarns in complex with elastic yarns, braided with 1 or 2 inverse layers.

Braiding includes covering of a yarn, called "standard thread", with one or several yarns called "coverage" to obtain elastic yarn.

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Knitting density depends on the links number per assumed length. Commonly it is determined in two directions: based on links courses and based on links columns (horizontal density (Hd) and vertical density (Vd), respectively).

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Links columns quantity (Dv) in devices of any types increases in accordance with the device size increase:

- sizes 0, 1 and 1+ contain 368 columns
- sizes 2, 2+ contain 420 columns
- sizes 3, 3+, 4 contain 472 columns

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Therefore, links columns quantity for 1 cm is constant independently from the size ensuring necessary superficial density of any design type of the medical device.

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Different parts of the device contain different quantity of links courses due to the link size increase in direction from the foot to the device buffet. In general, consideration this provides necessary compression level during respective part of the device elongation up to the needed length.

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For all design types of Medical compression hosiery SMARTLEG®:

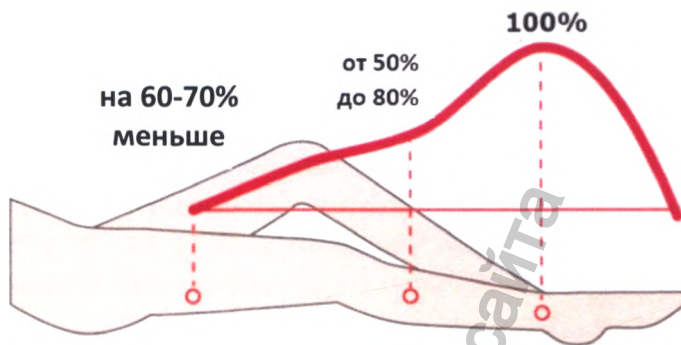
a) minimal diameter difference at maximal load 2 daN comprises:

- 30 % between ankle and calf;
- 50 % between ankle and thigh.

b) the compression between the ankle and hip is reduced as follows:

- residual pressure between 50% and 80% from the pressure at the ankle;

- residual pressure at the level of the upper third of the thigh is 60 - 70% less than the pressure created at the level of the ankle.
- the pressure at the thigh is lower than the pressure at the calf, which is lower than the pressure at the ankle.

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Potential users of the medical device are patients corresponding to following criteria:

- with venous insufficiency or varicose veins;
- after venous surgery;
- working for a long time in upright or, vice versa, in sitting;
- with low active mode of life;
- with overweighting;
- increase of low extremities load due to the pregnancy;
- long-time travelers or users of long-time flights.

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Before the medical device use patient should contact with his/her physician to get the recommendations about necessary version of the product model dependent on the venous system pathological process localization:

- commonly socks are recommended in cases of low vein-venous shunting or when the popliteal leg part is injured;
- stockings are recommended in cases of high vein-venous shunting or when the varicose is localized in thigh or popliteal areas;
- tights are commonly used in cases of varicose veins in iliac area.

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IMPORTANTLY! It is necessary to assure before the use that compression class is in compliance with the patient's needs.

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- Legs heavyness and pain, tiredness, legs edema after long staying or sitting (for example during the flights). Including cases of absense of external signs of veins injure;
- Starburst veins or vascular spider (telangiectasia) at the skin of the legs;
- Varicose veins;
- After sclerotherapy and leg veins surgery;
- During pregnancy and 6 weeks at postdelivery period (6 month after caesarean section)*;

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Use of Medical compression hosiery SMARTLEG® medical device doesn't influence the process of mutual dependence and links of medical devices.

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Medical compression hosiery SMARTLEG® may be used daily during determined time period. To determine respective time period patient should consult with physician.

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Mentioned devices should be put on after wakening in relaxed position, as commonly used stockings or tights. It should be worn all day long and put out only before the sleep or the rest.

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IMPORTANTLY! During compression hosiery wearing patient should use shoes with not more than 5 cm heels to avoid possible aggravation of existing venous problems.

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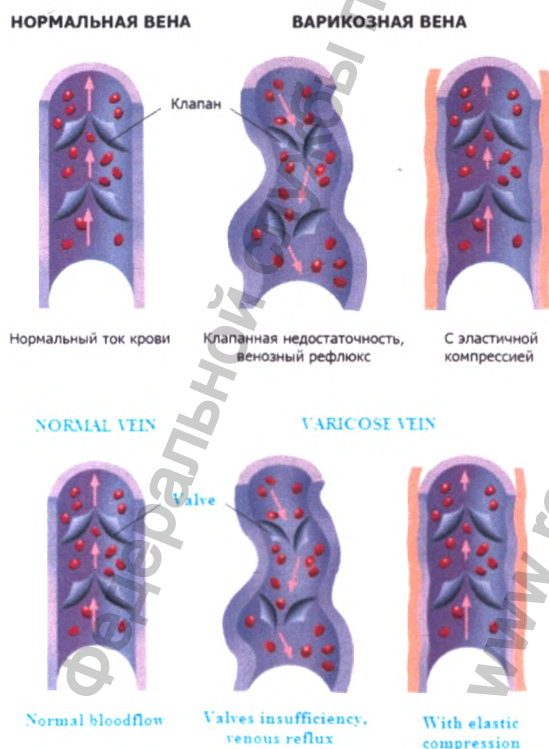
Compression stockings exerts decreasing pressure from the ankle to the leg upper part to facilitate venous blood return flow rate increase and microcirculation and lymphatic drainage amelioration.

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Due to elastic compression additional carcass is created, supporting veins and protecting them from excessive stretching. Constant vein compression leads to significant narrowing of their lumens, reducing the risk of thrombosis development because veins valves maintenance allows blood congestion avoidance.

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Legs soft tissues compression prevents water congestion, decreases tiredness and alleviates muscle pains.

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Put on the compression hosiery every day in the morning, wear it during the day and put out before the sleep.

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If it is necessary to put out the device in the middle of the day, lie down for 15 minutes with the legs position upper the body before its subsequent putting on.

Wash the medical device after each use so as it completely restored its properties.

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Wear the medical device not more than 6 months (it is a guaranteed shelf life of its compression properties) to get always the maximal effect.

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Way of the medical device putting on

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1. Turn the device inside out from upper part to the heel but keeping intact the toe part.

2-3. With two hands spread the toe part through the foot and carefully span the device to the heel (check if the knitted heel area of the device corresponds to your heel).

4-5. Unroll the device evenly through the leg, stretching places of folds or irregular position of the stockings*

*For tights: repeat the actions 1-5 at another leg.

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The medical device is intended for use at ambient temperatures from minus 30 °C to plus 40 °C, with a relative air humidity of not higher than 95% (at a temperature of 25 °C).

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During the device wearing:

- avoid sports with high load for calf muscles (tennis, running, squash...);
- try not to spend a lot of time in staying or sitting position;
- prefer shoes with 3-5 cm heels instead of high heels or heels absence.

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Useful recommendations for veins problems prevention:

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Perform regular physical trainings for your calf muscles: walking, riding a bicycle, swimming or simple exercises for calf muscles stretching and tension. Avoid long legs overheating (bath, warm floor, sauna...).

Try to balance your dietary regimen and control your weight.

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- The main condition for the device use beginning should be absence of its mechanical damages and its integrity;
- Wash the device before the first use;
- It is not recommended to use compression stockings with wounds or injured skin;
- In cases of cutaneous infection: treat the infection first, before putting on the compression medical device.

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IMPORTANT! In case of obliterative arteriopathy of the lower extremities with ankle-brachial index > 0.6 and < 0.9, evolved peripheral neuropathy, oozing dermatosis or eczematous dermatosis, a very regular medical monitoring of the benefit/risk balance has to be done.

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IMPORTANTLY! During the pregnancy, weight gaining or loss, edema disappearance etc. contact with specialists to precise the device size or type.

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This medical device is not sterile and should not be sterilized before or in process of use.

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It is recommended to wash the Medical compression hosiery SMARTLEG® after each use/ Washing is necessary to restore the properties of compression product which was some elongated after the application and to remove from it cosmetics and sweat traces.

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Washing regimen:

Water temperature – not more than 40 degrees

Detergent: soft (for delicate fabrics wash)

Washing regimen – hand washing or machine delicate washing (without wringing).

Devices should be well rinsed, wrapped with a towel and slightly ringed. Then it should be spread on the dry towel far from the sunlight and heaters. The device should not be ironed. Its chemical cleaning is prohibited.

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IMPORTANTLY! Devices with silicon insertions should not be presoaked with detergent for a long time.

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IMPORTANTLY! It is not recommended to use bleachers, active chemicals or fabric softeners during washing – this may injure the material and reduce its compression characteristics.

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Packaged medical devices may be transported by any transport in conditions of their protection from mechanical injuries, atmospheric precipitations and aggressive medias influence in accordance with the shipping rules applicable for given transport type.

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Wooden boxes and containers should be lined with polyethylene or any other moisture-protective material.

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The medical device may be transported at the temperature from minus 50 °C to plus 50 °C at relative humidity not more than 80% at 25 °C.

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The devices should be stored in protected from direct sunlight and atmospheric influences covered storage facilities.

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Up to realization this medical devices should be kept at shelves in customer or group packaging at the temperature from plus 5 °C to plus 45 °C at relative humidity not more than 70%.

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Distance between the floor and the lowest part of the shelf and between internal and external walls and the device should be not less than 0.2 m; from heaters to the device – not less than 1 m. between the shelves – not less than 0.7 m.

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Shelf life and storage period: not limited.

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The medical device construction doesn't consider any technical service or device elements reparation during all period of its application.

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Medical compression hosiery SMARTLEG® corresponds with determined in current legislation environment requirements for such medical devices types.

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The medical device construction and packing materials doesn't contain or emit during the storage, use or disposal any dangerous substances, which could damage environment, health or genetic fund of people.

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The medical device with injuries should not be used and should be disposed/ mentioned above medical devices and used medical devices should be disposed as Class A waste, namely epidemically safe waste, with composition approximated to solid domestic wastes.

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In case of question related to the Medical compression hosiery SMARTLEG® use contact with authorized representative of the medical device manufacturer Laboratoires INNOTHERA:

Limited liability company (under the laws of the Russian Federation) "INNOTECH"
(LLC "INNOTECH")

71, Sadovnicheskaya naberezhnaya, Moscow, 115035, Russia

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