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DICOM Service

Storage Print Worklist MPPS StorageCommitment Query/Retrieve HL7Query

Configure the New Service

Device: SETUP Service Name: SETUP-MPPS AE Title: Port: 104

Maximum Retries: 3 Interval Time(Sec): Timeout(Sec): 15

Add Cancel

Service List

	Device	Service Name	AE Title	Port	Default
1	SETUP	SETUP-MPPS		104	N

Delete Default Verify No service selected!

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Query/Retrieve mindray

**Server and Service**

Source: SETUP-QueryRetrieve  
Destination: Local Host  
Level: STUDY

**Query**

Patient ID:  Patient Name:   
Accession #:  Exam Date: DD/MM/YYYY To DD/MM/YYYY  
ModalitiesInStudy:

Query  Clear

Patient(Source) 0 records are listed.

Patient ID	Name	Accession #	Exam Description	Exam Date/Time	Gender	DOB
------------	------	-------------	------------------	----------------	--------	-----

Cancel

Patient(Destination) 0 records are listed.

Patient ID	Name	Accession #	Exam Description	Exam Date/Time	Content	Gender	DOB
------------	------	-------------	------------------	----------------	---------	--------	-----

Exit

03:26:12

**Option**  
// OPTION  
Keyword

Title	Status
<input checked="" type="checkbox"/> DICOM OB/GYN SR	Installed
<input type="checkbox"/> DICOM Cardiac SR	Installed
<input type="checkbox"/> DICOM Vascular SR	Installed
<input type="checkbox"/> DICOM Breast SR	Installed
<input type="checkbox"/> DICOM Abdomen SR	Installed
<input type="checkbox"/> DICOM Small Parts SR	Installed
<input type="checkbox"/> Abdomen/General Package	Installed
<input type="checkbox"/> Obstetrics Package	Installed
<input type="checkbox"/> Smart OB	Installed
<input type="checkbox"/> Smart NT	Installed
<input type="checkbox"/> Smart Fetal HR	Installed
<input type="checkbox"/> Gynecology Package	Installed
<input type="checkbox"/> Smart FLC	Installed
<input type="checkbox"/> Cardiology Package	Installed

Comment  
Condition: DICOM basic unit installed first.  
Purpose: The ultrasound system transfers the measurements obtained from OB-GYN studies to storage devices.

Trial      Install  
Select All      Batch Install  
Promote      Uninstall  
Auto Config

Caution: To trial, install and uninstall a component will cause the system shut down

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// OPTION

Keyword

Title	Status
<input checked="" type="checkbox"/> DICOM Vascular SR	Installed
<input type="checkbox"/> DICOM Breast SR	Installed
<input type="checkbox"/> DICOM Abdomen SR	Installed
<input type="checkbox"/> DICOM Small Parts SR	Installed
<input type="checkbox"/> Abdomen/General Package	Installed
<input type="checkbox"/> Obstetrics Package	Installed
<input type="checkbox"/> Smart OB	Installed
<input type="checkbox"/> Smart NT	Installed
<input checked="" type="checkbox"/> Smart Fetal HR	Installed
<input type="checkbox"/> Gynecology Package	Installed
<input type="checkbox"/> Smart FLC	Installed
<input type="checkbox"/> Cardiology Package	Installed
<input type="checkbox"/> Small Parts Package	Installed
<input type="checkbox"/> Urology Package	Installed

Comment

Condition: DICOM basic unit installed first.  
Purpose: The ultrasound system transfers the measurements obtained from Vascular studies to storage devices.

Trial      Install

Select All      Batch Install

Promote      Uninstall

Auto Config

Caution: To trial, install and uninstall a component will cause the system shut down

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// OPTION

Keyword

Title	Status
<input checked="" type="checkbox"/> DICOM Cardiac SR	Installed
<input type="checkbox"/> DICOM Vascular SR	Installed
<input type="checkbox"/> DICOM Breast SR	Installed
<input type="checkbox"/> DICOM Abdomen SR	Installed
<input type="checkbox"/> DICOM Small Parts SR	Installed
<input type="checkbox"/> Abdomen/General Package	Installed
<input type="checkbox"/> Obstetrics Package	Installed
<input type="checkbox"/> Smart OB	Installed
<input checked="" type="checkbox"/> Smart NT	Installed
<input type="checkbox"/> Smart Fetal HR	Installed
<input type="checkbox"/> Gynecology Package	Installed
<input type="checkbox"/> Smart FLC	Installed
<input type="checkbox"/> Cardiology Package	Installed
<input type="checkbox"/> Small Parts Package	Installed

Comment

Condition: DICOM basic unit installed first.  
Purpose: The ultrasound system transfers the measurements obtained from Cardiac studies to storage devices.

Trial      Install

Select All      Batch Install

Promote      Uninstall

Auto Config

Caution: To trial, install and uninstall a component will cause the system shut down

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The screenshot shows the Mindray Setup application with the 'DICOM/HL7' section selected in the left sidebar. The main window displays a list of DICOM options, each with a checkbox and a status of '90 days remaining in the trial period'. The 'DICOM Breast SR' option is checked and highlighted. A 'Comment' dialog box is open, displaying the following text:

Condition: DICOM basic unit installed first.  
Purpose: The ultrasound system transfers the measurements obtained from Breast studies to storage devices.

Below the list, there is a 'Select All' button and a 'Caution: To trial, install and uninstall a component will cause the system shut down' message. At the bottom of the dialog, there are 'OK' and 'Cancel' buttons. The system tray at the bottom right shows the time as 04:00:36.

Option	Status
<input type="checkbox"/> DICOM Basic	90 days remaining in the trial period
<input type="checkbox"/> DICOM Worklist	90 days remaining in the trial period
<input type="checkbox"/> DICOM Query/Retrieve	90 days remaining in the trial period
<input type="checkbox"/> DICOM MPPS	90 days remaining in the trial period
<input type="checkbox"/> DICOM OB/GYN SR	90 days remaining in the trial period
<input type="checkbox"/> DICOM Cardiac SR	90 days remaining in the trial period
<input type="checkbox"/> DICOM Vascular SR	90 days remaining in the trial period
<input checked="" type="checkbox"/> DICOM Breast SR	90 days remaining in the trial period
<input type="checkbox"/> DICOM Abdomen SR	90 days remaining in the trial period
<input type="checkbox"/> DICOM Small Parts SR	90 days remaining in the trial period
<input type="checkbox"/> Obstetrics Package	90 days remaining in the trial period
<input type="checkbox"/> Smart OB	90 days remaining in the trial period
<input type="checkbox"/> Pediatrics Package	90 days remaining in the trial period
<input type="checkbox"/> Gynecology Package	90 days remaining in the trial period
<input type="checkbox"/> Abdomen/General Package	90 days remaining in the trial period

**Option**  
// OPTION  
Keyword

Title	Status
<input checked="" type="checkbox"/> DICOM Small Parts SR	Installed
<input type="checkbox"/> Abdomen/General Package	Installed
<input type="checkbox"/> Obstetrics Package	Installed
<input type="checkbox"/> Smart OB	Installed
<input type="checkbox"/> Smart NT	Installed
<input type="checkbox"/> Smart Fetal HR	Installed
<input type="checkbox"/> Gynecology Package	Installed
<input type="checkbox"/> Smart FLC	Installed
<input type="checkbox"/> Cardiology Package	Installed
<input type="checkbox"/> Small Parts Package	Installed
<input type="checkbox"/> Urology Package	Installed
<input type="checkbox"/> Vascular Package	Installed
<input type="checkbox"/> Pediatrics Package	Installed
<input type="checkbox"/> Nerve Package	Installed

Comment  
Condition: DICOM Basic unit installed first.  
Purpose: The ultrasound system transfers the measurements obtained from Small Parts studies to storage devices.

Trial      Install  
Select All      Batch Install  
Promote      Uninstall  
Auto Config

Caution: To trial, install and uninstall a component will cause the system shut down

OK      Cancel

**Setup** mindray

System Exam Mode **Adult ABD** Export MeasDef Export Custom Import Custom

ExamMode Measure **Caliper** Report

Measure Measure Package **Abdomen** Advanced

Comment 2D M Doppler

iWorks Available Items Measure Sequence **None**

Stress Echo ABD Measurement Selected Items

DICOM/HL7 Aorta Bif Aorta Aneurysm Status Shunt Diam Portal V Diam M Portal V Diam Splenic V Diam PS Conflnc Diam Renal V Diam SMV Diam IMV Diam CHD

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**Setup** mindray

System Exam Mode **OB1** Export MeasDef Export Custom Import Custom

ExamMode Measure Caliper Report

Measure Measure Package **OB-early** Advanced

Comment 2D M Doppler

iWorks Available Items Measure Sequence **None**

Stress Echo OB Measurement Selected Items

DICOM/HL7

Network

Print

Maintenance

Security

About

Available Items	Selected Items
GS	GS
YS L	YS L
CRL	CRL
NT	BPD
BPD	HC
OFD	AC
HC	HUM
AC	FL
FL	NT
TAD	IT
APAD	NBL

Buttons: Add Study, Property, Default, Move Up, Move Down, OK, Cancel

System tray: 08-01-2021 11:11:20 100%

**Setup** mindray

System Exam Mode GYN Export MeasDef Export Custom Import Custom

ExamMode Measure Caliper Report

Measure Measure Package GYN Advanced

Comment 2D M Doppler

iWorks Available Items Measurement Measure Sequence None

Stress Echo GYN Measurement

DICOM/HL7

Network

Print

Maintenance

Security

About

**Available Items**

- UT L
- UT H
- UT W
- Endo
- Cervix L
- Cervix H
- Cervix W
- Ovary L
- Ovary H
- Ovary W
- Follicle1 L

**Selected Items**

- GYN
- Elasticity
- Uterus
- Uterine Cervix
- Ovary
- Follicle
- Fibroid 1
- Fibroid 2
- Fibroid 3
- Ovarian Cyst 1
- Ovarian Cyst 2
- Ovarian Cyst 3

**Buttons:** Add Study, Property, Default, Move Up, Move Down, OK, Cancel

08-01-2021 13:29:57 100%

**Setup** mindray

System Exam Mode **Adult Cardiac** Export MeasDef Export Custom Import Custom

ExamMode Measure **Caliper** Report

Measure Measure Package **Cardiac Meas** Advanced

Comment 2D **M** Doppler

iWorks Available Items Measure Sequence **None**

Stress Echo CARD Measurement Selected Items

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
The screenshot shows the 'Setup' dialog box for a Mindray ultrasound system, specifically for the 'Thyroid' exam mode. The interface is dark-themed and includes a sidebar on the left with various system settings like System, ExamMode, Measure, Comment, iWorks, Stress Echo, DICOM/HL7, Network, Print, Maintenance, Security, and About. The main area is divided into several sections: 'Exam Mode' (Thyroid), 'Measure Package' (Thyroid Meas), and 'Available Items' (SMP, Measurement). The 'Available Items' list includes Thyroid L, Thyroid H, Thyroid W, Isthmus H, and three THY Nodule Strain and THY Nodule Elas. items. The 'Selected Items' list shows a 'Thyroid Meas' folder containing Elasticity, Thyroid L, Thyroid W, Thyroid H, Isthmus H, and six Thyroid Nodule folders (1-6). Navigation buttons like '>', '>>', '<<', '<', 'New', 'Edit', and 'Delete' are present between the lists. On the right, there are buttons for 'Export MeasDef', 'Export Custom', 'Import Custom', 'Advanced', 'Add Study', 'Property', 'Default', 'Move Up', and 'Move Down'. The 'OK' and 'Cancel' buttons are at the bottom right. The system tray at the very bottom shows the date '08-01-2021', time '13:57:03', and '100%' zoom.

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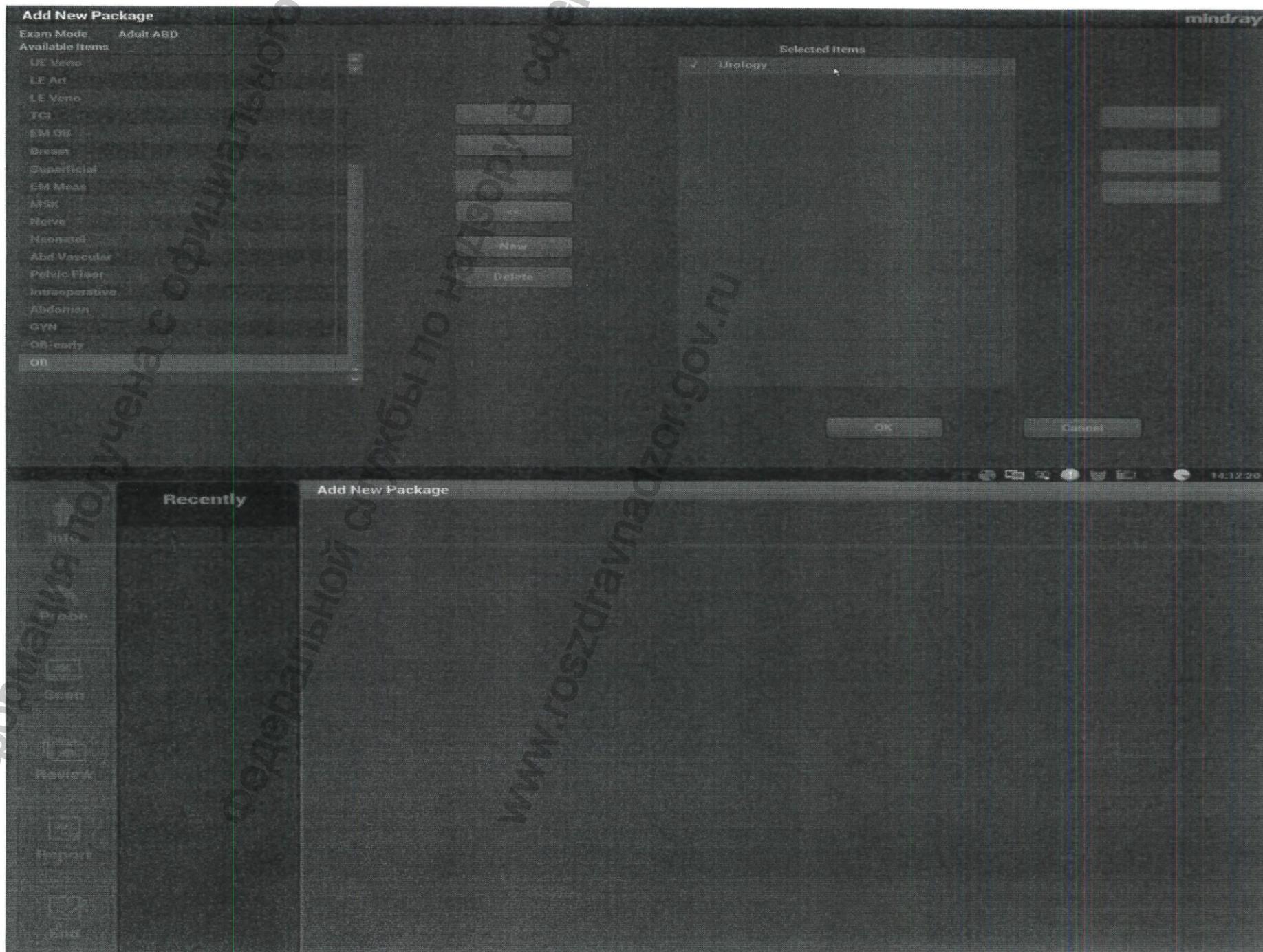**Setup** mindray

System Exam Mode Carotid Export MeasDef Export Custom Import Custom

ExamMode Measure Caliper Report

Measure Measure Package Carotid Advanced

Comment 2D M Doppler

iWorks Available Items Measure Sequence Repeat

Stress Echo VAS Measurement Selected Items

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The screenshot shows the 'Setup' dialog box for the 'Nerve' package in the Mindray software. The interface is dark-themed and includes a sidebar on the left with various system settings like System, ExamMode, Measure, Comment, iWorks, Stress Echo, DICOM/HL7, Network, Print, Maintenance, Security, and About. The main area is titled 'Exam Mode Nerve' and contains several sections: 'Measure Package' (set to 'Nerve'), 'Available Items' (with 'Nerve' and 'Measurement' dropdowns), 'Measure Sequence' (with a 'Repeat' dropdown), and 'Selected Items' (a list of measurement tools including Distance, Trace, Ellipse, Smart Trace, Angle(2L), Angle(3P), Volume, Ratio(D), and Ratio(A)). Action buttons like 'Add Study', 'Property', 'Default', 'Move Up', and 'Move Down' are positioned to the right of the 'Selected Items' list. At the top right, there are buttons for 'Export MeasDef', 'Export Custom', and 'Import Custom'. An 'Advanced' button is also present. At the bottom right, there are 'OK' and 'Cancel' buttons. The system tray at the very bottom shows the date '08-01-2021', time '13:48:04', and '100%' zoom level.

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**Setup** mindray

System | Exam Mode | EM Vascular | Export MeasDef | Export Custom | Import Custom

ExamMode | Measure | Caliper | Report

Measure Package | EM Meas | Advanced

Comment | 2D | M | Doppler

iWorks | Available Items | Measurement | Measure Sequence | Repeat

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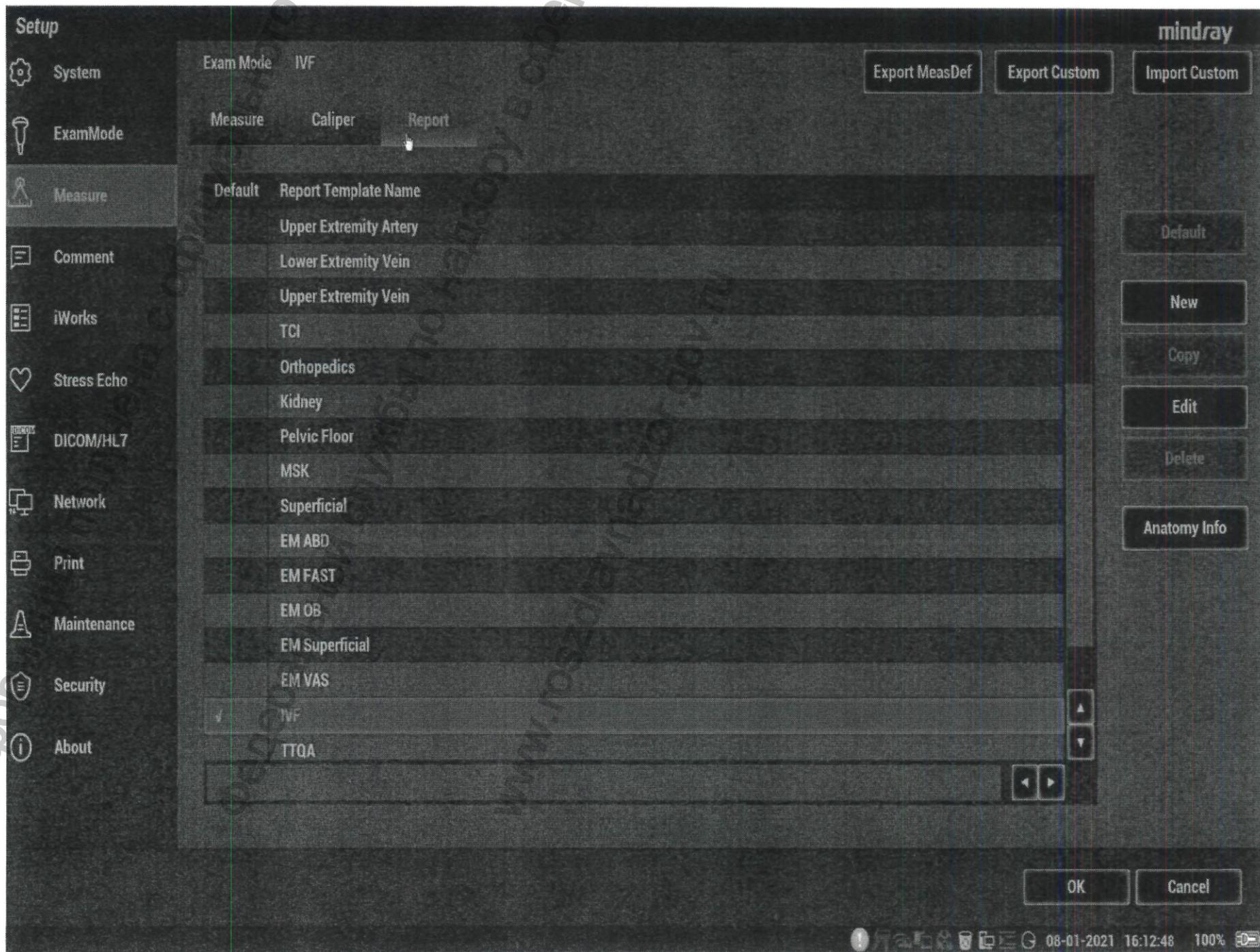
Stenosis 2 Stenosis 3 Stenosis 4

60° 90° 120° 150° 180° ANT LAT POST

Info Probe Scan Review Report End Wall Far Library Carotid Adjust Frame Side Right

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Median  
IQR  
IQH/IQL  
Average  
STD  
STD/Avg  
Depth 1.80 cm

ReCapture

BodyMark  
Capturing [Update] ReCapture. [Save Image] Send current elasticity measurement results to report when saving image.

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**Setup** mindray

System    Exam Mode Pelvic Floor    Export MeasDef    Export Custom    Import Custom

ExamMode    Measure Caliper Report

Measure    Measure Package Pelvic Floor    Advanced

Comment    2D M Doppler

iWorks    Available Items    Measure Sequence None

Stress Echo    GYN Measurement    Selected Items

DICOM/HL7

Network

Print

Maintenance

Security

About

Available Items: GYN Measurement

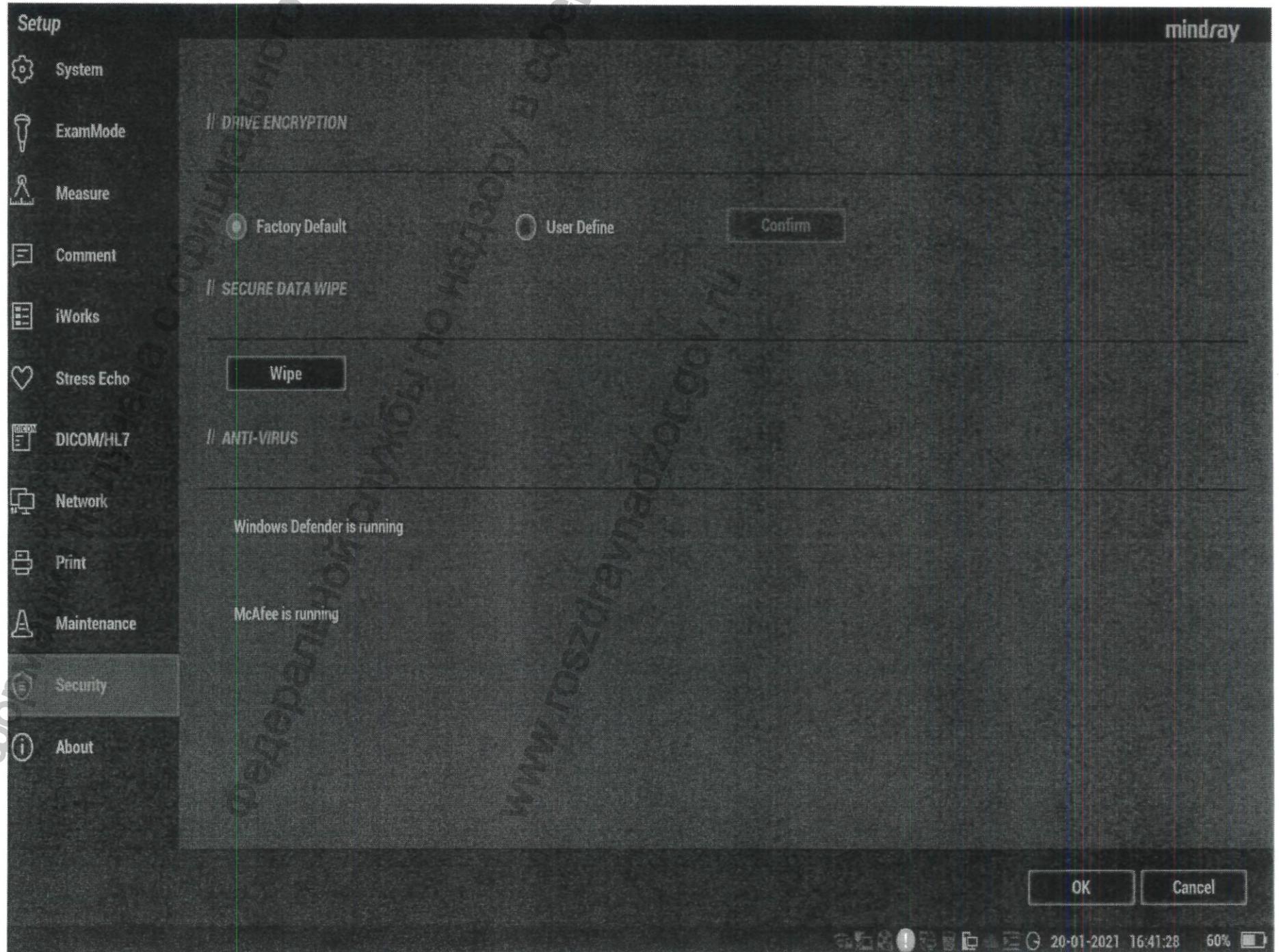
- UT L
- UT H
- UT W
- Endo
- Cervix L
- Cervix H
- Cervix W
- Ovary L
- Ovary H
- Ovary W
- Follicle1 L

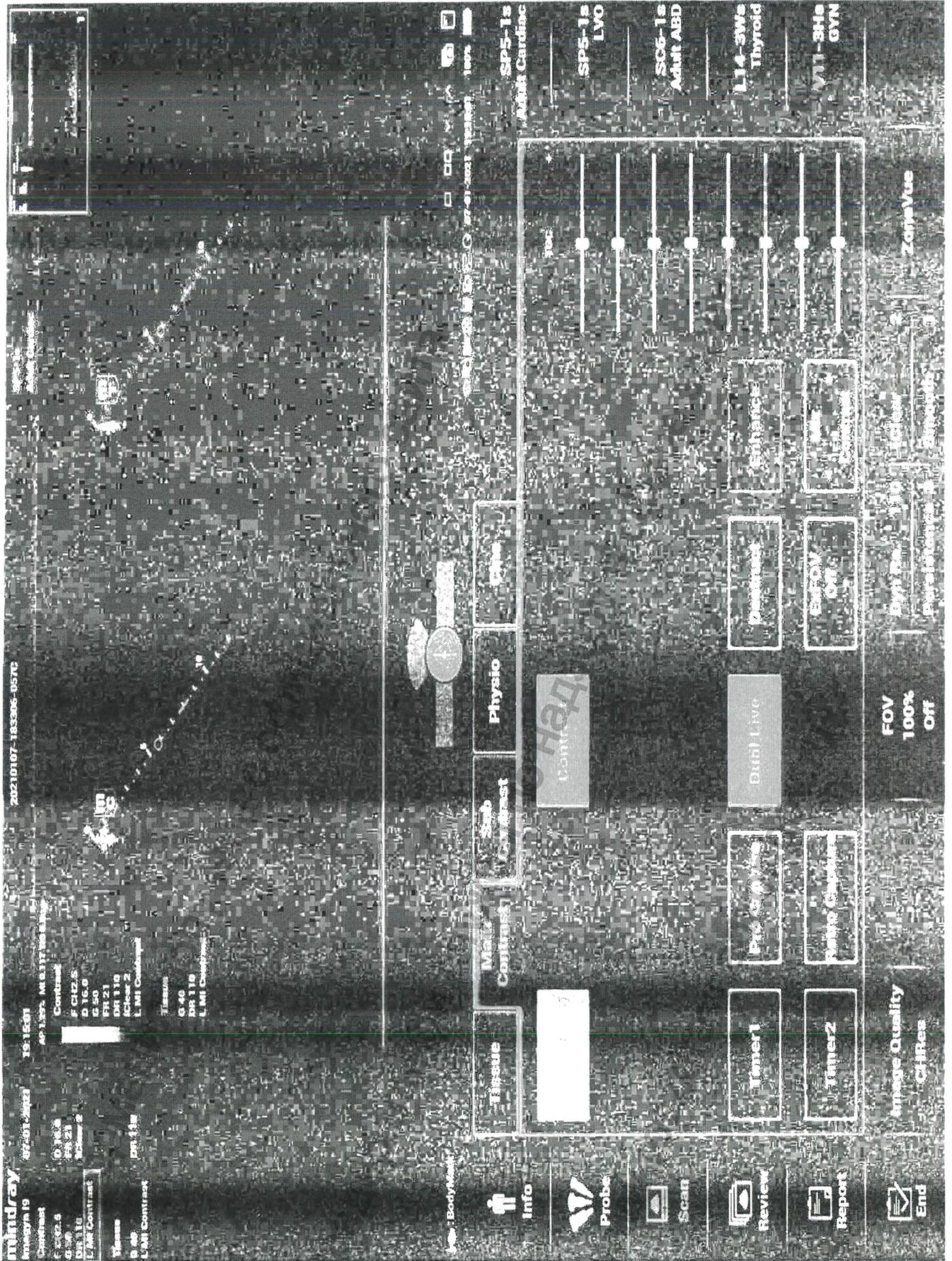
Selected Items: Pelvic Floor

- Mean DWT
- Residual Urine
- BSD(R)
- BSD(Va)
- UTA(R)
- RVA(R)
- UTA(Va)
- RVA(Va)
- BPW-SP Dist.(Va)
- Cx-SP Dist.(Va)
- RA-SP Dist.(Va)


Buttons: Add Study, Property, Default, Move Up, Move Down, OK, Cancel

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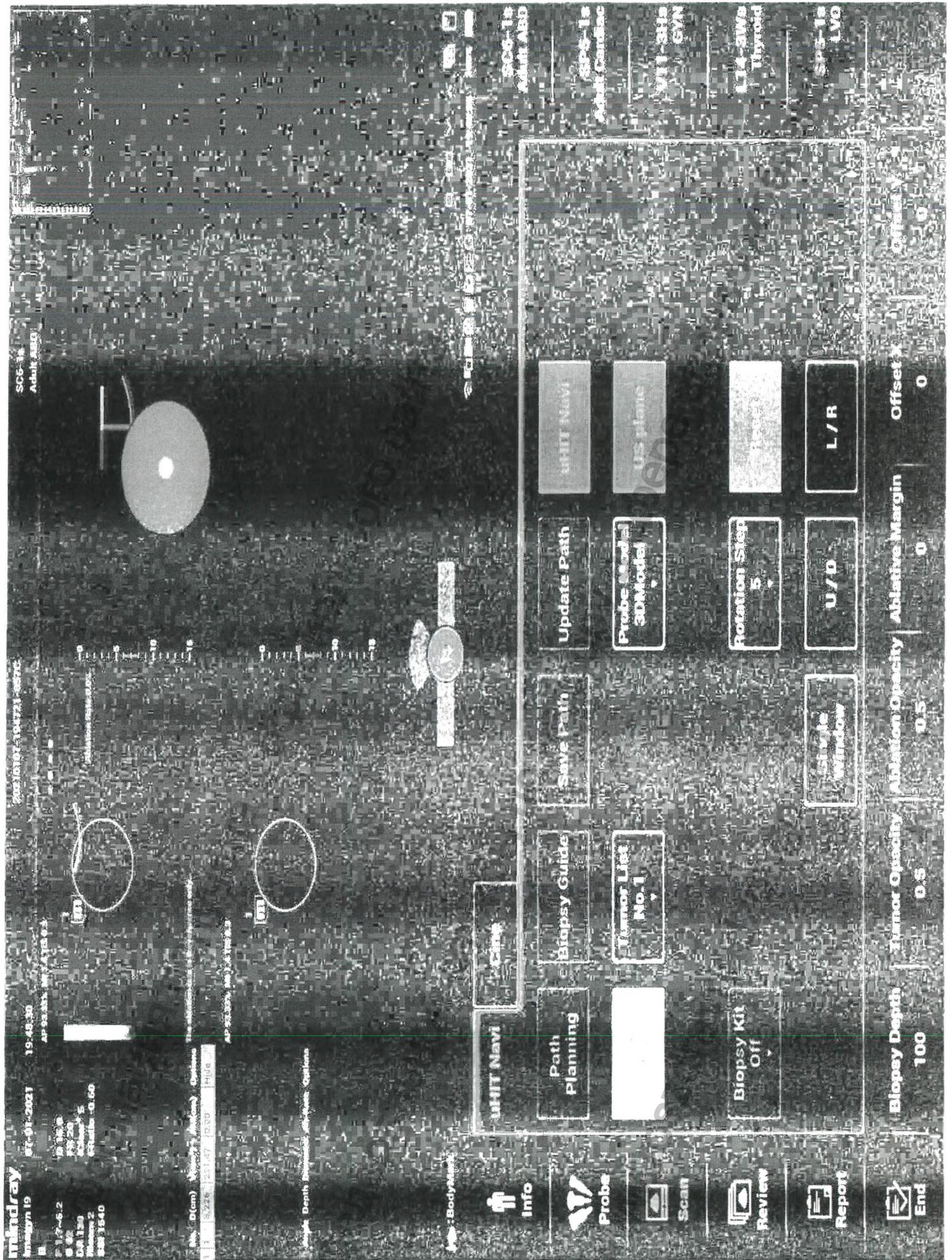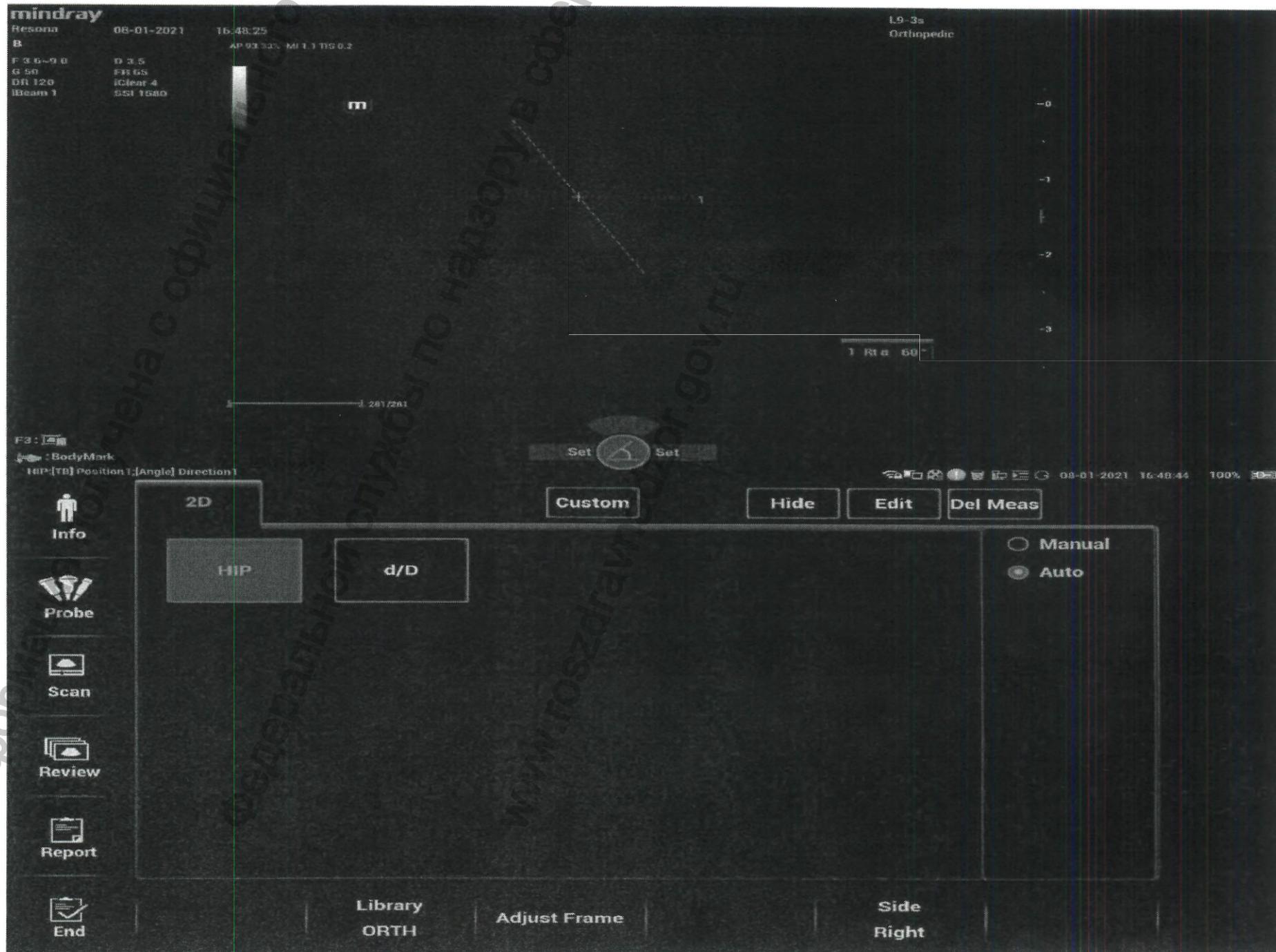
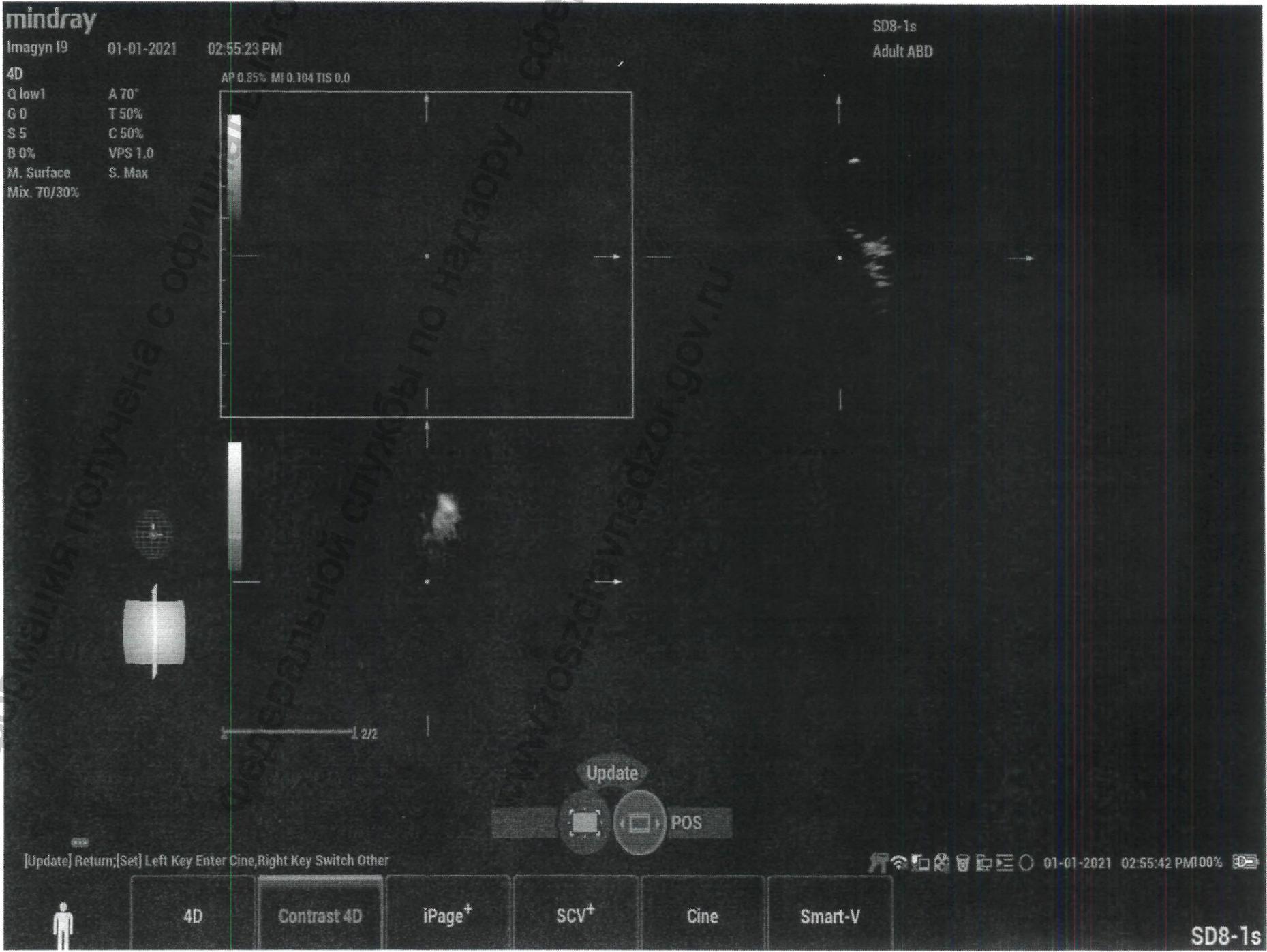
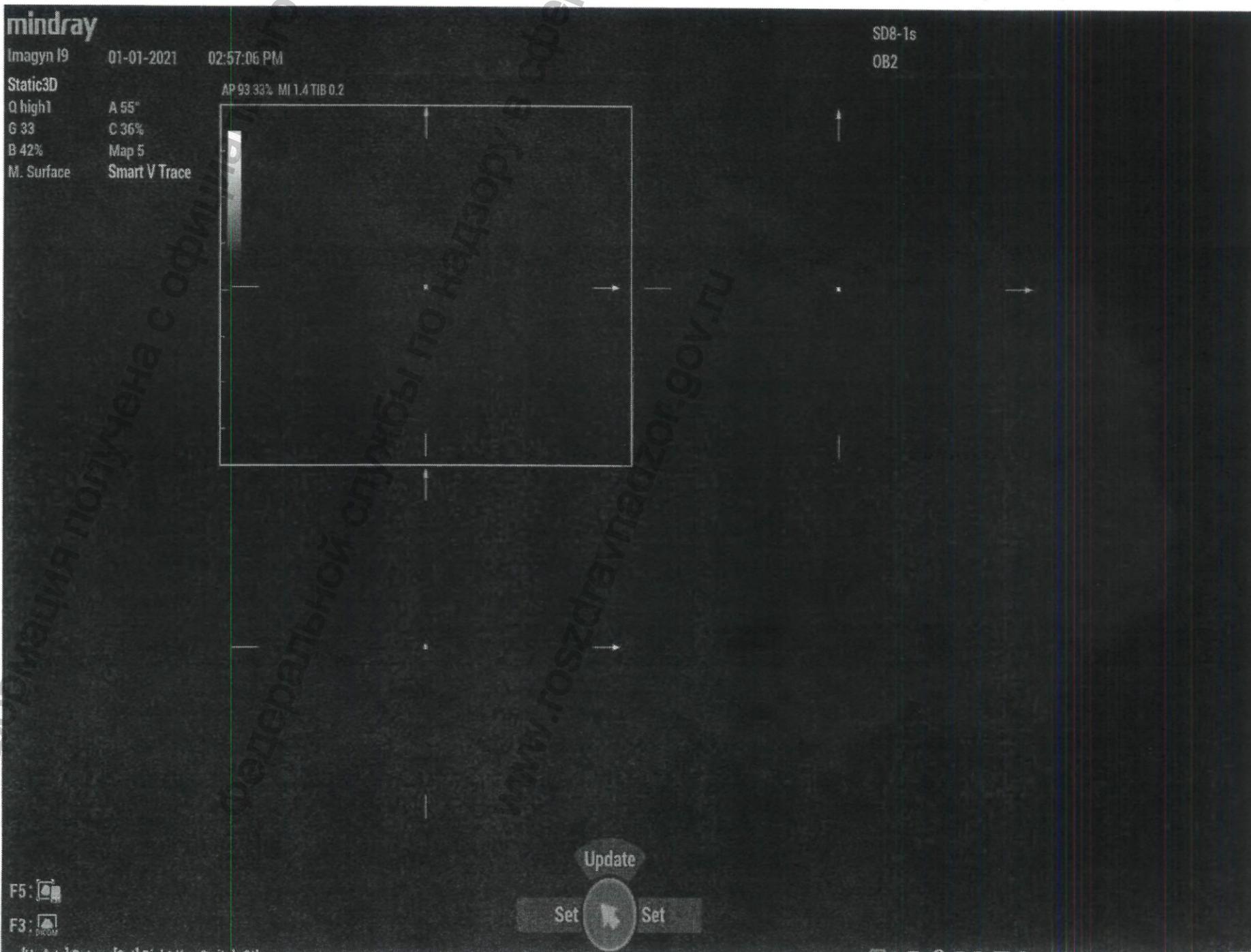
m

F4: [Icon] Update  
F3: [Icon]

Fetal heart is not identified in multiple frames. Manual acquisition of M sampling line is required.

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
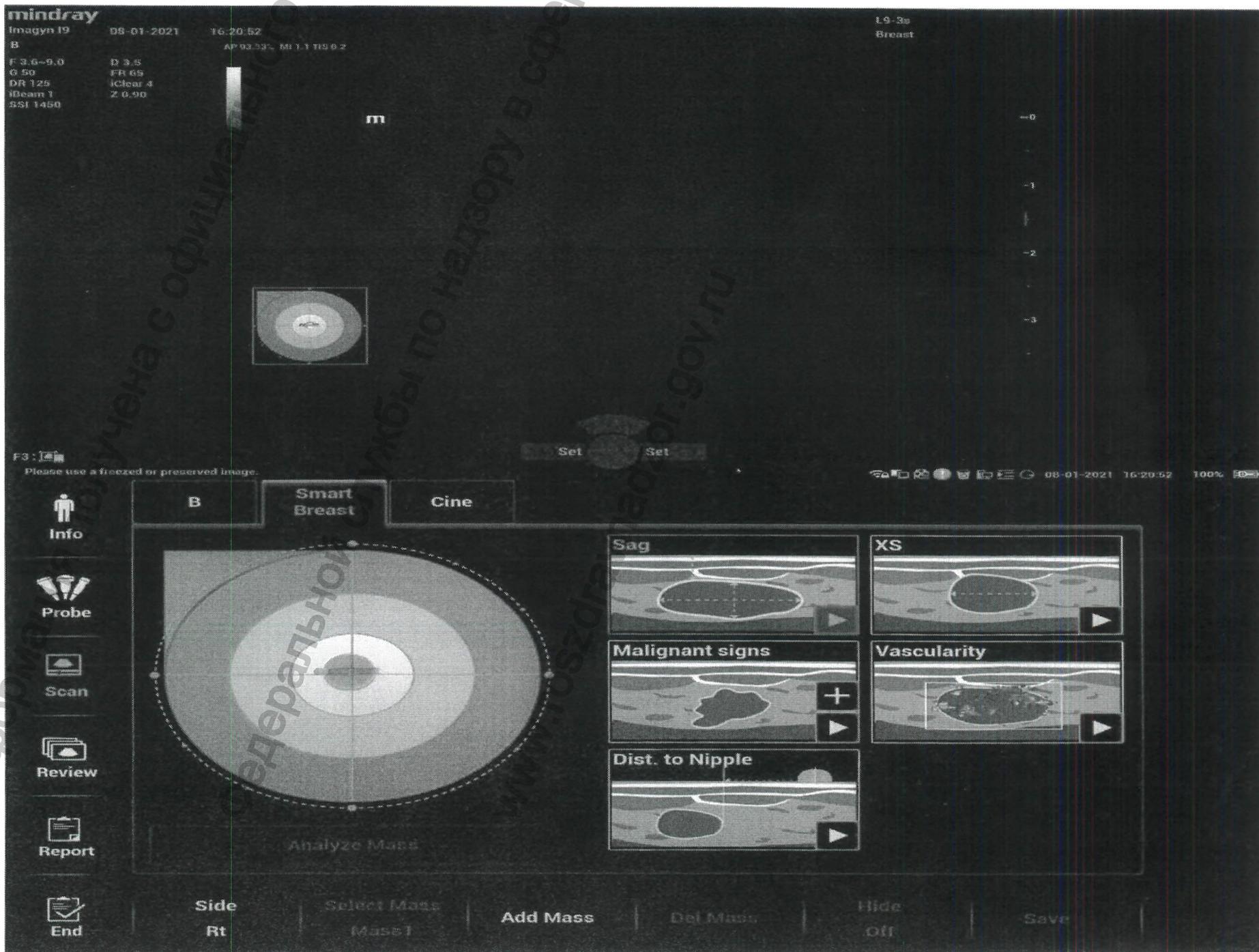
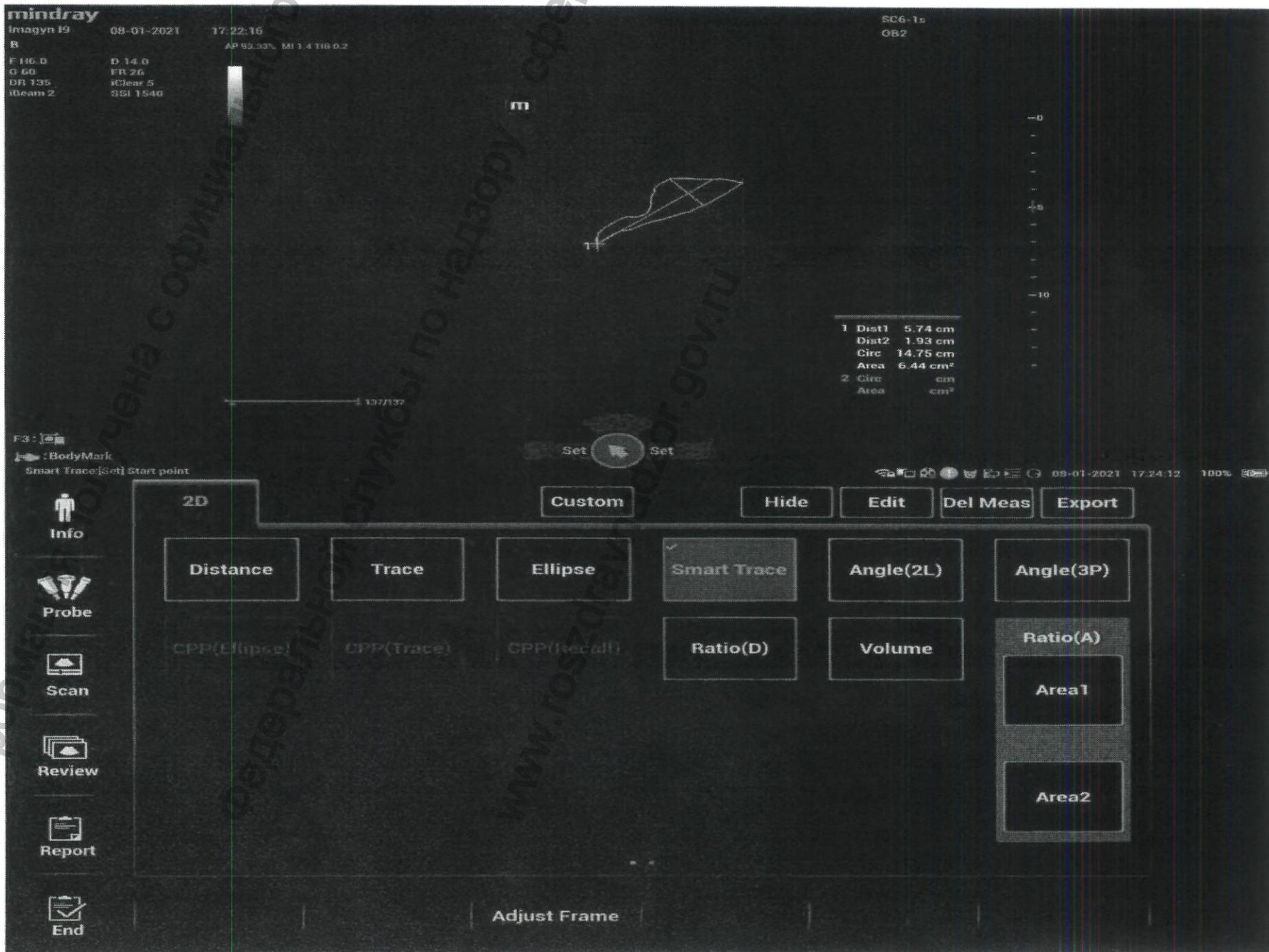
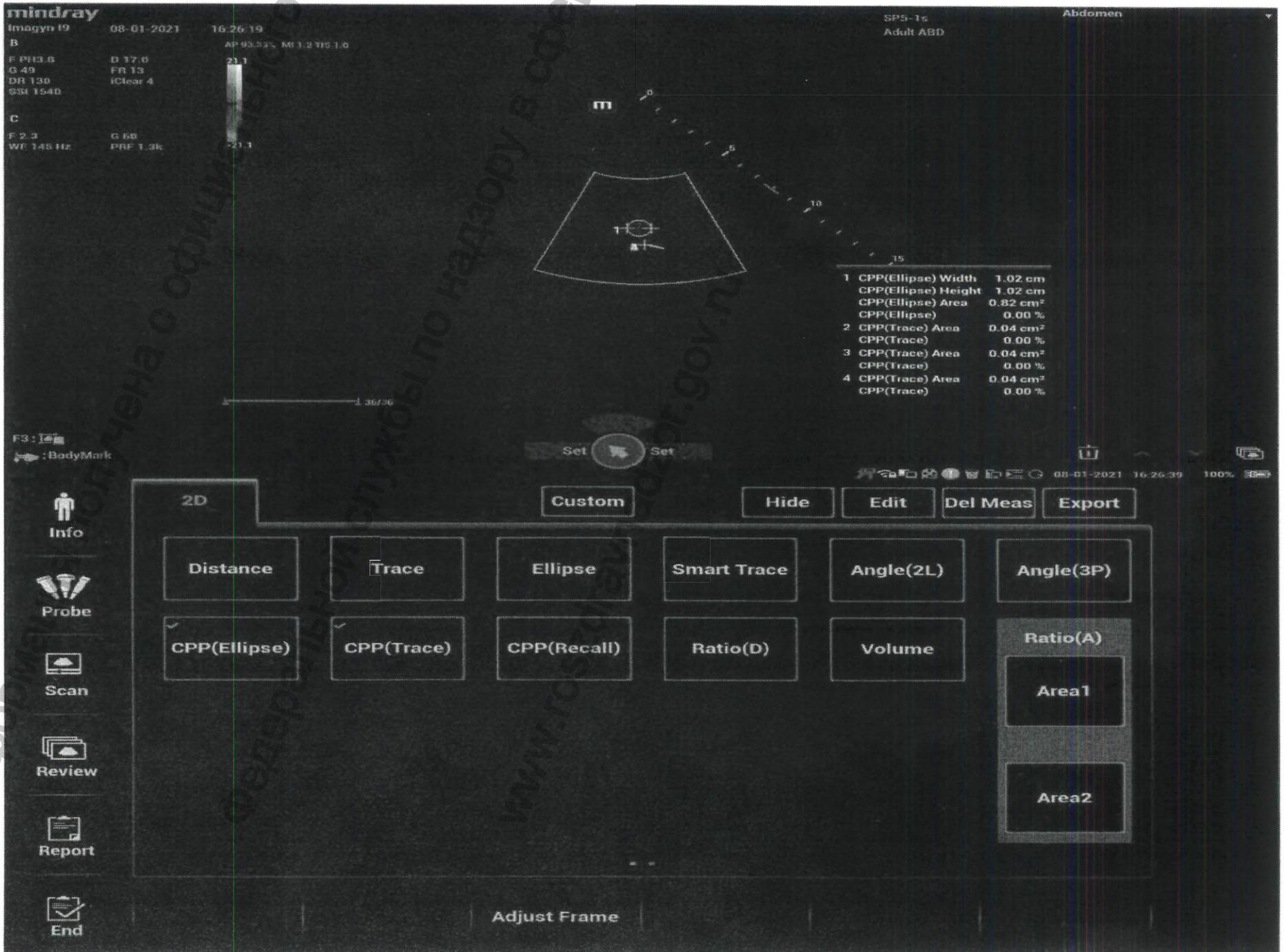
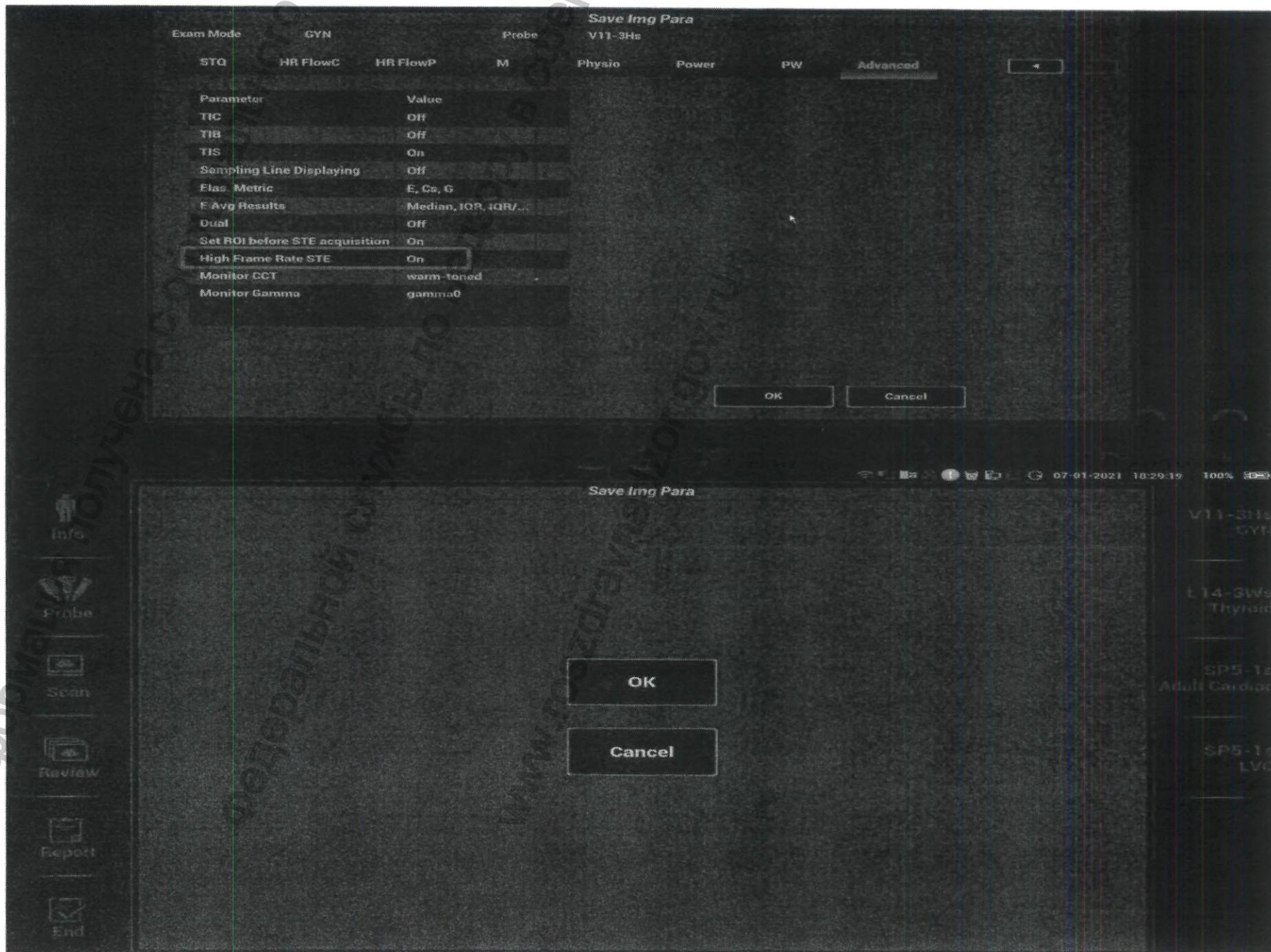
**mindray** SD8-1s  
Imagin 19 01-01-2021 03:04:37 PM OB2

Static3D AP 93.33%, MI 1.4 TIB 0.3

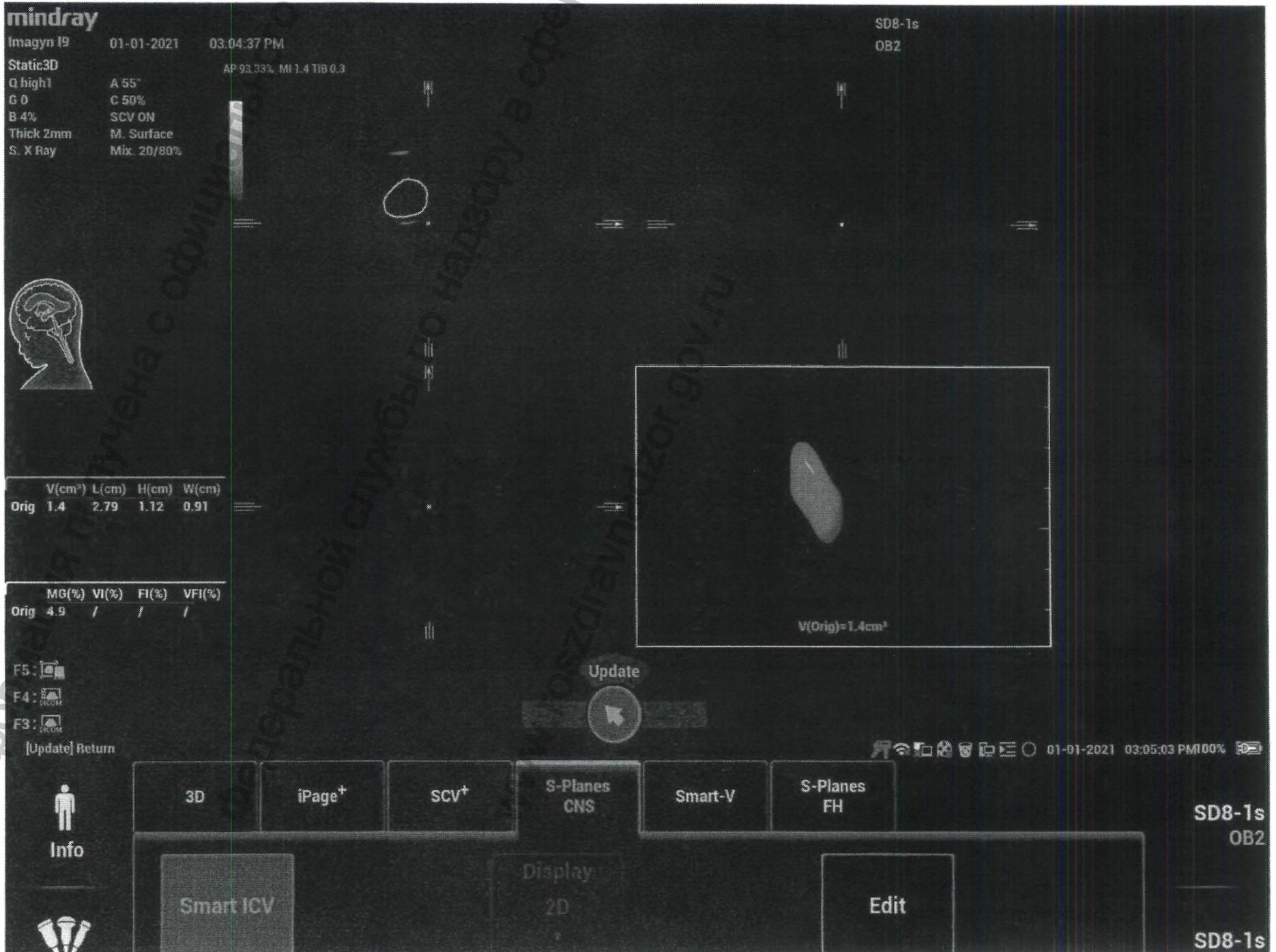Q high1 A 55°  
C 0 C 50%  
B 4% SCV ON  
Thick 2mm M. Surface  
S. X Ray Mix. 2D/80%

V(cm <sup>3</sup> )	L(cm)	H(cm)	W(cm)
Orig 1.4	2.79	1.12	0.91

MG(%)	VI(%)	FI(%)	VFI(%)
Orig 4.9	/	/	/

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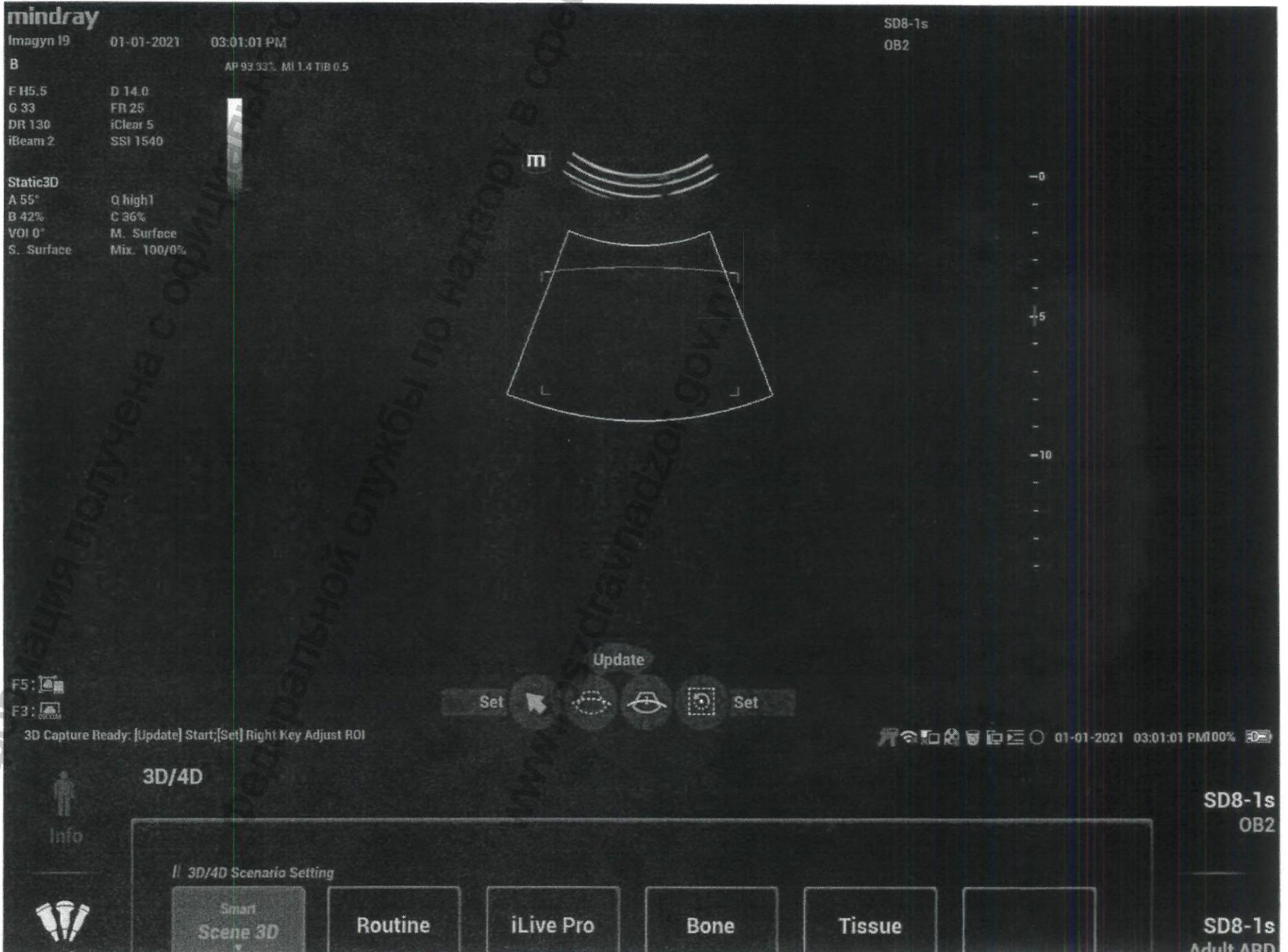
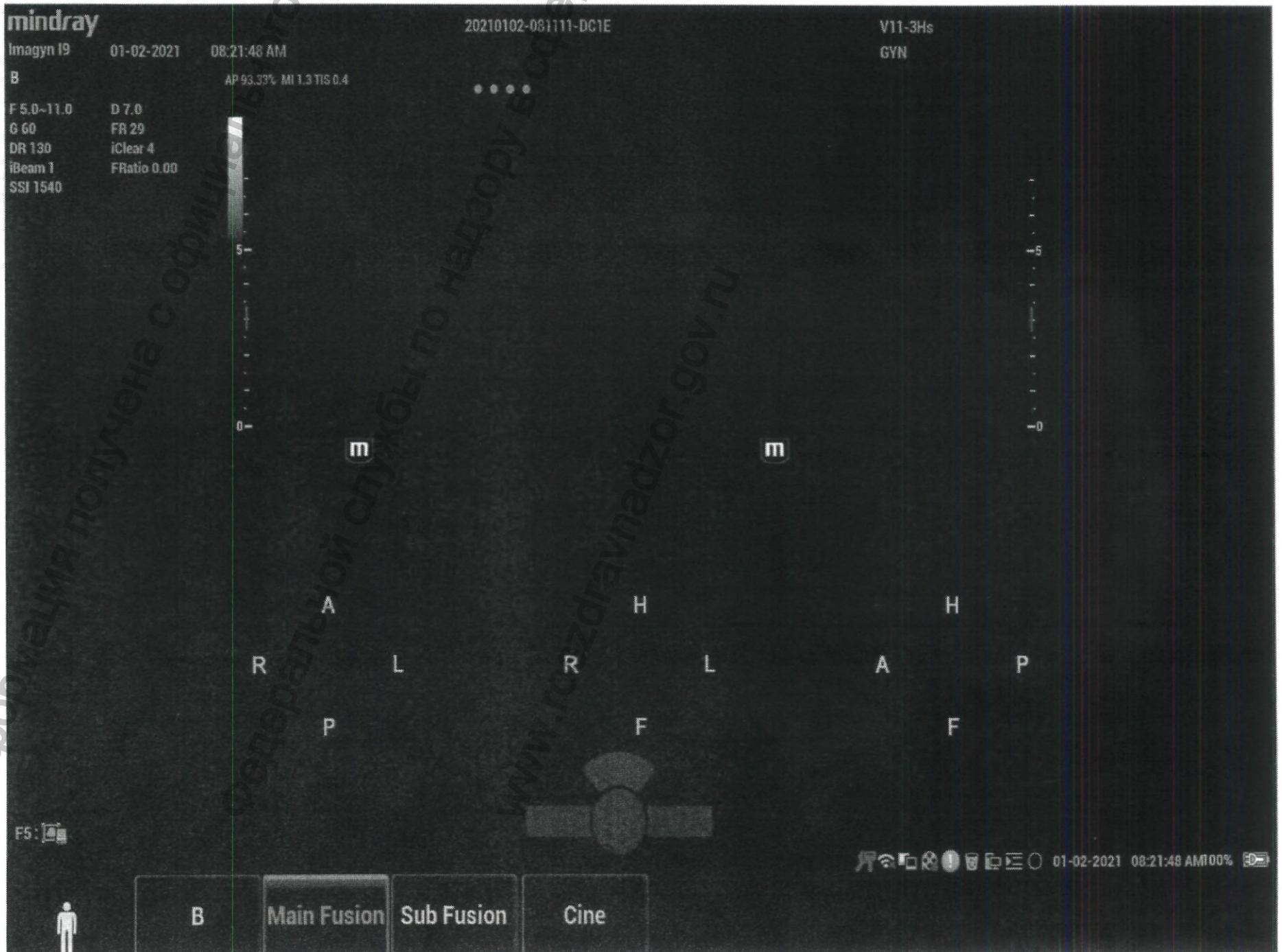
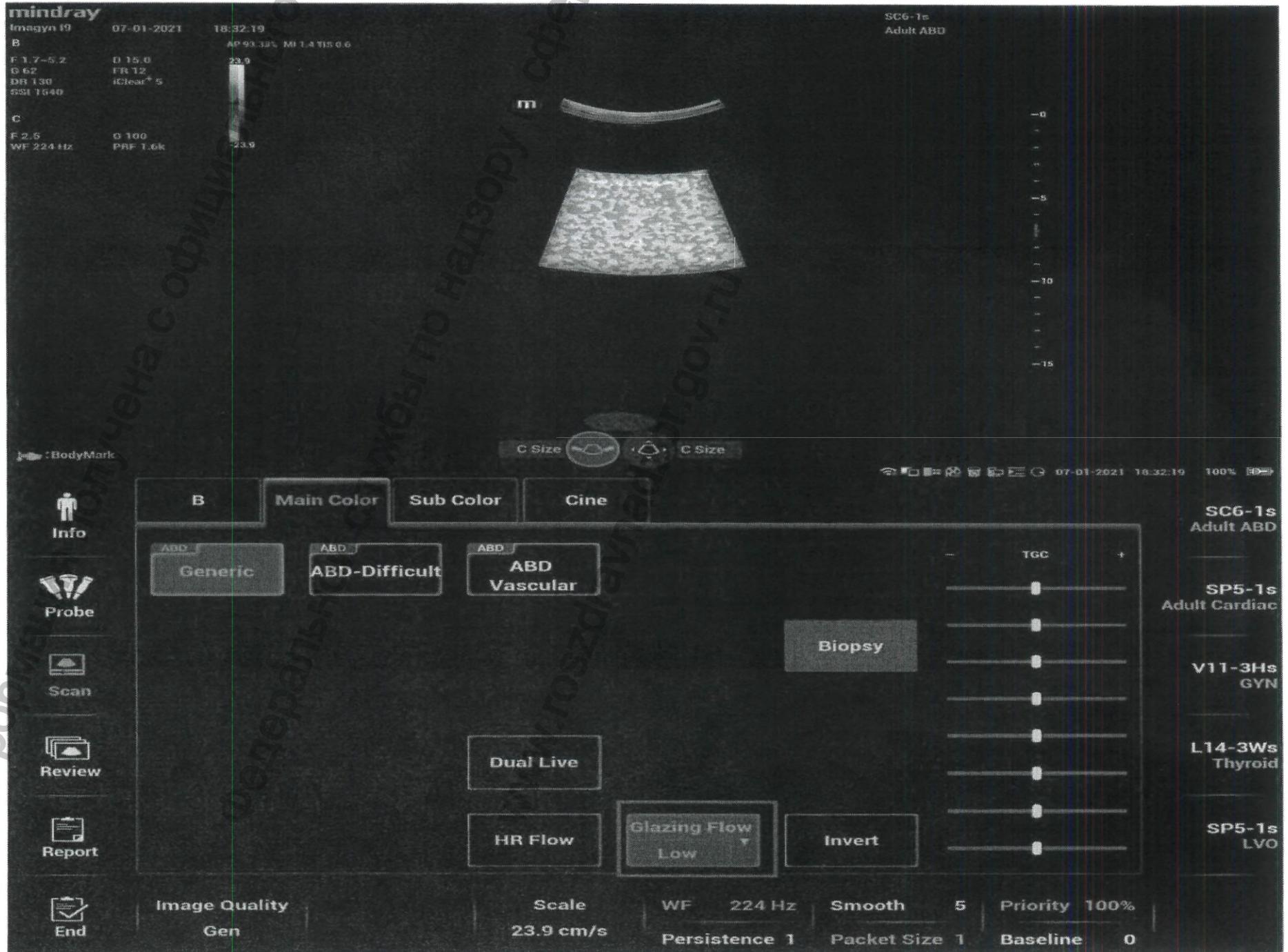
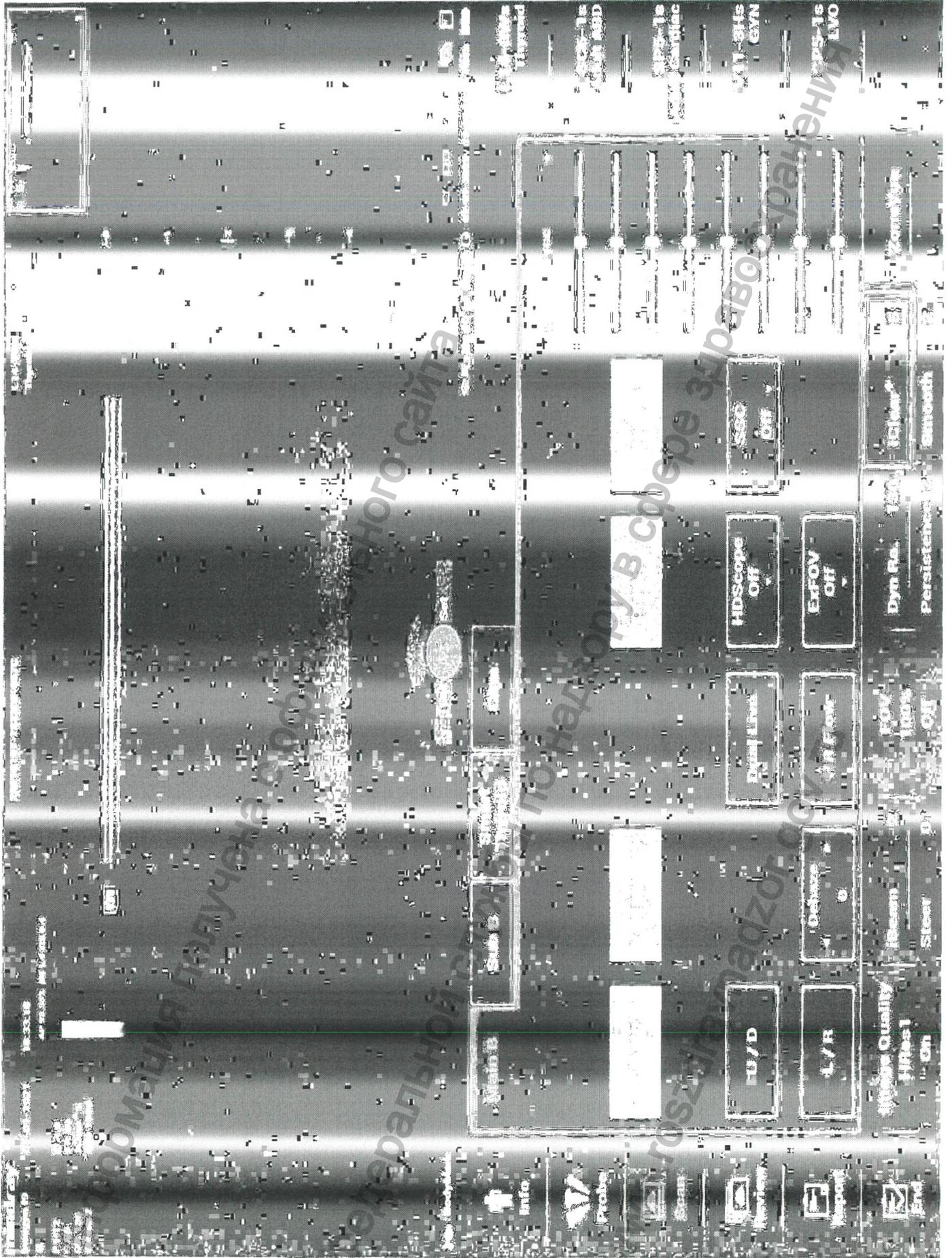
DVR-модуль (DVR Module)

**Option**  
// OPTION  
Keyword

	Title	Status
<input type="checkbox"/>	Smart Hip	Installed
<input type="checkbox"/>	Smart HRI	Installed
<input type="checkbox"/>	Smart Caliper	Installed
<input type="checkbox"/>	Smart Trace	Installed
<input type="checkbox"/>	CPP	Installed
<input type="checkbox"/>	Smart IVC	Installed
<input type="checkbox"/>	Smart VTI	Installed
<input type="checkbox"/>	Smart B-line	Installed
<input checked="" type="checkbox"/>	iWorks	Installed
<input type="checkbox"/>	iNeedle	Installed
<input checked="" type="checkbox"/>	DVR Module	Installed
<input type="checkbox"/>	McAfee	Not Install
<input type="checkbox"/>	V-Mapping	Installed
<input type="checkbox"/>	iVocal	Installed

Comment  
Digital Video Recorder, recording and playback ultrasound images and sounds.

Trial Install  
Select All Batch Install  
Promote Uninstall  
Auto Config

Caution: To trial, install and uninstall a component will cause the system shut down

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