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CERTIFICATE

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China Council for the Promotion of International Trade is China Chamber of International Commerce 11.2021

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hina Council for the Promotion of International Trade China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the signature of CHEN YONG of SHENZHEN WELL ELECTRIC COMPANY LIMITED and the seal of the said company on the annexed DOCUMENT are genuine.

China Council for the Promotion of International Trade

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Authorized

Hu Na

Signature:

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(Date: Nov. 09, 2021)

网址 Website for verifying the certificate: http://www.rzccpit.com/validate.html

Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K

Operation Manual

Director Shenzhen Well Electric Company Limited

Chan Yung 21 October 2021

2020

CONTENTS

1 INTENDED USE	
1.1 Intended use	
1.1.2 Operating conditions	1
1.2 Metrological and technical characteristics	6
1.3 The analyzer set	C
1.4 Arrangement and operation of the analyzer	11
1.5 Marking and sealing	
1.6 Packaging	19
2 DESCRIPTION AND OPERATION	10
2.1 Operational limits	1.0
2.2 Turning the analyzer on	10
2.3 Menu of the analyzer	20
2.4 Keypad. Functions of the virtual keys of the touchscreen	2.4
2.5 User settings menu	25
2.6 Preparing the analyzer for operation	20
2.6.1 Analyzer commissioning	20
2.6.2 Preparing the analyzer before starting work.	
2.7 Operating procedure in the measuring mode with data input	
2.8 Manual sampling mode	ا د
2.9 Refusal of measurement	
2.10 Operating procedure in the measuring mode without data input	40
2.11 Operating procedure in the screening mode	41
2.12 Reviewing and printing of measurement protocols from the analyzer memory	41
2.13 Printed measurement protocol content	44
2.14 The analyzer switching off	46
2.15 Possible faults of the analyzer.	4/
3 TECHNICAL MAINTENANCE	4 /
3.4 On-site technical maintenance.	50
3.4.2 Rechargeable power unit charging	50
3.4.2 Rechaigeable power unit charging	51
3.4.3 Refill the printer with paper	52
3.4.4 Lithium battery replacement.	52
3.4.5 Touchscreen calibration	53
3.4.6 Visual inspection 3.4.7 Cleaning and disinfection of the analyzer	53
3.5 Periodic technical maintenance	54
3.5 Periodic technical maintenance.	55
4 TRANSPORTATION, STORAGE AND UTILIZATION 5 METROLOGICAL VERIFICATION	55
6 MANUFACTURER'S WARRANTY AND ANALYZER REPAIR	57
APPENDIX A Degulatory references	58
APPENDIX A Regulatory references	59
APPENDIX B Electromagnetic compatibility information	6()
APPENDIX C Technical maintenance facilities	65
APPENDIX D Weight and Dimension Characteristics.	67
APPENDIX E Signs and Symbols Used in Marking	68
REGISTRATION SHEET OF CHANGES	71
METROLOGICAL VERIFICATION PROCEDURE	72

This Operation Manual (further – OM) is intended for a study of the device arrangement, the principle of operation, metrological and technical characteristics of the Breath Alcohol Analyzer ALCOTEKTOR PRO-100 touch-K (further – the analyzer) and contains the data necessary for its correct operation and technical maintenance.

The Breath Alcohol Analyzer ALCOTEKTOR PRO-100 touch-K is easy to use and convenient, based on the modern achievements of microelectronics.

The use of the analyzer provides reliability, dating, and documenting of measurement results.

The Operating manual contains references to the normative documents listed in Appendix A.

The persons who have studied the present OM and passed a personnel safety notification according to the established procedure are allowed to work with the analyzer.

Manufacturer: Shenzhen Well Electric Co., Ltd, China.

Address: 1-2F, No. 227, Hexicun, HengKeng, Guanlan Town, Longhua District, Shenzhen City, China, Tel: 86-755-83160728, Fax: 86-755-83160467.

The manufacturer's authorized representative (further – the Supplier): OOO ALCOTEKTOR

Legal address: 10 A, 2-N, 1st Sovetskaya str., St. Petersburg, 191036, Russian Federation, ph. (812) 320-22-97, web site: www.alcotector.ru, e-mail: info@alcotector.ru.

Address for correspondence: 5-7, Naberezhnaya reki Smolenki, Saint-Petersburg, Russia, p.o. box 256.

The inscription AЛКОТЕКТОР® is a registered trademark of OOO "ALCOTEKTOR".

The Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K (with accessories) are registered Federal Service for Surveillance in Healthcare and Social Development (the registration certificate of FS No. ΦC3 2011/10061 of

The Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K are registered Federal information fund for support of unity of measurements under No. 63415-16, the approval certificate of type of measuring instruments ______, validity period till

Certified copies of licensing documents, namely, the approval certificate of type of measurements, registration certificate for a medical device, and declarations of conformity (or other document confirming the compliance of the analyzer with mandatory requirements), are attached to the analyzer delivery set.

Approval documents also available on the website www.alcotector.ru¹. Archived versions of documents are available upon official request.

Regulatory documents referenced in the Operating Manual are listed in Appendix A.

In the Operating Manual following warning symbols are used:

• it indicates the need for the user to pay attention to the warnings and precautions for using mouthpieces given in the marked paragraph of the text.

- it indicates the need for the user to pay attention to the mouthpieces use rules given in the marked paragraph of the text.

¹ To find the required information on the site, it is advisable to use the site map.

1 INTENDED USE

1.1 Intended use

The analyzer is intended for the fast measurement of breath alcohol mass concentration.

1.1.1 Recommendations for use

- 1.1.1.1 The analyzer is recommended for application under the state traceability system:
 - state control and supervision in the road safety.
 - healthcare.
- 1.1.1.2 Potential consumers (users) using the analyzer for its intended purpose may be:
 - health service professionals (physicians);
- officials granted the right to state control over the safety of traffic and the operation of a vehicle, as well as Road Military Police officials;
- officials granted the right to state control over the internal regulations in the organizations;
 - private persons.
- 1.1.1.3 There are no contraindications and side effects to use the analyzer.

1.1.2 Operating conditions

- temperature range of ambient air from -5 to 50 °C;
- relative humidity range of ambient air: from 10 to 100% (without condensation);
 - atmospheric pressure range: from 84,0 to 106,7 kPa.

ATTENTION! To ensure safety when operating the analyzer with the AC power adapter connected, the user should additionally take into account the requirements for the operating temperature range of the ambient air of the AC power adapter.

1.1.3 The short name is ALCOTEKTOR PRO-100 touch-K.

Note: The front panel of the analyzer has a part of the short name of the analyzer: "PRO-100 touch-K".

- 1.1.4 The analyzer is resistant to the ambient temperature and humidity effects; their nominal values are specified in 1.1.3 of the OM.
- 1.1.5 In terms of mechanical stress resistance, the analyzer is classified as a wearable and portable product that is not intended to be used in moving.
 - 1.1.6 The analyzer conforms to requirements of standards IEC

- 60601-1 (Γ OCT P M9K 60601-1) and is a product with the internal power supply, a working part type B.
 - 1.1.7 The EMC information is given in Appendix B.
- 1.1.8 The analyzer housing is IP20 in terms of protection against the ingress of water and solid particles.
- 1.1.9 The analyzer meets the toxicological, sanitarian, and chemical requirements imposed for the skin-contacting materials.
- 1.1.10 The analyzer and supplied mouthpieces are not sterile and should not be sterilized.
 - 1.1.11 The analyzer is designed for continuous operation.
- 1.1.12 The noise level generated by the analyzer does not exceed 35 dBA.

1.2 Metrological and technical characteristics

1.2.1 The specified measuring range and allowable limits of intrinsic error of analyzer at the ambient temperature range from +15 to 25 °C inclusive are given in Table 1.

Table 1

Specified measuring range of breath alcohol mass	Allowable limits of Intrinsic Error at the temperature range from +15 to 25 °C, inclusive:	
concentration, mg/l	Absolute error	Relative error
from 0 to 0,200 inclusive	± 0,020 mg/1	_
above 0,200 to 1,200	Annual Control of the Control of Control or Control of Control or Control of	± 10 %

Note: In the analyzers is programmed the minimal result interval which is indicated and printed as null result: from 0,000 to 0,020 mg/l.

1.2.2 Readings range: from 0,000 to 2,500 mg/l.

Notes: 1 In case if the analyzer readings are exceeding an upper limit of measurements, the breath ethanol mass concentration is more than 1,000 mg/l.

- 2 The analyzer readings exceeding the upper limit of readings are indicated and printed as >2.500 mg/l".
 - 1.2.3 The scale interval is: 0,001 mg/l.
- 1.2.4 The allowable limits of intrinsic error at the operating temperature range of ambient air are given in Table 2.

Table 2

Temperature	Breath alcohol mass	Allowable limits of Error 1)	
range of ambient air	concentration range, mg/l	Absolute error, mg/l	Relative error,
from -5°C to	0 - 0.200 incl.	± 0,040	_
+5,0 °C incl.	above 0,200 – 1,200	_	± 20
above +5,0 °C	0 - 0.200 incl.	±0,030	- 0
to +15,0 °C incl.	above 0,200 – 1,200	_	± 15
above +15,0 °C	0 - 0.200 incl.	± 0,020	-07
to +25,0 °C incl.	above 0,200 – 1,200	_	± 10
above +25,0 °C	0 - 0.200 incl.	± 0,030	.0
to +40,0 °C incl.	above 0,200 – 1,200	_	± 15

The table shows the limits of the analyzer's allowable error under the operating conditions given in 1.1.3 of this OM.

- 1.2.5 There are no additional errors from non-measuring components in an analyzed gas mix.
- 1.2.6 Parameters of an analyzed gas mix in case if probe submission on an analyzer input (an automatic mode of breath sampling) are:
 - gas mix volumetric flow:

at least 20 l\min;

gas mix volume analyzed:

at least 1,2 1.

- 1.2.7 The time of preparing for operation after turning on is no more than 5 sec.
- 1.2.8 The measurement time after breath sampling is no more than 15 sec.
- 1.2.9 The time of preparing for operation after 0.50 mg/l gas measurement is no more than 30 sec.
- 1.2.10 The analyzer operating time interval without readings adjustment is no more than 12 months.

Readings adjustment is carried out before the metrological verification procedure by need.

- 1.2.11 The analyzer electrical power supply can be carried out in the following way:
 - from the rechargeable power unit not less than 7,4 V/4300 mAh;

 via the AC power adapter with following output characteristics:

= 12 V/2 A;

- via the vehicle power adapter with output voltage: == 12 V.

1.2.11.1 Characteristics of AC power adapter are:

- rated input voltage: $100-240 \text{ V} \sim$; - input voltage range: $90-264 \text{ V} \sim$;

- current frequency: 50±1 Hz; - voltage output: 12 V==;

- current output: not less than 2 A;

- electric shock protection class:

- operating temperature range of ambient air: from 0 to 40°C

1.2.11.2 Characteristics of the vehicle power adapter are:

- voltage input: 12 (24) V===;

- voltage output:

1.2.12 The output current is no more than 2 A.

1.2.13 The number of measurements on the analyzer without recharging the rechargeable power unit is:

without protocol printing: not less than 1000;

with protocol printing: not less than 200.

- 1.2.14 The thermal printer for printing out measurement protocols is structurally combined with the analyzer.
- 1.2.15 In the printer paper compartment of the thermal printer (further printer) the thermal paper roll is installed. The dimensions of the roll are: the paper width is 56±2 mm, the external diameter is 24±2 mm.
- 1.2.16 The analyzer is equipped with a built-in positioning system receiver (GLONASS/GPS) to determine the coordinates of the measurement site with automatic entry into the measurement protocol.

Note: The user activates the function of automatically entering the coordinates of the measurement site into the printout of the measurement protocol themselves (see 2.5.7.6 of the OM).

- 1.2.17 The analyzer operating modes are:
- measuring mode with data input is the mode of measurement of the breath alcohol mass concentration with the pre-filling the fields of the protocol form using the virtual keyboard;
- measuring mode without data input is the mode of measurement of the breath alcohol mass concentration without the pre-filling the fields of the protocol form using the virtual keyboard; the data can be written by hand in the fields of the printed protocol;

- screening mode is the mode of pre-estimation of the presence of ethanol in the exhaled air;
 - data transmitting mode (of saved measurement results) to PC;
 - mode of readings adjustment and check;
- mode of readings check through the dry gas sample is the mode of measurement of the alcohol mass concentration, used exclusively for readings check and metrological verification procedure through the gas mix of ethanol/nitrogen in cylinders under pressure;
 - touchscreen calibration mode.
 - 1.2.18 Breath sampling modes are:
- automatic mode: the analyzer breath sampling system is activated by the excessive pressure created by an exhalation of the tested person;
- manual mode: the analyzer breath sampling system is activated by clicking the virtual keys on the touchscreen.
- 1.2.19 The memory of the analyzer can hold 16000 results of the last measurements.
- 1.2.20 The user-selectable time of auto power off of the analyzer is 1/2/5/10 minutes.
- 1.2.21 The overall analyzer dimensions (Length x Width x Height) are no more than 220 mm x 80 mm x 40 mm.
- 1.2.22 The mass of the analyzer (with rechargeable power unit) is no more than 600 g.
- 1.2.23 The work term of the analyzer fuel cell sensor is at least 2 years.
 - 1.2.24 The average work term of the analyzer is at least 5 years.
 - 1.2.25 The average time between failures (MTBF) is 8000 hours.
- 1.2.26 The weight and dimension characteristics on components and accessories are given in App. D.

1.3 The analyzer set

The Breath Alcohol Analyzer ALCOTEKTOR PRO-100 touch-K contains the analyzer (measuring device), installed-inside rechargeable power unit and thermal paper roll, and attached mouthpieces or mouthpiece cups. The AC power adapter or vehicle power adapter (connected with the analyzer as needed) can be used.

The USB cable should be connected with the analyzer to work with the "PRO-100 touch-K Statistics" program.

1.3.1 The analyzer is a compact one-piece device containing a mainboard, measuring board, and keypad board; they are placed in the

insulating shockproof ABC plastic housing. The analyzer housing consists of two main parts, mouthpiece holder, pusher of mouthpiece, as well as the cover of power supply compartment, printer cover, and jagged paper ripping saw. There is a slot for stylus in the analyzer housing. On the mainboard are mounted: microcontroller, memory chips, backup lithium battery, real-time chip, buzzer, touchscreen, GLONASS/GPS module with external antenna, charge controller printer jack, mini USB jack, power supply jack, rechargeable power unit contacts. On the measuring board are mounted: fuel cell sensor, temperature sensor, and sampling system, in its turn, containing solenoid, gaiter, pressure transducer, sleeves, and connecting tubes.

1.3.2 The analyzer supplied set is given in table 3.

Table 3

Name	QTY
Analyzer	1 pcs.
Mouthpieces	105 pcs.
Mouthpiece cup	1 pcs.
Rechargeable power unit	1 pcs.
AC power adapter	1 pcs.
Vehicle power adapter	1 pcs.
USB cable	1 pcs.
Thermal paper	6 pcs.
Pouch	1 pcs.
Case	1 pcs.
Operation Manual	1 exs.
Technical data sheet	1 exs.
Metrological Verification Procedure MΠ-242-1956-2015	1 exs.

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1.3.3 Additionally, the Supplier provides the buyer with access to download the software «Статистика PRO-100 touch-К» [«Statistika PRO-100 touch-K] and «Статистика PRO-100 touch-К+» [«Statistika PRO-100 touch-K+] software for internal use. It is the external software that has no metrological effect. This software cannot lead to distortions of measurement results displayed on the analyzer display or printed; it is intended for collecting, preserving, and printing data from the analyzer memory on the personal computer; and it is not used by the analyzer for fast measurement of breath alcohol mass concentration in the breath sample. "Statistika PRO-100 touch-K" or "Statistika PRO-100 touch-K+" software with its operating manual is placed in the "Software" section on www.alcotector.ru website.

1.3.4 The list of consumables is given in table 3A.

Name	Application	Replacement
Mouthpiece	Is installed in the analyzer in the measuring mode (for every examined person)	Should be replaced after every test
Mouthpiece cup	Is installed in the analyzer in the screening mode	Should be replaced for cleaning, disinfec- tion, and in case of damage
Rechargeable power unit	Internal analyzer power source	Should be replaced after resource exhaustion
Thermal paper	For printer	Should be replaced as consumed

1.4 Arrangement and operation of the analyzer

1.4.1 The analyzer operating principle is based on the use of an fuel cell alcohol sensor for measuring of breath alcohol mass concentration.

The applied patented fuel cell alcohol sensor contains two platinum electrodes with a spongy platinum catalyst on an anode. A platinum catalyst is selective to ethanol. The fuel cell sensor's features are: high specificity to ethanol, high sensitivity, accuracy, and stability.

1.4.2 The embedded microcontroller of the analyzer manages all measurement processes. The analyzer has the firmware TouchK.

The internal system application software (further – software) of the analyzer is developed by the manufacturer especially for a solution of a problem of breath alcohol mass concentration measuring, and also displaying of measurement results on the display, storage of the measured data and transmitting of the measured data on external devices. Identification of the firmware is made by an output of version number to the display when analyzer turning on.

The software effect to analyzer metrological characteristics was taken into account under their rationing. The software protection level is "medium" according to the P 50.2.077. The analyzer firmware identification data are given in table 4.

Table 4

Firmware Identification data (characteristics)	Value
Firmware identification name	TouchK.RU
Version number (firmware ID) not less than	RU V1.28
Software digital ID of the	8D18DDC205F3914221B
	B36F18A042993
Digital ID generation algorithm	MD5
Note: The above digital ID value applies to the a sion only.	bove-specified software ver-

1.4.3 The analyzer is controlled by a touchscreen or keypad located on the front panel.

1.4.4 The working and breath sampling stages are followed by sound signals.

1.4.5 The analyzer is equipped with a real-time chip supplied by the lithium battery installed on the mainboard.

1.4.6 The non-volatile memory of the analyzer stores service settings, and also 10000 last measurements results.

Besides the result, the sequence number, date, and time of the test (measurement), the breath sampling mode, the date of the last readings adjustment, the date of metrological verification procedure, and data en-

tered from the virtual keyboard before the measurement (the examined person surname, test location and so on) are stored in the memory.

Note: If the analyzer's positioning system receiver has determined the coordinates of the location during the measuring, then these coordinates are also saved along with the measurement result. After the analyzer memory is filled, the follow-on measurement results will remain, replacing the results with the smallest sequence number, and the test numbering will begin again from 00001.

- 1.4.7 The analyzer blocks working if the measuring board (with the fuel cell sensor) temperature is below -5°C or above +50 °C.
- 1.4.8 The analyzer provides the possibility of blocking of work after the metrological verification interval is over. When the metrological verification interval is over, screening mode and measuring mode are blocked. For further use, the analyzer must be submitted for periodic metrological verification.

This option is activated in the service center under request (or in case of the amending legislation) or is carried out by the organizations accredited according to RF Accreditation legislation for the metrological verification procedure providing.

- 1.4.9 The appearance of the analyzer is shown on Figure 1.
- 1.4.10 The on/off key is intended to analyzer on/off.
- 1.4.11 The touchscreen is intended to display the information to a user, to control the analyzer, and for data input in the measuring mode with data input.

The data may be input by Cyrillic and Latin letters and numerals. The menu selection and symbols input are made by touching the touchscreen by stylus or by finger on appropriate icons.

Note: The new touchscreen may be covered with a protective film (one or two levels). Before use, you may remove the upper layer of two (if applicable). To remove it you should pull carefully the sticker of the upper layer (pull so that the lower layer of the protective film remains).


- 1.4.12 The **printer** built into the analyzer housing is designed to print measurement protocols on thermal paper.
- 1.4.13 The mini USB communication port (mini USB connector) located under a rubber cover on the right-side panel of the analyzer is intended for data exchange between the analyzer and the PC via the USB

cable.

ATTENTION! It is not allowed to connect devices other than PC to the analyzer mini USB port via USB cable.

- 1.4.14 The rechargeable power unit is installed in the analyzer power supply compartment located under the cover of power supply compartment on the analyzer back panel.
- 1.4.15 The **power supply jack** located under the protective rubber cover on the right-side panel of the analyzer is used to connect AC power adapter or vehicle power adapter.
- 1.4.16 The **pusher of mouthpiece** is intended to remove the mouthpiece.
- 1.4.17 The stylus is inserted into the top of the analyzer housing and secured with a cord; it is designed to touch the touchscreen during data entry or analyzer operation.

The stylus has a tip made of special soft plastic that does not scratch the touchscreen.

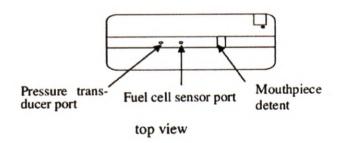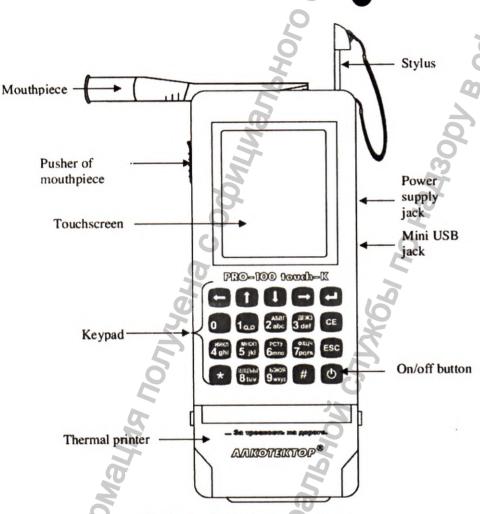
Figure 1 - Analyzer appearance

- 1.4.18 The **keypad** is designed to control the analyzer operation and to enter data in the measurement mode with data input. Data can be entered in Cyrillic, Latin, and numerals.
- 1.4.19 There are two input sampling system ports (fittings) between the mouthpiece guides formed by the front and back of the housing. One port leads breath into pressure transducer; another one leads breath into alcohol fuel cell sensor. (see Figure 1, view from above).
 - 1.4.20 The mouthpiece is intended to form the breath sample en-

tering the breath sampling system.

The specific shape of the mouthpiece (see Figure 2) provides excess air pressure in the breath sampling system input during exhalation and thus makes it possible to control the flow and volume of air exhaled. The mouthpiece has two holes for both of input fittings on its bottom side.

The mouthpiece is used as disposable in the measuring modes with data input or without data input. With the analyzer are used mouthpieces supplied by the manufacturer, or mouthpieces Mundshtuk po TU 22.29.29-001-82139963-2017 (identichny TU 2291-001-82139963-2015), ispolnenie "Mundshtuk ALCOTEKTOR s dvumya otverstiyami"

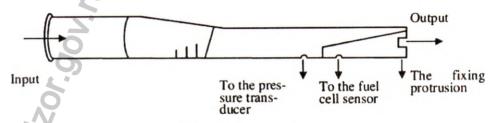
Figure 2 - Mouthpiece

1.4.21 The reusable mouthpiece cup (see Figure 3) forms breath flow for breath sampling in the screening mode.

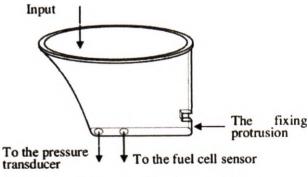
Figure 3 - Mouthpiece cup

1.4.22 It is allowed to use the mains and vehicle power adapters supplied with the analyzer or adapters with the output characteristics specified in 1.2.11.1-1.2.11.2 that meet the mandatory requirements provided for by the legislation of the Russian Federation.

Obligatory design requirements for power adapters:

- outer/inner diameter of the plug is 3.5 / 1.3 mm;
- cable length is no more than 3 m.
- 1.4.23 The USB cable is a four-wire cable no more than 3 m long with a USB connector for connection to a PC and a mini USB connector for connection to the analyzer.
- 1.4.24 To ensure safety when operating with the software Statistika PRO-100 touch-K and Statistika PRO-100 touch-K+ the analyzer should be connected to PC meeting ΓΟCΤ IEC 60950-1 requirements.

1.5 Marking and sealing

1.5.1 The following marking is put on the analyzer:

- keys names;
- part of the short name of the analyzer: "PRO-100 touch-K";
- registered Supplier trademark AJIKOTEKTOP®.
- name of the analyzer;
- quick start (using the following graphic images: on/off button; measuring mode menu icon; the virtual navigation key (to go to the next menu window); the enter key on the keypad);
- analyzer serial number;
- month and year of manufacture;
- Supplier name;
- Manufacturer name and address;
- metrological approval sign: , Common Mark of Products Circulation in the Market of the Customs Union member-states: [];
 - type B device sign: 🐧 ;
 - symbol of international protection (IP): IP20;
- number of registration certificate of Federal Service for Surveillance in Healthcare and Social Development;
- rated value of supply characteristics and type of current (near the power supply jack): 12 V = 2 A.
 - 1.5.2 The Supplier seals the left upper mounting screw in a power

supply compartment of the analyzer. The Supplier seals the analyzer with a label with the "Breaking the seal voids warranty" inscription, which self-destructs when trying to open the analyzer.

- 1.5.3 At the request of the Customer, additional marking can be applied, indicating the explanatory inscriptions about the purpose of one or the other accessories included in the analyzer delivery set.
- 1.5.4 A list of signs and symbols used in the marking of components and accessories and their explanation is given in Appendix E.

1.6 Packaging

For transportation the analyzer in a complete set is packed into a transport container of the Supplier – corrugated fiberboard boxes.

The mouthpieces packed into individual primary packaging of a polyethylene or polypropylene film at exceeding of the 5 pcs are packed into secondary packaging (bags from a polyethylene film).

2 DESCRIPTION AND OPERATION

2.1 Operational limits

- 2.1.1 To ensure safety requirements and prevent unforeseen risks, the user must comply with and be responsible for the fulfillment of the following 2.1.2-2.1.14 requirements.
- 2.1.2 Before the use of the analyzer, be ensured that operating conditions satisfy (see 1.1.2 of the OM).
- 2.1.3 It is not allowed to store and use the analyzer in rooms in which storage of alcohol-containing substances in open tanks is carried out, and to process of surfaces or the equipment by alcohol-containing solutions.
- 2.1.4 It is not allowed to store and use the analyzer in rooms with increased dust content and gas contamination to avoid pollution of the analyzer breath sampling system.
- 2.1.5 \(\text{\text{Do not allow the condensate accumulation in the mouthpiece cup when using the analyzer in the screening mode to prevent moisture from entering the analyzer sampling system.}
- 2.1.6 When using the analyzer with a power adapter, it is recommended to remove the rechargeable power unit from the analyzer power supply compartment.
- 2.1.7 It is not allowed to connect devices and cables that do not meet the requirements of the OM to the analyzer.

- 2.1.8 It is not allowed to connect devices other than PC to the analyzer mini USB port via USB cable.
- 2.1.9 It is required to comply with the safety operating rules for devices connected to the analyzer.

ATTENTION! The mains plug is the means for electrically separating the analyzer from all poles of the mains supply when the analyzer is powered from the AC power adapter.

For additional protection, you can use the analyzer in a carrying case.

- 2.1.10 To exclude the possible effect caused by ethanol from the oral mucosa to the measurement result, at least 20 minutes should pass before the measurement after the consumption of alcohol-containing medications and oral sprays, as well as low-alcohol food products (fermented milk products, kvass, etc.).
- 2.1.11 In avoidance of a contamination of the sampling system, the analyzed sample should not contain particles of tobacco smoke, phlegm (saliva), and the remains of food. Therefore, before measurement, the user has to:
 - pass at least for 2 minutes after smoking;
 - rinse a mouth water if there was a meal just before measurement.
- 2.1.12 It is not allowed to use the mouthpiece if primary packaging is broken.
 - 2.1.13 ATTENTION! It is not allowed to modify the analyzer.
- 2.1.14. During operation, the conditions for excluding the analyzer falling from the user's hands should be provided to avoid damage to it. Manufacturing the analyzer case of impact-resistant material without sharp protrusions reduces the risk of damage from sharp edges or shards when the instrument is dropped. For additional protection, you can use the analyzer in a pouch.

2.2 Turning the analyzer on

2.2.1 To turn the analyzer on, click the on/off key, holding it until a sound signal (about 2 seconds). At the same time, the splash screen with the analyzer short name and software version number is brought to the touchscreen, the self-test is carried out, and then the main menu of the analyzer appears on the touchscreen (see Fig.4).

Note: If, during self-testing, ethanol vapors are detected in the taken ambient air sample or the analyzer's sampling system, the message "Detected" will appear on the analyzer touchscreen for a short time. In this case, follow the instructions given in Table 6.

2.2.2 At the moment analyzer is turning on, the date of the next metrological verification is automatically checked. If the validity of the verification results has expired, after the analyzer is turned on, the following message appears on the touchscreen accompanied by a sound signal:

The verification is necessary.

Touch the screen to continue

This message is a reminder of the need to present the analyzer for periodic metrological verification.

2.2.3 If the temperature of a measuring board with the fuel cell sensor is above 50 °C or below 5 °C, the message of "Temperature Extreme high (low)" is displayed on the touchscreen, and the analyzer is switched-off.

In this case, sustain the analyzer under operating conditions, specified in 1.1.2 the OM, and try again.

2.2.4 If there is no thermal paper in the printer, when the analyzer is turned on, a short message "No paper" appears on the touchscreen.

In this case, you should install a new roll of thermal paper in the printer (see 3.4.3 of the OM).

2.3 Menu of the analyzer

- 2.3.1 The main menu of the analyzer consists of 9 items-icons and indicators informing about the charge level of the rechargeable power unit, the current time and date, as well as the determination of the coordinates of the current location by the positioning system receiver (Figure 4).
- 2.3.2 The power indicator is on the upper part of the main menu. It informs on a charge level of rechargeable power unit in the power supply compartment under adapter disconnection. A fully colored power indicator indicates a fully charged rechargeable power unit; in the process of charge level lowering (discharge), the painted-over indicator part will decrease.
- 2.3.3 The positioning system indicator appears in the upper part of the main menu when current coordinates are determined.
- 2.3.4 In the upper right part of the main menu the current day of the week, time and date in the following format are displayed:

Day of the week, hours:minutes:seconds;

Day/Month/Year.

Current date and time are recorded at test breath sampling, kept in the analyzer memory together with the result of measurement, and printed in the measuring protocol.

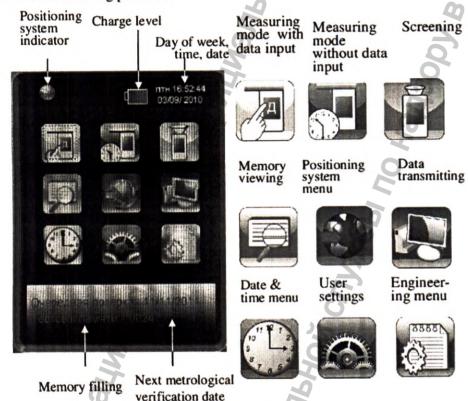
Figure 4- The main menu

- 2.3.5 At the bottom of the main menu the following information is displayed:
- next metrological verification date (the metrological verification date is installed or corrected under the positive verification results);
- memory filling information (number of measurement results (tests) stored in the memory.
- 2.3.6 The measuring mode with data input is intended for a measurement of breath alcohol mass concentration in the taken breath sample. This mode implies the ability to enter data (by filling the meas-

urement protocol fields via virtual keyboard or keypad) before measuring; as well as to record entered data into the analyzer memory with the measuring results.

- 2.3.6.1 Before the sale, the analyzer is set to one of two options of factory settings with a specific set of fields for data entry (see 2.3.6.2 or 2.3.6.3 of the OM, depending on the application scope of the analyzer); or an individual set of fields (see 2.3.6.4 of the OM).
- 2.3.6.2 The analyzers that will be used for the examination of the alcoholic intoxication by the traffic police have six fields for data entry with the following names (option 1 of the factory settings):
 - field 1: Examined Person Name;
 - field 2: Examination Place;
 - field 3: Vehicle registration number;
 - field 4: Policemen ID;
 - field 5: Policemen Name;
 - field 6: Police Department.

Entered data of 2, 4, 5, and 6 field are saved in the analyzer memory until it will be changed by the user (see the Note of 2.7.4 of the OM).

- 2.3.6.3 The analyzers that will be used for the medical examination of the alcoholic intoxication by physicians have six fields for data entry with the following names (option 2 of the factory settings):
 - field 1: Examined Person Name, First Name;
 - field 2: Year of Birth;
 - field 3: Referral (by whom is he referred for the examination);
 - field 4: Referral ID;
 - field 5: Examination Place;
 - field 6: Physician Name, First Name.

Entered data of 3, 5, 6 field are saved in the analyzer memory until it will be changed by the user (see the Note of 2.7.4 of the OM).

2.3.6.4 For the analyzers used in other scopes (driver medical checkup, for example), as well as when the corresponding legislative normative documents appear, the number and name of the fields can be changed by the Supplier before selling; or at service centers during the life of the analyzer.

The analyzer allows to set from 0 to 10 fields, as well as set (mark) those fields, the entered data of which will be stored in the analyzer's memory until the user enters new data instead.

- 2.3.6.5 The data entered before the measurement is saved in the analyzer's memory and printed out in the measurement protocol together with the measurement result and other data (see 1.4.6 of this OM).
- 2.3.7 The measuring mode without data input is intended for a measurement of breath alcohol mass concentration in the taken breath sample. This mode does not imply the ability to enter data; the fields should be filling by hand after printout.
- 2.3.8 The screening mode is intended for a preliminary assessment of the ethanol in the breath. The test result in this mode is displayed as the text message about alcohol presence or absence.

ATTENTION! The screening mode should be used for the assessment of the presence of ethanol in the breath only. The error in this mode may exceed the allowable limits of Intrinsic Error (see 1.2.4 the OM).

The screening test results are stored in the analyzer memory, but they can't be printed.

In this mode, it is advisable to use a reusable mouthpiece cup from the analyzer set.

- 2.3.9 The memory viewing menu allows viewing the recorded measurement results and printing measurement protocols.
- 2.3.10 In the positioning system menu the user can view the current position coordinates (longitude and latitude) and, if necessary, activate the function of automatically entering them into the measurement protocol.
- 2.3.11 The data transmitting mode is intended for storing data from the analyzer memory (saved measurement protocols) to a database on a personal computer using the "Statistika PRO-100 touch-K", " Statistika PRO-100 touch-K+", or other similar programs.
- 2.3.12 The date & time menu is intended for correcting the current date and time by the user during the operation of the analyzer.
- 2.3.13 The user settings menu is intended for setting the time for the automatic shutdown of the analyzer, the print options, and the dry gas measuring mode.
- 2.3.14 The Engineering menu is intended for using in service centers and in metrological verification procedure only. Access to the menu is password protected. The engineering menu includes: readings adjustment, setting of the sampling system parameters, the set of the data entry

fields, the date of metrological verification in the analyzer's memory, and other options.

Description of the menu is given in the "The Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K. Maintenance instructions. Engineering Menu" document, which is provided by the Supplier to service centers and organizations accredited to calibrate ethanol vapor analyzers on request.

2.4 Keypad. Functions of the virtual keys of the touchscreen

- 2.4.1 The keypad situated on the analyzer front panel is intended for analyzer controlling and data entering in the measuring mode with data input.
 - 2.4.1.1 The keypad has the following special keys:
 - the enter key (choice confirmation);
 - the key to delete the last typed character from the input line;
 - the key to go to the main menu;
 - the key to go to the test search window in the analyzer's memory, and is also used in a key combination to enter the touchscreen calibration mode;
- the key to go to the next or previous 10 records in the memory viewing menu;
- the navigation keys (to go to the left or to the right);
- 2.4.1.2 Selecting an item, symbol or activating virtual buttons displayed on the touchscreen using the keypad is carried out in two stages:
- a) move the red or dotted frame to the required icon or virtual button on the touchscreen by pressing the navigation keys or on the keypad;

Notes:

I In the main menu of the analyzer, the initially red frame marks the icon of the measurement mode with data input .

2 When switching to some windows of the analyzer menu, it may appear that none of the icons or virtual keys will be marked with a red or

dotted frame; pressing the navigation keys or on the keypad will cause a frame to appear.

- b) confirm the selection of the icon or virtual key (marked by a red or dotted frame) by pressing the enter key on the analyzer's keypad.
- 2.4.1.3 The letters and numerals entering is carried out by pressing one or another symbolic key (key of the keypad with the image of numerals and letters) a certain number of times (first a numeral appears, then alternately Cyrillic letters and Latin letters depicted on this key).

You can also enter letters and numerals from the virtual keyboard by selecting the desired character using the navigation and input keys on the keypad.

2.4.2 The following located at the bottom of the touchscreen virtual control keys are used to switch between the menu windows or to print the measurement protocol:

switch to the following window of the menu (at achievement of the last window of the menu - an out-put in the main menu);

return to the previous window;

switch to the main menu;

result printing;

- transition to the following (bypassing the main menu).

To activate the desired virtual key user should touch it by stylus, or select it via keypad (see 2.4.1.2 of the OM).

2.5 User settings menu

During operation, the user has an opportunity to adjust the current date and time, and to set the number of protocol printing copies, time of auto power off of the analyzer, the measuring mode (with or without data

input)at his discretion.

The above settings are made in the corresponding menus, which are

accessed from the user settings menu.

- 2.5.1 The time of auto power-off of the analyzer, printing settings, the dry gas sample measuring are made in the user settings menu.
- 2.5.2 To enter the "User settings" menu in the main menu of the analyzer (Figure 4) select the "User settings" icon item. The touchscreen will display a three-item menu (Figure 5).

BЫКЛ. (OFF) – the time of auto power off setting;

ΠΕΥΑΤЬ (**PRINT**) – the printing settings (the number of protocol copies);

IPOBEPKA – the dry gas sample measuring.

Figure 5 - The user settings menu

- 2.5.3 Time of auto power off setting
- 2.5.3.1 To set the time of auto power off user should enter into the user settings menu (see 2.5.2 of the OM) and choose "Off". (see Figure 6). The window "Setting auto power off" will appear.

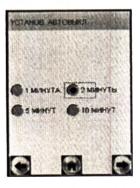
Figure 6 - The time of auto power-off setting

- 2.5.3.2 User should choose one of four settings by touching the appropriate checkbox near the selected item.
 - 2.5.3.3 The analyzer will turn off automatically after the preset

time after the last user action. Before 10 seconds up to auto power-off, the message "Turning Off" will appear. Then, a countdown of 10 seconds will perform accompanied by a sound signal, after which the analyzer turns off. To prevent auto power-off, the user should touch the touchscreen or press any key during count down. The analyzer will return to the main menu without input data save.

It is recommended to select the optimum auto power-off time to save power.

2.5.4 Printing settings

2.5.4.1 During operating, the user can activate the function of the automatic protocol printing of the preset number of protocol copies.

In this case, the printer will automatically print the preset number of protocol copies right after the result displaying.

If the function is not activated, the protocol printing will be performed only by touching the corresponding virtual button in the touchscreen bottom.

2.5.4.2 During operation, the user at his discretion (or according to the requirements of the legislation) can set the number of copies of the automatically printed after result displaying measurement protocol. The maximum number of copies is 5.

The analyzer allows to set the different number of copies depends of readings: for readings 0.00 mg/l and readings > 0.00 mg/l. (see Figure 7).

2.5.4.3 Enter the user settings menu (see 2.5.2 of the OM) and press the virtual key "Print" (see Figure 5). The menu "Print" will appear (see Figure 7).

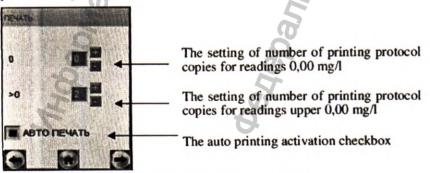
Figure 7 – The printing settings

User should set the number of printing protocol copies by pressing "+"&"-" virtual keys.

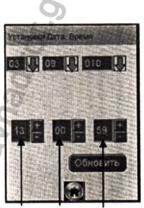
To print the measurement protocol in a specified number of copies automatically after the measurement result is displayed on the touchscreen, you should activate the automatic print function by clicking on the box "AUTO PRINTING".

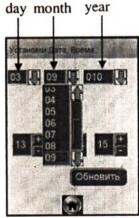
2.5.5 Dry gas sample measuring mode

Description of dry gas measuring mode is given in the document "Breath alcohol analyzers ALCOTEKTOR PRO-100 touch-K. Instruction for readings adjustment. Engineering Menu" provided upon additional request.

2.5.6 Date and time correction

To correct analyzer date and time user should press the "Date & time" key in the main menu – the window "Date & Time Settings" will appear (see Figure 8).

hours minutes seconds

Figure 8 - The date and time settings

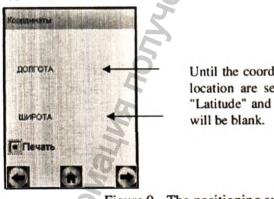
To change date and time user should press the "+"&"-" virtual keys near the parameter have to be changed.

2.5.7 Features of the operation of the positioning system receiver 2.5.7.1 The built-in receiver of the positioning system (GLONASS /GPS) (further - the receiver) is activated when the analyzer is turned on. If the analyzer has been off for the last half-hour, then the initialization of the receiver can be longer and take about 5-10 minutes.

For stable and accurate operation, the analyzer receiver needs an "open sky", that is, no interference overhead and near (buildings, trees, tunnels, etc.). The analyzer's receiver is hard to operate on cloudy days and in densely built areas. When using the analyzer in buildings, tunnels, etc., where the satellite signal is usually absent or extremely weak, the operation of the receiver is hampered by the obstacles.

In case of interference, the receiver may either not determine the coordinates at all, or transmit inaccurate data. The level of accuracy depends on many factors and is not guaranteed.

- 2.5.7.2 The positioning system indicator will appear in the upper left corner of the main menu on the analyzer touchscreen when the position coordinates are determined by the receiver.
- 2.5.7.3 To view the current coordinates of the location of longitude and latitude) and activate the function of automatically entering them into the measurement protocol, the user should select the "Positioning system menu" icon in the analyzer main menu. The menu "Coordinates" will appear (see Figure 9).

Until the coordinates of the location are set, the fields "Latitude" and "Longitude"

Figure 9 - The positioning system menu

ATTENTION! After printing out the measurement protocol, the receiver turns off (stops updating the position coordinates), wherein the positioning system menu saves the coordinates of the analyzer location recorded at the time the measurement protocol was printed. Therefore, to resume the operation of the receiver and update the location coordinates, it is necessary to turn off and turn on the analyzer again. If the location of the analyzer was not changed since the moment the receiver was turned off; the signal search time will be less than on first turned on.

Figure 10 - Google Maps service

On the page that opens, enter the coordinates indicated in the printed measurement protocol, in the search bar, as shown in Figure 10, and click the search button.

Note: When entering coordinates in the search bar, use Latin letters, use spaces to separate degrees and minutes, use a dot to separate the integer and fractional parts of minutes, enter coordinates separated by commas, for example:

E030 15.69541, N59 56.94355

- 2.5.7.5 If at the time of measurements the analyzer receiver has determined the location coordinates, then these coordinates are saved together with the measurement result.
- 2.5.7.6 To activate the function of automatic entry of location coordinates into the measurement protocol, click on the "Print" box (Figure 9). After that, a field with coordinates of the measurement site will be added to the printed measurement protocol.
 - 2.6 Preparing the analyzer for operation
 - 2.6.1 Analyzer commissioning

To put the analyzer into operation, the user should check the completeness of the analyzer check for compliance with the technical datasheet and/or delivery contract and information in the Federal information Fund for ensuring the uniformity of measurements (FIF) about the results of the analyzer metrological verification.

2.6.2 Preparing the analyzer before starting work

Before working user should perform the visual inspection of the analyzer and preparing to operation:

- be convinced that no mechanical damages influencing the operability and safety of the analyzer, including the mechanical damages of all adapters and USB cable;
- check a voltage level of the rechargeable power unit and charge it if necessary (see 3.4.2 of the OM); or connect the analyzer to the vehicle power supply or the AC main power supply of 220 V, using the appropriate adapter (see 1.2.11 of the OM);
- check the paper is present and is installed correctly (see 3.4.3 of the OM):
 - check the printing settings and correct if necessary (see 2.5.4).
- check the current date and time which are installed in the analyzer and are displayed in the main menu; correct date and time following to 2.5.6 of the OM if necessary.

ATTENTION! The user should monitor the correctness of the current date and time and adjust them by themselves if necessary during the operation of the analyzer.

2.7 Operating procedure in the measuring mode with data input

The user should be ensured before the work, that conditions specified in 2.1 of the OM, are fulfilled; then, he should complete the analyzer operating preparation following 2.6.2 of the OM, and inform the examined person about the measuring procedure and the rules of exhalation (according to 2.7.11 of the OM).

- 2.7.1 Turn the analyzer on following to 2.2 of the OM.
- 2.7.2 Choose the measuring mode with data input by touching the
- icon. The data input line and virtual keyboard will appear (see Figure 11).

Note: If the metrological verification interval is over and, at the same time, the blocking after its termination is activated (see 1.4.8 of the

OM), the message "Verification is necessary" will appear. For further use of the analyzer, it should be submitted on a periodic metrological verification.

2.7.3 Enter the data via keypad or virtual keyboard.

When the user clicks the virtual key "ABB" the virtual keyboard layout changes from the Cyrillic alphabet to the Latin alphabet with numerals (see Figure 11).

When the user clicks the virtual key ("DEL"), the last entered symbol in the data entry line will deleted.

Latin & numeral

Figure 11 - Data input windows for the measuring mode with data input
(The field "Name of the Examined Person")

Note: The entered data of 2, 4, 5, and 6 fields (option 1 of the factory settings according to 2.3.6.3 of the OM) or 3, 5, and 6 fields (option 2 of the factory settings according to 2.3.6.3 of the OM) are stored in the memory until new data will be input, and they appear in the data entry line at once. The previously saved data can be adjusted by deleting the previous value and entering a new one, which will also be stored until the next change.

2.7.5 To go to the previous field, press the virtual key

ATTENTION! If the user goes to the previous field, the entered in the current field data will be lost except one that is stored in memory.

2.7.6 When all the fields are entered, a message box will appear on the touchscreen: "Insert a clean mouthpiece" (see Figure. 12, left).

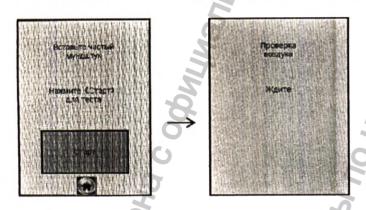
Figure 12 - The measuring procedure start

2.7.7 After verifying that the primary package of the mouthpiece is not damaged, open one.

ATTENTION! If the primary packaging of the mouthpiece was damaged, don't use the mouthpiece.

Note: When carrying out several consecutive measurements of the breath alcohol mass concentration, it is allowed to use the mouthpiece repeatedly for one examined person.

2.7.8 Set a mouthpiece as follows: holding the mouthpiece with the bottom holes down under the 45° angle to horizontal, make the fixing protrusion under the abutment in the upper part of the analyzer housing (see Figure 1, Figure 2), then lower the mouthpiece until it stops.

Check the correct insertion of the mouthpiece visually (look toward the pusher of mouthpiece): both of the input fittings of the analyzer should be completely inserted in the mouthpiece bottom holes.

2.7.9 To start the measuring procedure press the virtual key "Start".

At the pressing, the ambient air sample will be taken for its checking on ethanol and other cross-reactive components in the ambient air, the sampling system of the analyzer, or the mouthpiece (see Figure 12, right):

- if the ethanol and other components are absent, the analyzer will go to the state of readiness for sampling (see the window on Figure 13, right);

- if the ethanol and another components are present, the analyzer will not go to the state of readiness for sampling; the message "Alcohol detected" appears; in this case, user should act following 0 of the OM.

2.7.10 In the state of readiness for sampling the examined person should complete the exhalation following 2.7.11 of the OM.

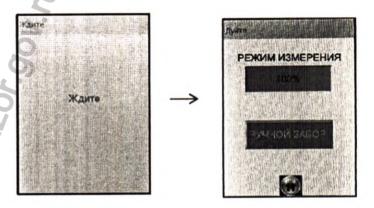
Figure 13 – The state of readiness for sampling

If the exhalation was not made for 30 seconds after the appearance of the state readiness window, the user should act following 2.7.19 of the OM to return to the state of readiness for sampling.

To interrupt the measurement procedure to fixation the refusal of measurement (when the examined person refuses to make an exhalation) user should act following 2.9 of the OM.

2.7.11 The examined person should exhale into the input hole of the mouthpiece (see Figure 2).

The exhalation should be quiet and uniform (without forcing), and it should last about 4-5 seconds (before the end of the reference sound). If the examined person provides the breath volumetric flow not less than 20 l/min, then during an exhalation, the reference buzzer signal sounds,

and the remained exhalation time is displayed as a percentage on the exhalation progress bar (see Figure 14).

At the end of the reference sound signal, the analyzer will automatically take a breath sample, after which it is advisable to stop exhalation.

To provide the deep-lung air analysis, the automatic sampling is made close to the end of the exhalation.

Figure 14- The exhalation progress bar

2.7.12 If the breath sample does not meet the required volumetric flow or volume (see 1.2.6 of the OM), namely, if the exhalation is interrupted, the message "Exhalation interrupted" will appear. In this case, the exhalation may be repeated according to 2.7.20 of the OM.

To interrupt the measurement procedure to fixation the refusal of measurement, the user should act following 2.7.21 of the OM.

2.7.13 If the examined person cannot exhale the sample with the required volumetric flow or volume (see 1.2.6 of the OM) for physiological reasons, it is advisable to use a manual sampling mode according to 2.8 of the OM.

2.7.14 After the breath sampling, the measurement of breath alcohol mass concentration takes place, the message "Analyzing. Please, Wait..." appears on the touchscreen (see Figure 15 left).

After the measurement is finished, the measurement result is shown on the touchscreen (see Figure 15 right).

If the auto-printing is activated, according to 2.5.4.1 ÷ 2.5.4.4 of the OM, printing starts automatically after the measurement result presentation.

2.7.15 If the user needs to print a measurement protocol, he should

press the virtual key

0

Note: The measurement protocol is printed according to the printing settings (see 2.5.4 of the OM).

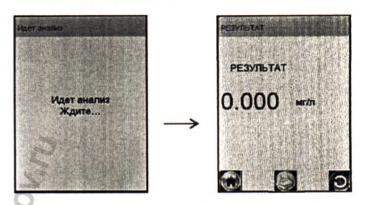
Figure 15 - The measurement result presentation

2.7.16 The user should remove the used mouthpiece and dispose it of following the requirements specified in 4.3.3 of the OM.

2.7.17 To go to the next measurement, press the virtual key and repeat the measuring procedure following 0 ÷0 of the OM.

To return into the main menu, the user should press the virtual key

For the work finishing, the user should turn the analyzer off by long-pressing the on/off button before the message "POWER OFF" will appear on the touchscreen.

2.7.18 If at accomplishment 2.7.9 of the OM in a mouthpiece, ambient air, or sampling system an alcohol or other, cross-reactive components will be found, the message "Alcohol Detected" will appear on the touchscreen (see Figure 16, right).

To go to the state of readiness for sampling, the user should do the following:

- return to the measuring procedure start by pressing the virtual key

in this case all input data will not be lost;

- ensure that the operating rules specified in 2.1.3 of the OM are respected, as alcohol could get into the mouthpiece from the ambient air;

- repeat the 0 procedure step of the OM;
- if the message "Alcohol Detected" will appear once more, repeat the procedure steps following 2.7.7÷0 of the OM.

Figure 16 – In a mouthpiece, ambient air, or sampling system, an alcohol is found

ATTENTION! As long as the alcohol is present in the mouthpiece or the sampling system, the analyzer will not go to the state of readiness for sampling.

2.7.19 If the exhalation was not done in time according to 2.7.10 of the OM, the window of readiness for sampling "Blow" is replaced by the window "No exhalation" (see Figure 17, left) in 30 seconds (see Figure 17, right).

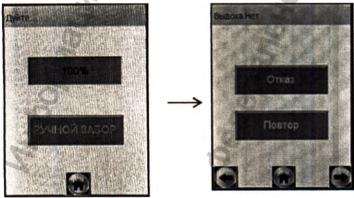
Figure 17 - Change of windows in the absence of exhalation

To resume the measuring procedure, press the virtual key "Repeat", in this case, all input data will not be lost.

2.7.20 If the breath sample does not meet the required volumetric flow or volume (see 1.2.6 of the OM) during the procedure according to 2.7.11 of the OM, namely if the exhalation is interrupted, the message "Exhalation interrupted" will appear (see Figure 18, right).

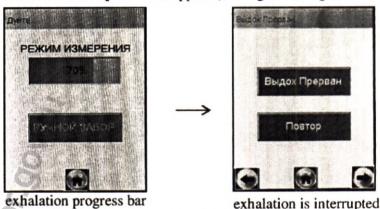
Figure 18-Exhalation is interrupted

It is advisable to stop exhaling. Press the virtual key "Repeat" to resume the measuring procedure. After the analyzer will go to the state of readiness for sampling, (see Figure 13, right) the examined person should repeat the exhalation following 2.7.11 of the OM.

2.7.21 If the examined person performs several exhalations with the volumetric flow or volume inadequate for automatic sampling, there is a possibility to interrupt the measuring procedure and to fix the fact (facts) of interrupting, if necessary.

To do this, when the window "Exhalation interrupted" is displayed (see Figure 18, right), touch the virtual key "Exhalation interrupted" on the touchscreen, and the appropriate message will appear (see Figure 19).

In this case, the inscription "Exhalation is interrupted" is remained in the analyzer memory instead of the measurement result. It indicates an exhalation attempt, which did not result in automatic sampling and subsequent analysis; in the printed measurement protocol, the message "Exhalation is interrupted" appears instead of measurement result; the line with the breath sampling mode is absent.

Figure 19 - The result "Exhalation is interrupted".

To continue, follow the instructions 2.7.15÷2.7.17 of the OM.

2.8 Manual sampling mode

2.8.1 If the examined person cannot perform the adequate exhalation to start the automatic sampling, it is advisable to use the manual sampling mode.

To do it, press the virtual key "Manual sampling" during exhalation (see Figure.13), and the analyzer will perform a manual sampling. Make manual sampling close to the exhalation end, what the examined person can.

2.8.2 The fact that the alcohol concentration in the breath is in direct ratio with the alcohol concentration in the venous blood allows the use of exhaled air as a diagnostic medium for the detection of alcohol in the blood. This ratio is a consequence of Henry's law, which states that the pressure of the volatile (gaseous) component in the solution and, consequently, the vapor content of this component above the solution surface is directly proportional to its content in the solution.

The closest contact with the blood has a lung air close to the alveolar, an analysis of which will be possible only with a deep exhalation (not less than 1,2 l volume). The air in the oral cavity or the upper respiratory tract has no close contact with venous blood.

That is why the automatic sampling mode is preferable for the measurement of breath alcohol mass concentration, which ensures compliance with the parameters (volumetric flow and volume) of air exhaled.

In the manual sampling mode the parameters of air exhaled (see 1.2.6 of the OM) are not provided.

The result of the breath alcohol mass concentration measurement in the manual sampling mode may be underestimated² compared with the result obtained in the automatic sampling mode due to the fact that in manual sampling mode the sample contains a greater proportion of air from the upper respiratory tract. The air from the upper respiratory tract is less saturated with ethanol than pulmonary air, sampled during automatic sampling mode.

2.9 Refusal of measurement

2.9.1 If the exhalation (see - of the OM) was not made in time, the window of readiness for sampling "Blow" will replaced by window "No exhalation" (see Figure 20, left).

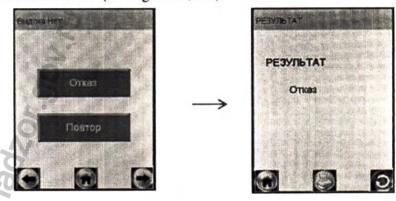
Figure 20 - The result "Refusal of measurement"

2.9.2 If the examined person refuses to exhale, the analyzer allows to interrupt the measurement procedure, with fixing the fact of refusal of measurement, if necessary. To do it the user should press the virtual key "Refusal" and the message about refusal will appear on the touchscreen (see Figure 20, right).

In this case, the inscription "Refusal of measurement" is remained in the analyzer memory. It indicates that there was no exhalation of the examined person; in the printed measurement protocol, the message "Refusal of measurement" appears instead of measurement result; the line with the breath sampling mode is absent.

To continue, follow the OM instructions $2.7.15 \div 2.7.17$.

² If there is no resident ethanol present on the mucosa of the oral cavity.

40

2.10 Operating procedure in the measuring mode without data input
The user should be ensured before the work, that conditions specified in 2.1 of the OM, are fulfilled; then, he should complete the analyzer operating preparation following 2.6 of the OM, and inform the examined person about the measuring procedure and the rules of exhalation
(according to 2.7.11 of the OM).

2.10.1 Turn the analyzer on following 2.2 of the OM.

2.10.2 Choose the measuring mode by pressing the icon The message will appear on the touchscreen: «Insert a clean mouth-piece» (see Figure 21, left).

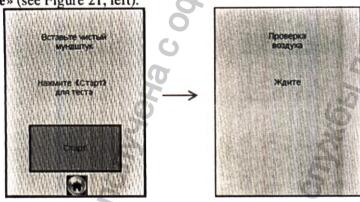
Figure 21 – Starting a measurement procedure in measurement mode without data entry

Note: If the metrological verification interval is over and, at the same time, the blocking after its termination is activated (see 1.4.8 of the OM), the message "Verification is necessary" will appear. For further use of the analyzer, it should be submitted on a periodic metrological verification.

- 2.10.3 Perform the measurement following paragraphs 2.7.7-2.7.17 of the OM.
 - 2.10.4 The empty protocol fields can be filled in by hand.

2.11 Operating procedure in the screening mode
The user should be ensured before the work, that conditions specified in 2.1 of the OM, are fulfilled; then, he should complete the analyzer operating preparation following 2.6 of the OM, and inform the examined person about the measuring procedure and the rules of exhalation

(according to 2.11.4 of the OM).

2.11.1 Turn the analyzer on following 2.2 of the OM.

2.11.2 Choose the screening mode by pressing the icon emessage about readiness for sampling will appear on the touchscreen (see Figure 22 left).

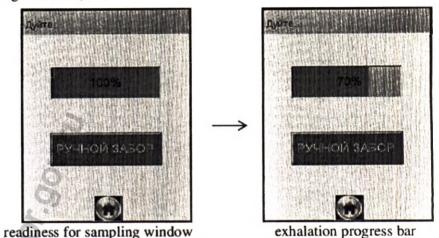
Figure 22 – The screening mode

Note: If the metrological verification interval is over and, at the same time, the blocking after its termination is activated (see 1.4.8 of the OM), the message "Verification is necessary" will appear. For further use of the analyzer, it should be submitted on a periodic metrological verification.

2.11.3 Set a mouthpiece cup as follows: holding the mouthpiece with the bottom holes down under the 45° angle to horizontal, make the fixing protrusion under the abutment in the upper part of the analyzer housing (see Figure 1, Figure 3), then lower the mouthpiece until it stops.

Check the correct insertion of the mouthpiece visually (look toward the pusher of mouthpiece): both of the input fittings of the analyzer should be completely inserted in the mouthpiece cup bottom holes.

2.11.4 The examined person should blow into the mouthpiece cup for 2 seconds continuously, strong, but without forcing to start the screening procedure. The mouthpiece cup should be positioned at a distance of $1.5 \div 2$ cm perpendicular to the examined person's face directly

in front of the mouth.

Note: For better automatic sampling, it is recommended to direct the breath flow into the elongated part of the mouthpiece cup (in front of the hole to the pressure transducer, arrowed in Figure 3).

2.11.5 If the examined person provides the appropriate breath volumetric flow, then during an exhalation, the reference buzzer signal sounds, and the remained exhalation time is displayed as a percentage on the exhalation progress bar (see Figure 22, right).

The exhalation should be stopped upon termination of a reference sound: at the moment, the analyzer will make automatic breath sampling.

ATTENTION! Before the manual sampling is finished, do not move the analyzer from the examined person's mouth. Otherwise, a sample of ambient air, rather than exhaled, will be sampled.

- 2.11.6 If the examined person cannot perform the adequate exhalation to start the automatic sampling, it is advisable to use the manual sampling mode according to 2.8 of the OM.
- 2.11.7 Depends on the result, one of two messages will appear (see Figure 23).

Figure 23 - The screening mode test results

ATTENTION! The screening mode can be used for estimation of breath alcohol presence only; the error may exceed the allowable limits of error according to the 1.2.1 of the OM.

If the breath alcohol presence is found, the user should complete one of two measuring modes following 2.7 or 2.10 of the OM.

2.11.8 To return into the main menu, press the virtual key

After completing the work, turn off the analyzer by long-term (about two seconds) pressing the on/off button until the message "POWER OFF" appears on the touchscreen.

2.11.9 To go to the next measurement, press the virtual key and repeat the measuring procedure following 2.11.4 ÷2.11.9 of the OM.

ATTENTION! If the analyzer is used for multiply measurements in the screening mode, at a low temperature especially, it is necessary to wipe the mouthpiece inside periodically to prevent a condensate (according to 3.4.7.3 of the OM)). The condensate infiltration into the sampling system can lead to fuel cell sensor failure.

2.12 Reviewing and printing of measurement protocols from the analyzer memory

2.12.1 The user should choose the item "Memory reviewing" of the main menu for the reviewing and printing of measurement protocols from the analyzer memory.

The window "Record" with the last measurement protocol will appear on the touchscreen (see Figure 24).

2.12.2 If the measurement has been made in the measuring mode with data input, the measurement results are stored with the automatically recorded data (date, time, etc.), and the manually entered data (see Figure 24, left).

Figure 24 - Memory reviewing

2.12.3 If the measurement has been made in the measuring mode without data input, the measurement results are stored with the automatically recorded data (see Figure 24, right).

- 2.12.4 The serial search of records of measurement protocols one by one is carried out by pressing the virtual keys and .
- 2.12.5 The record searching of measurement protocols in tens is carried out pressing the keys and .

Note: viewing records in the analyzer's memory (one by one or in tens tests) is carried out by the numbers of the measurement protocols, without taking into account the date and time of their performance. It should be noted, that after the analyzer has carried out more than 16000 measurements (tests), subsequent protocols are written into the analyzer's memory instead of those saved starting from 00001. For example, it may turn out that after the measurement protocol with the number 00001 with the date of 15.12. 2015, the next in the analyzer's memory will be the measurement protocol with the number 00002 with the date of 10.10.2014 (the data of the protocol with the number 00002 will be replaced with new ones only after another measurement).

- 2.12.6 To quickly jump to the first record, to the last record or the record with a given number, you should carry out the following steps:
- being in the window «**Record**» (see Figure 24), press the key, the window of transition to the searched protocol will appear (see Figure 25);

- to jump to first or last record press the virtual key "Start" or "End" respectively:

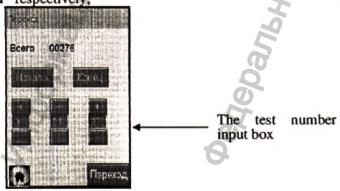
Figure 25 - Quick transition to the searched protocol

Note: after the analyzer has carried out more than 16000 measurements, pressing the "Start" virtual key switches to the protocol with the number 000001, pressing the "End" virtual key - to the last measurement protocol in memory.

- to jump to record with searched number, input the searched protocol number by «+» or «-» keys (the high-order digit of the number is indicated on the left), and press the virtual key "Transition". If the record with the input number does not exist, there is a transition to the last protocol number.
- 2.12.7 After pressing the virtual key , the displayed measurement protocol will be printed.

2.13 Printed measurement protocol content

2.13.1 The measurement protocol information according to Table 5 is printed (the example corresponds to the fields given in 2.3.6.2 of the OM).

Table 5

i aute 5		
The protocol inscription	The protocol content	
АЛКОТЕКТОР	The short name of medical device	
PRO-100 touch-K	(designation of the measuring instrument type)	
Номер Прибора:	The analyzer serial number	
Tect NO.:	The number of measurement (according to the	
9/	internal analyzer numbering)	
Дата: ДД/ММ/ГГГГ	The date of measurement (day/month/year)	
Время: ЧЧ:ММ	The date of measurement (hour: minute)	
Дата регулировки	The date of the last readings adjustment	
ДД/ММ/ГГГГ	(day/month/year)	
Дата поверки:	The date of the last metrological verification	
ДД/ММ/ГГГГ	(day/month/year)	
Режим: Автоматический	The sampling mode 1)	
Результат:	The result of the measurement of breath alcohol	
X.XX MT/II	mass concentration in the breath sample: nu-	
	meric value and unit designation «mg/l» 2)	
Имя Обследуемого: 6)	The surname and initials of the examined person 3)	
Место Обследования: 6)	The place of measuring 3)	
Гос. Номер Машины: 6)	Vehicle identification number 3)	
Нагрудный Знак: 6)	Inspector's ID 3)	
Инспектор: 6)	The surname and initials of the inspector 3)	

Отдел ДПС: 6)	The number of the Road Police Department 3)
Координаты:	The coordinates of the measurement site 4)
Подпись Обслед.:	The examined person signature 5)
Подпись:	The inspector signature 5)

Notes:

In the manual sampling mode the inscription "Mode: manual sampling" («Режим: РУЧНОЙ ЗАБОР») is printed.

²⁾ The inscription "Refusal of measurement" is printed in the case of the fixing fact of refusal of measurement; the inscription "Exhalation interrupted" is printed in the case of the fixing fact of interrupting of measurement. In this case, information on the mode of sampling is not shown in the protocol.

³⁾ The data are input via the virtual keyboard on the touchscreen before measuring or are entered by hand into the printed measurement protocol.

⁴⁾ The data is printed out in the measurement protocol when the positioning system signal is present, and the function of entering coordinates into the measurement protocol is activated.

5) The data are entered by hand into the printed measurement protocol.

⁶⁾ The set of filling fields may be different from the example in the table (the number of the fields is from 0 to 10, field names can be changed). Filling field names are indicated in the analyzer Technical Data Sheet.

2.14 The analyzer switching off

- 2.14.1 The analyzer is turned off by the long (about three seconds) pressing the on/off button of the analyzer (until to appearance of the message "POWER OFF").
- 2.14.2 The analyzer's automatic shutdown function is designed to conserve the charge of rechargeable power unit. The user can set the period of automatic switching off by themselves (following 2.5.3 of the OM).

2.15 Possible faults of the analyzer

Probable faults of the analyzer, as well as ways of their elimination, are given in Table 6.

Table 6

No	Probable faults	Probable cause	Way of elimination
1	The analyzer		Charge/set the rechargeable
	does not turn on		power unit following 3.4.2.2 of the OM or con-
			nect the AC power adapter to analyzer

2	Real-time clock stopped (the	2. The contact of the rechargeable power unit with the power supply compartment is poor 3. The analyzer is faulty 1. The voltage of the lithium battery is	Put out and put in the re- chargeable power unit into the power supply com- partment following 3.4.2.3 of the OM and repeat turn- ing on The analyzer needs to be sent to the service center* Replace lithium battery according to 3.4.4 of the
	current date and time cannot be set)	2. The chip of the real-time clock is faulty	OM The analyzer needs to be sent to the service center*
3	The analyzer does not make automatic sam- pling of breath	The volumetric flow or volume of an exhaled air is low The analyzer is faulty	Repeat the exhalation fol- lowing 2.7.11 of the OM The analyzer needs to be sent to the service center*
/4 _/	There is no sound accompa- nying the ana- lyzer operation	The analyzer is faulty	The analyzer needs to be sent to the service center*
5	The text in the measurement protocol is	1. The charge of the rechargeable power unit is low	Charge the rechargeable power pack following 3.4.2 of the OM
	printed pale or nonexistent	2. The pressure of the paper with the clamping rubber roller of the printer is low	Make sure that the clamping rubber roller of the printer is clear; clear it if necessary; close the printer cover firmly. If there is damage to the pressure roller, replace it **
		3. The thermal pa- per roll is set incor- rectly, or the unsuit- able paper is used	Make sure that a suitable paper is used, and check that it is installed correctly (following 3.4.3.4.3 of the OM)

		4. The printer is faulty	The analyzer needs to be sent to the service center*
6	The measure- ment protocol is not printed after	1. The printing settings are incorrect	Set the necessary number of copies following 2.5.4.2 of the OM
	the result dis- playing	2. The printer is faulty	The analyzer needs to be sent to the service center*
7	At turning on of the analyzer the message of "Temperature Extreme high (low)" is dis- played and the analyzer is	The measuring board temperature is above +50° C (below -5° C)	Sustain the analyzer in the appropriate operating conditions (following 1.1.1 of the OM) and repeat turning on
8	switched-off The analyzer does not go to the state of the readiness for sampling after the pressing of the "Start" key; the message "Alcohol de-	1. There is an alco- hol in the mouth- piece or in the sam- pling system 2. The analyzer is faulty	Act following 0 of the OM The analyzer needs to be sent to the service center*
9	When the analyzer is turned on, the message "Detected" appears briefly	There is an alcohol in the ambient air or the analyzer sampling system The analyzer is faulty	Make sure that the operating rules according to 2.1.3 of this OM are followed; switch analyzer off, and switch one on in 2-3 minutes The analyzer needs to be sent to the service center*
10	The analyzer does not respond to a vir-	1. The touchscreen calibration is lost	Complete the touchscreen calibration following 3.4.4 of the OM

2. The analyzer is faulty	The analyzer needs to be sent to the service center*

* During the warranty period, the analyzer should be sent to OOO "AL-COTEKTOR"; after the end of the warranty period, the analyzer should be sent to OOO "ALCOTEKTOR", or to a service center which has all necessary permits. Information about the service centers is given in 6.3 of the OM.

** The clamping rubber roller is delivered on an additional order.

3 TECHNICAL MAINTENANCE

- 3.1 The technical maintenance of the analyzer is carried out to maintain the analyzer in good condition and ensure its constant readiness for operation. It can be performed by the user or service center.
- 3.2 When carrying out maintenance of analyzers to ensure safety, including environmental safety, the requirements of regulatory documents on labor protection and safety measures, including those contained in the operating documentation used for maintenance of tools, equipment, and devices, must be observed.
 - 3.3 The technical maintenance of analyzer includes:
 - -on-site technical maintenance (by user);
 - periodic technical maintenance (by service center).
 - 3.4 On-site technical maintenance
- 3.4.1 On-site technical maintenance is carried out by the user as needed.

The technical maintenance may be due to the analyzer checking results before starting work, which is carried out during a visual inspection and preparing the analyzer for operation according to 2.6.2 of the OM. It includes:

- the charging of analyzer rechargeable power unit at low voltage;
- the refill the printer with paper;
- the lithium battery replacement: as needed;
- the touchscreen calibration: as needed (if the analyzer does not respond or respond incorrectly to a virtual keys click);
- visual inspection to check the legibility of the marking, the presence of all fasteners and seals, and checking the correctness of the printer; it is recommended before the metrological verification, or more often, according to the frequency set by the user;

- cleaning and disinfection of the analyzer: as it gets dirty (the frequency is set by the user).
 - 3.4.2 Rechargeable power unit charging
- 3.4.2.1 To check the rechargeable power supply voltage, the user should go to the analyzer's main menu.

If the charge level is too low, only a small part of the power indicator in the main menu remains shaded; when it is lower than allowed, the analyzer will turned off. In this case, user should charge the analyzer rechargeable power unit.

Note: The analyzer turning off due to low power can occur at any stage of its work, so watch out for the power indicator in the main menu and charge the rechargeable power unit beforehand.

- 3.4.2.2 To charge the rechargeable power unit user should follow the instructions:
 - a) turn the analyzer off;
- b) connect AC power adapter or vehicle power adapter from the set of analyzer to power connector;
 - c) connect power adapter for charger to power supply main;

the indicator light near the analyzer power connector will turn red charging should be continued until the light will turn green;

- d) disconnect power adapter from power supply main;
- e) disconnect power adapter from printer.
- 3.4.2.3 Rechargeable power unit removing/setting
- 3.4.2.3.1 To remove the rechargeable power unit user should follow the instructions:
 - a) open the cover of power supply compartment; to do this, first turn the fixing screw of the cover of power supply compartment cover 90 degrees counterclockwise, then remove the cover of power supply compartment;
- b) pull the rechargeable power unit up, holding on to the specific tab on the rechargeable power unit (there is a recess for it at the bottom of the analyzer power compartment), and remove the rechargeable power unit from the power compartment;
- c) set the cover of power supply compartment; to do this, first align the tabs on the cover with the internal slots in the analyzer case, then turn the fixing screw of the cover of power supply compartment 90 degrees clockwise.

- 3.4.2.3.2 To set the rechargeable power unit user should follow the instructions:
- a) open the cover of power supply compartment; to do this, first turn the fixing screw of the cover of power supply compartment 90 degrees counterclockwise, then remove the cover of power supply compartment;
- b) set the rechargeable power unit into power supply compartment; to do this, first align the rechargeable power unit contacts with the analyzer contacts and put down one into the power supply compartment;
- c) set the cover of power supply compartment; to do this, first align the tabs on the cover with the internal slots in the analyzer case, then turn the fixing screw of the cover of power compartment 90 degrees clockwise.

3.4.3 Refill the printer with paper

- 3.4.3.1 In the printer is set the roll of thermal paper with overall dimensions according to 1.2.14 of the OM.
- 3.4.3.2 To load the thermal paper roll into the printer, follow the instructions:
 - a) pull the tabs on the printer cover to myself;
- b) set the new thermal paper roll, placing the roll as shown in fig. 26:
 - c) close the printer cover, press the cover until it clicks.

Note: The paper thermal side can be detected easily: user should quickly draw with a stylus on the paper. There will be a black trace on the thermal surface of the thermal paper.

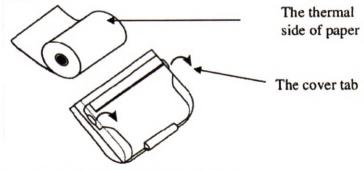
Figure 26- The refill the printer with paper

3.4.4 Lithium battery replacement

3.4.4.1 To replace the lithium battery user should follow the instructions:

- a) remove rechargeable power unit from the power supply compartment following 3.4.2.3.1 of the OM.
- b) displace the metal contact strip carefully, and remove the lithium battery from the slot;
- c) set new lithium battery CR 1220 3V type into the slot, mind the +/- ("+" contact from above), and press on top until it clicks;
- d) set the rechargeable power unit from the power supply compartment following 3.4.2.3.2 of the OM.
- 3.4.4.2 The lithium battery should be replaced if the real-time clock has stopped (the current date and time cannot be set)

3.4.5 Touchscreen calibration

To enter the touchscreen calibration mode user should following the instructions:

- a) press the on/off button and immediately, without releasing it, briefly press the button (in 0.5 sec);
- b) the white screen with the warning about touchscreen calibration mode will appear on the touchscreen;
- c) touch any point on the touchscreen by stylus: "CALIBRATION 3 touches left" instruction will appear in the center of the touchscreen;
- d) touch all the points which will be pointed in course on the touchscreen:

Note: touching should be neat but strong enough (for example, like when writing by ball pen on paper).

- e) after the touchscreen calibration will be completed, "Touch screen to continue" instruction will appear;
- f) touch any point on the touchscreen by stylus: the analyzer will come out of the touchscreen calibration mode, and the main menu will appear on the touchscreen.

3.4.6 Visual inspection

During visual inspection check:

- the presence of all fastenings and seals, as well as inscriptions and markings on the analyzer case; they must be clear, corresponding to the OM:
- check the serviceability of the printer; to do this, print the measurement protocol from the analyzer's memory following 2.12 of this OM to assess the print quality (prints of characters in the measurement protocol on paper should be easy to read).

If discrepancies are found, the analyzer should be sent for maintenance and / or repair.

3.4.7 Cleaning and disinfection of the analyzer

3.4.7.1 The analyzer's housing should be cleaned gently with a slightly damp cloth.

Abrasive chemicals should not be used for cleaning, since as this may damage the housing, touchscreen and/or fuel cell sensor.

During the cleaning, it is necessary to make sure that the dirt and dust in the input ports are absent. If the contamination is present, you should remove it with a dry lint-free napkin.

- 3.4.7.2 The cleaning of the clamping rubber roller of printer should be made with dry lint-free napkin.
- 3.4.7.3 During operation, the condensate removal from the mouthpiece cap should be made with a dry lint-free napkin (not impregnated by ethanol and other chemical cleaners).
- 3.4.7.4 The analyzer should be periodically sanitized under requirements MY-287-113, according to the standards established by the institution to use the device (the periodicity of the procedure is established as the contamination of the device during operation takes place, as it recommended for medical devices which have no contact with the human mucosa, according to the profile of the medical institution).

The sanitization procedure should be completed by chemical method under requirements "MY-287-113". To the sanitization procedure should be used as the method of double wiping by cloth of calico or gauze. The cloth should be moistened with a disinfectant solution. Then it should be squeezed so that the moisture has not got into the analyzer's housing, the cloth should be moistened by a disinfectant solution; after that, it should be squeezed so that the moisture has not got into the analyzer's housing. For the sanitization procedure, it allows to use of chemicals recommended for plastic, excluding substances containing alcohol, e.g. 3% hydrogen peroxide solution with the addition of 0.5% detergent type "Lotus".

The mouthpiece cup should be carefully naturally dried after disinfection so that the moisture has not got into the analyzer's housing.

ATTENTION! Mouthpieces should not be disinfected but are disposed of after use according to 4.3 of the OM.

3.5 Periodic technical maintenance

- 3.5.1 Periodic maintenance of the analyzer is performed by the service center and during the entire period of operation includes:
- technical condition control: is recommended before metrological verification:
- checking of readings³: is recommended before the metrological verification (or more often according to the frequency set by the user);
- readings adjustment² (under the negative results of checking of readings or before the metrological verification if necessary).
- 3.5.2 During the analyzer metrological verification, the readings are adjusted as necessary according to the instructions 6.2.3 of the standard MΠ-242-1956-2015 «Breath alcohol analyzer ALCOTEKTOR PRO-100 touch-K. Metrological Verification Procedure».
- 3.5.3 The analyzer maintenance operations are given in the document "Breath alcohol analyzer ALCOTEKTOR PRO-100 touch-K. Maintenance instructions. Engineering menu", provided upon additional request to service centers and organizations accredited to verify the breath alcohol analyzers.
- 3.5.4 Technical maintenance facilities required for periodic maintenance of the analyzer at service centers are listed in Appendix B.
- 3.5.5 It is recommended that the fact of checking or adjusting the analyzer readings be noted in the maintenance accounting table of the analyzer's Technical Datasheet, for example, with the entry "Readings Checking" ("Readings Adjustment").

ATTENTION! After adjusting the analyzer readings, it is imperative to verify the analyzer.

4 TRANSPORTATION, STORAGE AND UTILIZATION

4.1 Transportation

4.1.1 The analyzer in transport packaging is transported by all types of transport in covered vehicles according to the current carriage of goods rules applied to all kinds of transport.

Placement and fastening of packed analyzers in vehicles must ensure their stable position, exclude the possibility of impacts against each other, as well as against the walls of vehicles.

55

- ambient temperature range is from -50 to +50°C;
- upper value of relative humidity air: 100 % at +25 °C.
- 4.1.3 It is recommended to apply manipulation signs on the transport package corresponding to the values: "Fragile, careful!", "Protect from moisture".
- 4.1.4 After transportation under freezing temperatures, it is recommended to hold the analyzer under operating conditions for at least an hour before turning it on. If the "Low temperature" message appears on the analyzer screen when the analyzer is turned on, the holding time should be increased to two hours.

4.2 Storage

- 4.2.1 The storage of analyzer should be indoor under the following conditions:
 - ambient temperature range is from 5 to 40°C; relative humidity is no more than 80 % at 25 °C.
- 4.2.2 It is not allowed to store analyzers in places with increased dustiness and gas content, as well as in which alcohol-containing substances are stored in open containers, or surfaces (equipment) are treated with alcohol-containing solutions.
- 4.2.3 For long-term storage, remove the rechargeable power unit from the analyzer power supply compartment to prevent damage to the analyzer due to the risk of electrolyte leakage.
- 4.2.4 The rechargeable power unit should be stored charged (the optimal charge level is 40%). If the rechargeable power unit is left to be discharged for three months or longer, an unrecoverable decrease in its capacity or parts corrosion may occur.
 - 4.2.5 The analyzer shelf life is unlimited.
- 4.2.6 When commissioning after long-term storage (more than one verification interval), periodic metrological verification is performed/
- 4.2.7 Analyzers should be stored in packaging and placed on racks in no more than three rows.
 - 4.2.8 No temporary corrosion protection required.

4.3 Utilization

4.3.1 In avoidance of environment contamination the analyzer should not be disposed of with the municipal solid waste. It should be disposed of according to the $\mbox{Сан}\Pi\mbox{H}H\mbox{ }2.1.3684-21,\mbox{ class }\Gamma.$

³Checking and adjustment of the analyzer can be performed as an independent maintenance operation or as a part of the analyzer's technical condition monitoring procedure.

Electrical and electronic equipment should be disposed of through designated local authorities, not with the municipal solid waste.

4.3.2 In the medical institution used mouthpieces should of according to the СанПиН 2.1.3684-21, class Б.

5 METROLOGICAL VERIFICATION

- 5.1 According to article 13 of Federal law of 26.06.2008 № 102-ФЗ "On ensuring the traceability of measurements", "The intended for use in the field of the state regulation of ensuring the traceability of measurements measuring instruments, before commissioning and after repair are subject to initial metrological verification, and in operation are subject to periodic metrological verification. Both legal entities and individual entrepreneurs who use the measuring instruments in the field of the state regulation of ensuring the traceability of measurements must present them to metrological verification in time. Metrological verification of measuring instruments is carried out by legal entities and individual entrepreneurs accredited according to the legislation of the Russian Federation on accreditation in the national accreditation system for metrological verification of measuring instruments».
- 5.2 Metrological verification is provided according to the MΠ-242-1956-2015 «Breath alcohol analyzer ALCOTEKTOR PRO-100 touch-K. Metrological Verification Procedure», approved 23 October 2015 by D. I. Mendeleev All-Russian Institute for Metrology.
 - 5.2.1 The verification period is 1 year.
- 5.2.2 It is recommended to mark the fact of metrological verification in the maintenance table in the analyzer Technical Datasheet.
- 5.2.3 At the user request or if there are requirements in the regulatory documents governing the use of analyzers, when the date of verification is entered into the analyzer's memory, can be activated the blocking of analyzer's operation by the date of metrological verification.

When the function of blocking the analyzer's operation by the missed date of metrological verification is activated, it is necessary to make an appropriate note in the maintenance table of analyzers Technical Data Sheet.

6 MANUFACTURER'S WARRANTY AND ANALYZER REPAIR

6.1 Manufacturer's warranty

OOO ALCOTEKTOR, being an authorized representative of the manufacturer, accepts claims, complaints, and reviews on the quality of the analyzer and also bears warranty obligations according to the provisions of the corresponding section of the analyzer Technical Datasheet.

6.2 Warranty repair

Warranty repair of the analyzers is carried out in OOO "AL-COTEKTOR" located: 5-7, Naberezhnaya reki Smolenki, Saint-Petersburg, 199178, Russian Federation, p.o. box 256, www.alcotector.ru, e-mail: info@alcotector.ru..

6.3 Post-warranty maintenance and repair

Post-warranty maintenance and repair of the analyzer is carried out in OOO "ALCOTEKTOR", as well as in regional service centres.

An up-to-date list of service centres, with which OOO "AL-COTEKTOR" signed a cooperation agreement, is posted on the website www.alcotector.ru in the section "Service in the regions".

To provide services for the maintenance and repair of analyzers, the organization must have all the necessary permits according to the current legislation of the Russian Federation.

When contacting these organizations for the provision of maintenance, repair services ask about its availability.

APPENDIX A

(OBLIGATORY)

Regulatory References

The Operating Manual contains references to the following regulatory documents:

- 1 IEC 60601-1:2005 «Medical electrical equipment Part 1: General requirements for basic safety and essential performance» (ГОСТ Р МЭК 60601-1-2010 Medical electrical equipment Part 1: General requirements for basic safety and essential performance).
- 2 FOCT IEC 60950-1-2014 Information technology equipment. Safety. Part 1. General requirements.
- 3 MY-287-113 Methodical instructions for disinfection, presterilization cleaning and sterilization of medical devices.
- 4 Sanitary rules and norms CaHΠιΗ 2.1.3684-21 "Sanitary and Epidemiological Requirements for the Maintenance of the Territories of Urban and Rural Settlements, for Water Bodies, Drinking Water and Drinking Water Supply of the Population, Atmospheric Air, Soils, Living Quarters, Operation of Industrial, Public Premises, Organization and Implementation of Sanitary and Anti-epidemic (Preventive) Measures".
- 5 Federal law of 26.06.2008 № 102-Φ3 "On ensuring the traceability of measurements".

Note: – When using this OM, it is advisable to check the validity of reference standards in the public information system - on the official website of the Federal Agency on Technical Regulating and Metrology. If the reference standard is replaced (amended), then when using the OM, a replacement (modified) standard should be used. If the reference standard is canceled without replacement, then the corresponding thesis is applied in the part that does not affect this reference.

APPENDIX B

(OBLIGATORY)

Electromagnetic Compatibility Information

Medical Electrical Equipment needs special precautions regarding Electromagnetic Compatibility (EMC) and needs to be installed and put into operation according to the electromagnetic compatibility information provided in the OM.

Mobile radiofrequency devices can effect to electrical medical devices.

The use of accessories, converters, and cables not listed in the OM, except of converters and cables supplied by the manufacturer of the product or system as replacement parts for internal parts, may lead to an increase in electromagnetic emissions or a decrease in the immunity of the product or system.

Electromagnetic compatibility tables

Table B.1 – Manufacturer Guidance and Declaration – electromagnetic emissions

Manufacturer Guidance and Declaration - electromagnetic emissions		
The system is intended to be used in the electromagnetic environment specified		
below. The customer or the user should assure that it is used in the specified		
environment.		
Electromagnetic	Compliance	Electromagnetic Environment: Guid-
Emissions Test		ance
The group of medical equipment under	Group 1	The system uses radio frequency energy for its internal functions only.
СИСПР 11 (ГОСТ Р		RFI emissions are low and likely will
51318.11 - 2006)		not cause malfunctioning of nearby electronic equipment
The class of medical equipment under CИСПР 11 (ГОСТ Р 51318.11-2006)	Class B	The system is suitable for use in all establishments, including domestic es-
Harmonic current components under M3K 61000-3-2 (FOCT 30804.3.2- 2013)	Class A	tablishments, and buildings directly connected to a distribution power sup- ply network that supplies buildings used for domestic purposes
Voltage fluctuations	Complies	

/flicker emissions	,0
under M9K 61000-3-	
3 (FOCT 30804.3.3-	2
2013)	C

Table B.2 – Guidance and Manufacturer Declaration – electromagnetic immunity

Manufacturer Guidance and Declaration: electromagnetic immunity

The system is intended to be used in electromagnetic environment specified below. The customer or the user should assure that it used in the specified environment

Immunity test	Level of Test	Compli- ance level	Electromagnetic environment guidance	
	±6 kV - contact discharge	Complies	Floors in the room should be wood,	
Electrostatic discharges (ESD) under M9K 61000- 4-2 (FOCT 30804.4.2-2013)	±8 kV – air dis- charge	Complies	concrete, or ceramic- tile. If floors are covered with synthetic material, the relative humidity should be at least then 30%.	
Nanosecond pulse interferences under MЭK 61000-4-4 (ГОСТ 30804.4.4)	±2 kV - for power supply lines ±1 KB - for input/output lines	Complies	Mains power quality should be according to the typical commercial or hospital environment	
Microsecond pulse interferences of high energy under M9K 61000-4-5 (FOCT P 51317.4.5-99)	±1 kV when giving interferences according to the scheme "line to line " ±2 kV when giving interferences according to the scheme "line to earth"	Complies	Mains power quality should be is according to the typical commercial or hospital environment	
Voltage dips, short- term interruptions	<5% U _H (voltage failure >95%	Complies	Mains power quality should be is	

and voltage variations on	U _H) during half- period		according to the typical commercial
power supply input lines under M9K 61000-4-11 (FOCT 30804.4.11 - 2013)	40% U _H (voltage failure 60% U _H) during 5 periods 70% U _H (voltage failure 30% U _H) during 25 periods <5% U _H (voltage failure >95% U _H) during 5 sec		or hospital environment
Power frequency magnetic field (50/60 Hz) under M3K 61000-4-8 (FOCT P 50648 - 94)	3 A / M	Complies	Power frequency magnetic field levels should be provided according to the typi- cal commercial or hospital conditions.

Table B.3 – Guidance and Manufacturer Declaration – electromagnetic immunity

Manufacturer Guidance and Declaration – electromagnetic immunity IT IS NOT DESIGNED TO MAINTAIN VITAL FUNCTIONS

The system is intended to be used in the electromagnetic environment specified below. The customer or the user should assure that it is used in the specified environment

Immunity Device	Test Level	Level of	Specified electromagnet-
Test		Conformity	ic environment
			Portable and mobile radio-frequency equipment, incl. cables should be used near the analyzer no closer than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter:

33333 19.11.2021

Recommended separation distance				
Conducted interference induced by radiofrequency electromagnetic fields under M9K 61000-4-6 (FOCT P 51317.4.6-99)	3V from 150 kHz to 80 MHz	V1 - 3 (V)	$d = \left[\frac{3.5}{V_1}\right] \sqrt{P}$	
Radiofrequency electromagnetic fields under MЭK 61000-4-3 (ГОСТ R 30804.4.3-2013)	3 V/m from 80MHz to 2,5GHz	E1 - 3 (V/m)	$d = \left[\frac{3.5}{V_1}\right] \sqrt{P}$ from 80 MHz to 800 MHz $d = \left[\frac{7}{E_1}\right] \sqrt{P}$ from 800 MHz to 2.5 GHz	

where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer specifications, d is recommended separation distance in meters (m).

The electric field intensity under the propagation of radio waves from fixed radio frequency transmitters, as determined by an electromagnetic site survey a, should be less than the compliance level in each frequency band b

The interference influence may occur in the vicinity of equipment marked with the

a) The electric field intensity under the propagation of radio waves from fixed radio frequency transmitters (such as base stations for radio (cellular/cordless) telephones, and land mobile radio stations, amateur radio, AM and FM broadcast transmitters, television broadcast transmitters) cannot be predicted theoretically with enough accuracy.

For this, practical electric field intensity measurements should be made. If the measured electric field intensity in the location [in which Medical Electric System or Medical Electric Device is used] exceeds the applicable RF compliance level above, the user should observe its [Medical Electric System or Medical Electric Device] work to verify its correct operation. If abnormal performance is observed, additional measures may be necessary, such as re-orientating or relocating [Medical Electric System or Medical Electric Device].

b) Over the frequency range from 150 kHz to 80 MHz, electric field intensity should be less than (V1) V/m.

Table B.4 – The recommended separation distance between portable and mobile RF communications equipment and the system

Recommended separation distance between portable and mobile RF communications equipment and the system IT IS NOT DESIGNED TO MAINTAIN VITAL FUNCTIONS

The system is intended to be used in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the system as recommended below, according to the maximum output power of the communications equipment

	Separation distance according to the frequency of transmitter (m)			
Maximum output rated	150 kHz ÷ 80 MHz	80 MHz ÷ 800 MHz	800 MHz ÷ 2,5 GHz	
power of transmitter (W)	$d = \left[\frac{3.5}{V_1}\right] \sqrt{P}$	$d = \left[\frac{3.5}{E_1}\right] \sqrt{P}$	$d = \left[\frac{7}{E_1}\right] \sqrt{P}$	
0.01	0,12	0,12	0,23	
0,1	0,37	0,37	0,74	
g = 1	1,17	1,17	2,33	
10	3,69	3,69	7,38	
100	11,67	11,67	23,33	

Notes:

¹ At 80 and 800 MHz the separation distance for the higher value is applied.

² These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

³ When determining the recommended separation distances d for transmitters with a rated maximum output power not listed in the table, substitute the rated maximal output power, P, in watts, from the transmitter manufacturer's documentation in the expressions given.

APPENDIX C

(REFERENCE)

Technical Maintenance Facilities

Table C.1 - Technical Maintenance Facilities

	Designation of a normative document regulating tech-
lame and type of the Tech-	nical requirements and (or) metrological and base tech-
ical Maintenance Facilities	nical characteristics of the Technical Maintenance Fa-
	cilities
esto 622 Scientific Ambient Monitor*	Specified measuring range of temperature is from +10 to +30 °C; allowable limits of intrinsic absolute error are 3 0.5 °C. Specified measuring range of relative humidity is from 30 to 80 %; allowable limits of intrinsic absolute error are ± 3 %. Specified measuring range of absolute pressure is from 80 to 110 kPa, allowable limits of intrinsic absolute error are ± 0.5 kPa.
The alcohol breath test simula- or* Guth model 10-4D complete	gas mix ethanol mass concentration in the range from 40 to
with the standard samples of composition of water solutions of	80 mg/m³ are ± 4 mg/m³. Allowable limits of intrinsic relative error of reproduction o
thanol VRE-2 GSO 8789-2006	gas mix ethanol mass concentration in the range from above
or	80 to 2000 mg/m 3 are \pm 5%.
he standard samples of composi-	
ion of the gas mix etha- nol/nitrogen in cylinders under pressure* (hereinafter referred to as GM in cylinders under pres-	pressure are given in table C.2.
sure) according to TU 6-16-2956- 22: GSO 10338-2013	
Rotameter *	The upper limit of measurements of volumetric flow is
RM-1,6 GUZ	m ³ /h; limits of intrinsic relative error of the upper limit of measurements are ± 2.5 %.
Digital Multimeter*	The specified measuring range of DC voltage is from or 0 A
APPA-72	60 V, the specified measuring range of DC от 0 до 6 A. Al
	lowable limits of intrinsic absolute error of measurements of
	DC voltage are $\pm (0.005 \cdot U_{mw} + 0.02) \text{ V}$. Allowable limits of intrinsic absolute error of measurement
Q	of DC are $\pm (0.01 \cdot I_{\text{user}} + 0.002)$ A.
Compressor with flow controller	
Distilled water	FOCT 6709-72 room with supply and exhaust ventilation, providing an air ex

C.1 It is allowed to use other facilities not listed in table C.1, but providing the required measurement accuracy.

C.2 All technical maintenance facilities marked *, should have a valid certificate of verifycation; GM in cylinders under pressure and GM in cylinders under pressure should have a valid technical datasheet.

C.3 When carrying out the periodic maintenance of analyzers, gas mix (hereinafter referred to as GM) with nominal ethanol mass concentration values listed in table C.2 are used.

65

Регистрация МИ в Росздравнадзоре www.nevacert.ru | info@nevacert.ru Table C.2 - GM Metrological Characteristics

.m Otte stantionopie	di Cildi deteriotien	
GM nominal ethanol mass concentration values, allowable lim- its of deviation, mg/l	Nominal ethanol mass con- centration values of water solutions of ethanol ¹⁾ , allowable limits of deviation, mg/cm ³	Nominal ethanol mass con- centration values of GM in cylinders under pressure ²⁾ , allowable limits of deviation, mg/l
0,00	distilled water	air
0.150 ± 0.015	0.386 ± 0.039	$0,150 \pm 0,015$
0.475 ± 0.048	$1,22 \pm 0,12$	0.475 ± 0.048^{31}
0.850 ± 0.085	2.19 ± 0.22	0.850 ± 0.085
	GM nominal ethanol mass concentration values, allowable limits of deviation, mg/l 0,00 0,150 ± 0,015 0,475 ± 0,048	GM nominal ethanol mass concentration values, allowable limits of deviation, mg/l 0,00 distilled water 0,150 ± 0,015 0,386 ± 0,039 0,475 ± 0,048 Nominal ethanol mass concentration values of water solutions of ethanol ¹ , allowable limits of deviation, mg/cm ³ distilled water

¹⁾ The standard samples of composition of water solutions of ethanol VRE-2 GSO 8789-2006. The limits of the reliable error are ± 1 % under P=0,95.

The standard samples of composition of the gas mix C2H5OH/N2 in cylinders under pressure: GSO 10338-2013. The limits of the reliable error are \pm 2 % under P=0.95.

When carrying out the periodic maintenance of analyzers through the gas mix C2H5OH/N2 in cylinders under pressure, it allows to use the gas mix C2H5OH/N2 in cylinders under pressure with ethanol mass concentration from 0.33 to 0.52 mg/l as GM N3.

APPENDIX D (REFERENCE)

Weight and Dimension Characteristics

Characteristics	Value
Mo	outhpiece
Length	Not more than 111 mm
Transverse overall dimension	Not more than 16 mm
Weight	Not more than 5 g
001-82139963-2015), исполнение «Мунд [Mundshtuk po TU 22.29.29-001-82139	29-001-82139963-2017 (идентичны ТУ 2291- ритук АЛКОТЕКТОР с двумя отверстиями» 963-2017 (identichny TU 2291-001-82139963- LCOTEKTOR s dvumya otverstiyami]
Length	Not more than 111 mm
Transverse overall dimension	Not more than 16 mm
Weight	. Not more than 5 g
Mout	thpiece cup
Length	Not more than 53 mm
Transverse overall dimension	Not more than 40 mm
Weight	Not more than 5 g
Ther	mal paper
Width	Not more than 56±2 mm
External diameter	Not more than 24±2 mm
Weight	Not more than 25 g
	able power unit
LxWxH	Not more than 82x67x22 mm
Weight	Not more than 200 g
	wer adapter
LxWxH (of the adapter body)	Not more than 100x60x85 mm
Length of the cable	Not more than 3 m
Weight	Not more than 300 g
	power adapter
LxWxH (of the adapter body)	Not more than 100x35x35
Length of the cable	Not more than 3 m
Weight	Not more than 150 g
	бель USB
Length of the cable	Not more than 3 m
Weight	Not more than 150 g
	ch (folded)
LxWxH	Not more than 250x100x50 mm
Weight	Not more than 250 g
., ., .,	Кейс
LxWxH	Not more than 500x400x200 mm
Weight	Not more than 300x400x200 filli
W CIZIII	Not more than 2,0 kg

APPENDIX E (REFERENCE)

Signs and Symbols Used in Marking

©	Metrological approval sign	
EHC	Eurasian conformity mark	
ර්	On/off button	
沐	Symbol of device with applied part type B	
IP20	International protection symbol	
	Measuring mode menu icon	
	Virtual navigation key (to go to the next menu window)	
₩	Enter key on the keypad	
12 V=== 2A	Rated value of supply characteristics and type of current	
	Name and address of the manufacturer;	
②	"Do not re-use" symbol	
NON	"Non sterile" symbol	
®	"Do not use if package is damaged" symbol	
\triangle	"Caution! Consult Instructions For Use" symbol	
Ţi	"Consult Instructions For Use" symbol	
سا	Year of manufacture	

LOT	Lot number		
	Graphic warning "do not burn"		
C€	CE marking		
Li-ion	Symbol for batteries, accumulators and battery packs separate collection under the Directive 2006/66/EC		
£3	General symbol for recovery/recyclable		
	Symbol of IEC protection Class II		
	Symbol for indoor use only		
XX.	WEEE symbol for waste electrical and electronic equipment under the Directive 2012/19/EU		
⊝	Positive polarity designation		
RoHS	Symbol of the restriction of the content of hazardous substances		

.,		Numbers (pa	ges)		In total sheets		Incoming number of		
N chang c.	chan ged	re- placed	new	can- celled	(pages) in the docu- ment	Number of document	accompany document and date	Signature	Date
1	-	2-91	92-95	-	95		Ref. No. 125 dated 19.10.2021	T.N. Goryachkina	
		5							
		5							
		<u> </u>							
9									

83333 19.11.2021 APPRUVED BY:
Director of the D.I. Mendeleev All-Russian Institute for Metrology (VNIIM)
N.I. Khanov

23/10/2015

Breath alcohol analyzers ALCOTEKTOR PRO-100 touch-K

Metiological Verification Procedure

MP-242-1956-2015

Head of the Department of Measurement Standards and Investigations in the field of Physical-Chemical Measurements

of D.I. Mendeleev VNIIM

L.A. Konopelko, Lead Engineer O.V. Fatina

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3333 9.11.2021 The present Metrological Verification Procedure extend to the Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K (further – analyzers) intended for the fast measurement of breath alcohol mass concentration, and set the methods of their primary metrological verification and periodic metrological verification during operation.

The verification period is 1 year.

1 METROLOGICAL VERIFICATION PROCEDURE OPERATIONS

1.1 When carrying out metrological verification, the operations specified in table 1 are performed.

Table 1 Verification procedure operations

	Paragraph	Operations are performed during:		
Name of operation	number of Metrological Verification Procedure	Primary verifica- tion before com- missioning	Primary verifica- tion after repair 19 and periodic verification	
1 Visual inspection	6.1	yes	yes	
2 Assay	6.2			
- checking of the general functioning	6.2.1	yes	yes	
- checking of the automatic mode of breath sampling	6.2.2	yes	yes	
- readings adjustment ^a	6.2.3	yes	yes	
software compliance confirmation	6.2.4	yes	yes	
3 Determination of metrological characteris- tics	6.3		9	
 determination of an error at (20 ± 5) °C temperature of ambient air: 	6.3.1	no	yes	
 determination of an error at ambient air temperature in operating conditions; 	6.3.2	yes	no	

¹⁾ In case that the temperature sensor installed on the fuel cell sensor board has been replaced, or the adjustment of the coefficients of thermal compensation was carried out, during the primary verification after repair of analyzer the operations indicated in "Primary verification before commissioning" column should be carried out.

the password is required to enter the menu. The manufacturer's authorized representative in Russia OOO ALCOTEKTOR (Address for correspondence: Naberezhnaya reki Smolenki, 5-7, 199178, Saint-Petersburg, e-mail: info@alcotector.ru, tel./fax: (812) 320-22-97) provides a password on request to accredited in the field of metrological verification organizations.

1.2 If, when carrying out any verification procedure operation the negative result is received, further verification is stopped.

2 METROLOGICAL VERIFICATION FACILITIES

2.1 When carrying out verification, the facilities listed in Table 2 are used.

Table 2 Metrological verification facilities

Paragraph number of Metrological Verification facility; designation of the nor- mative document regulating technical requirements and (or) metrological character- istics (MC) and main technical characteristics of verification facility The equipment for measuring of temperature, relative humidity, and absolute pres- sure, for example: The equipment for measuring of temperature is from +10 to +30 °C; allowable limits of intrinsic absolute error are ± 0.5 °C. Specified measuring range of temperature is from +10 to +30 °C; allowable limits of intrinsic absolute error are ± 0.5 °C. Specified measuring range of relative humidity is from 30 to 80 %; allowable limits of intrinsic absolute error are ± 3 %. Specified measuring range of absolute pressure is from 80 to 110 kPa, allowable limits of intrinsic absolute error are ± 0.5 kPa. The standard null gas air ¹⁰ brands B on TV 6-21-5-82 in a cylinder under pressure or purisarium speciale gas nitrogen 1 or 2 class on FOCT 9293–74 in a cylinder under pressure. RM-0.63 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 0.63 m³/h, limits of the al- lowed relative error ± 2,5% of upper limit of measurement. RM-1.6 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 0.63 m³/h, limits of the al- lowed relative error ± 2,5% of upper limit of measurement. Precise control valve BTP-1 or BTP-1-M160 Range of working pressure is from 0 to 150 kgfs/cm³, diameter of conditional pass is 3 mm. Medical polyvinylchloride tube, 6x-1,5 mm. Working standards of the 1st or 2nd range according to FOCT 8.578–2014: a) The alcohol breath test simulator* Guth model 10-4D complete with the stand- ard samples of composition of water solutions of ethanol VRE-2 GSO 8789-2006 (MC are given in table B.1 of Appendix B). Limits of the allowed relative error: ±5%. or b) Standard samples of composition of gas mix ethanol/nitrogen in cylinders under pressure (further – GM in cylinders under pre		
The equipment for measuring of temperature, relative humidity, and absolute pressure, for example: Testo 622 Scientific Ambient Monitor providing the following metrological characteristics: Specified measuring range of temperature is from +10 to +30 °C; allowable limits of intrinsic absolute error are ±0.5 °C. Specified measuring range of relative humidity is from 30 to 80 %; allowable limits of intrinsic absolute error are ±3 %. Specified measuring range of absolute pressure is from 80 to 110 kPa, allowable limits of intrinsic absolute error are ± 0.5 kPa. The standard null gas air "b brands B on TY 6-21-5-82 in a cylinder under pressure or purissimum speciale gas nitrogen 1 or 2 class on FOCT 9293–74 in a cylinder under pressure or purissimum speciale gas nitrogen 1 or 2 class on FOCT 9293–74 in a cylinder under pressure or purissimum speciale gas nitrogen 1 or 2 class on FOCT 9293–74 in a cylinder under pressure or before the second of the allowed relative error ±2.5% of upper limit of measurement. RM-0.63 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 0,63 m³/h, limits of the allowed relative error ±2.5% of upper limit of measurement. RM-1.6 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 1,6 m³/h, limits of the allowed relative error ±2.5% of upper limit of measurement. Precise control valve BTP-1 or BTP-1-M160 Range of working pressure is from 0 to 150 kg/s/cm², diameter of conditional pass is 3 mm. Medical polyvinylchloride tube, 6×1,5 mm. Working standards of the 1st or 2nd range according to FOCT 8.578–2014: a) The alcohol breath test simulator* Guth model 10-4D complete with the standard samples of composition of water solutions of ethanol VRE-2 GSO 8789-2006 (MC are given in table B.1 of Appendix B). Limits of the allowed relative error: Error 0 °C to 40 °C, and the overall internal dimensions of the chamber should ensure the placement of a verification procedure, in a room with supply and ex	number of Metrological Verification	mative document regulating technical requirements and (or) metrological character-
6.3 Distilled water. FOCT 6709–72 6.3.2 The climatic chamber of any type ²¹ , TXB-150 for example. The temperature maintenance accuracy is ± 2 °C. The temperature maintenance range should ensure the reproduction of the temperatures from 0 °C to 40 °C, and the overall internal dimensions of the chamber should ensure the placement of a verified analyzer. 11 During the verification procedure, in a room with supply and exhaust ventilation according to 3.1 of the Methods, it is allowed to use compressed air according to FOCT 17433-80, instead of air or nitrogen in a cylinder under pressure. 2 The climatic chamber should be used for verification if the operation according to the 6.3.2	4050 4050 4050 4050 4050 4050 4050 4050	sure, for example: Testo 622 Scientific Ambient Monitor providing the following metrological characteristics: Specified measuring range of temperature is from +10 to +30 °C; allowable limits of intrinsic absolute error are ± 0,5 °C. Specified measuring range of relative humidity is from 30 to 80 %; allowable limits of intrinsic absolute error are ± 3 %. Specified measuring range of absolute pressure is from 80 to 110 kPa, allowable limits of intrinsic absolute error are ± 0,5 kPa. The standard null gas air ¹⁾ brands B on TY 6-21-5-82 in a cylinder under pressure or purissimum speciale gas nitrogen 1 or 2 class on FOCT 9293–74 in a cylinder under pressure FM-A-0,16 rotameter GUZ according to FOCT 13045–81. of volumetric flow is 0,16 m³/h, limits of the al-lowed relative error ± 2,5% of upper limit of measurement. FM-0,63 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 0,63 m³/h, limits of the al-lowed relative error ± 2,5% of upper limit of measurement. RM-1,6 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 1,6 m³/h, limits of the al-lowed relative error ± 2,5% of upper limit of measurement. RM-1,6 rotameter GUZ according to FOCT 13045–81. The upper limit of measurements of volumetric flow is 1,6 m³/h, limits of the allowed relative error ± 2,5% of upper limit of measurement. Precise control valve BTP-1 or BTP-1-M160 Range of working pressure is from 0 to 150 kgts/cm², diameter of conditional pass is 3 mm. Medical polyvinylchloride tube, 6×1,5 mm. Working standards of the 1st or 2nd range according to FOCT 8.578–2014: a) The alcohol breath test simulator* Guth model 10-4D complete with the standard samples of composition of water solutions of ethanol VRE-2 GSO 8789-2006 (MC are given in table B.1 of Appendix B). Limits of the allowed relative error: ±5%. or b) Standard samples of composition of gas mix ethanol/nitrogen in cylinders under pressure (further – GM in cylinders under pressure) on TY 6-16-2956
The temperature maintenance accuracy is ± 2 °C. The temperature maintenance range should ensure the reproduction of the temperatures from 0 °C to 40 °C, and the overall internal dimensions of the chamber should ensure the placement of a verified analyzer. **During the verification procedure, in a room with supply and exhaust ventilation according to 3.1 of the Methods, it is allowed to use compressed air according to FOCT 17433-80, instead of air or nitrogen in a cylinder under pressure. **The climatic chamber should be used for verification if the operation according to the 6.3.2	6.3	
¹⁾ During the verification procedure, in a room with supply and exhaust ventilation according to 3.1 of the Methods, it is allowed to use compressed air according to ΓΟCT 17433-80, instead of air or nitrogen in a cylinder under pressure. ²⁾ The climatic chamber should be used for verification if the operation according to the 6.3.2	6.3.2	The temperature maintenance accuracy is $\pm 2^{\circ}\text{C}$. The temperature maintenance range should ensure the reproduction of the temperatures from 0 $^{\circ}\text{C}$ to 40 $^{\circ}\text{C}$, and the overall internal dimensions of the chamber should ensure the placement of a
	of the Metho nitrogen in a ²⁰ The clim	verification procedure, in a room with supply and exhaust ventilation according to 3.1 ds, it is allowed to use compressed air according to FOCT 17433-80, instead of air or cylinder under pressure. natic chamber should be used for verification if the operation according to the 6.3.2

2.2 The application of metrological verification facilities, not listed in table 2, is allowed if characteristics of facilities are not worse specified in table 2.

²⁾ These verification procedure operations are carried out under the engineering menu;

2.3 All metrological verification facilities should have a valid certificates on verification, GM in cylinders under pressure and standard samples of composition of water ethanol solutions should have a valid technical datasheet.

3 SAFETY REQUIREMENTS AND REQUIREMENTS FOR THE QUALIFICATION OF PERSONS

- 3.1 The room in which verification is carried out should be equipped with supply and exhaust ventilation, providing an air exchange rate of at least 4 per hour.
- 3.2 Concentration of harmful components in air of a working zone (in addition to ethanol) should conform to requirements of FOCT 12.1.005-88.
- 3.3 When carrying out verification safety rules according to FOCT 12.2.007.0–75 and "Rules for the Construction and Safe Operation of the vessels working under pressure", approved by Gosgortekhnadzor should be followed.
- 3.4 Allow the persons acquainted with FOCT 8.578-2008, FOCT P 8.676-2009, and the analyzer Operation Manual (further OM) with the qualification of the verification specialist, the valid certificate and which passed a personnel safety notification according to the established procedure.

4 VERIFICATION CONDITIONS

- 4.1 When carrying out verification, meet the following conditions:
- range of ambient air temperature is from 15 to 25°C;
- range of a relative humidity of an ambient air is from 30 to 80%;
- range of atmospheric pressure is from 84 to 106 kPa:
- ethanol mass concentration in an ambient air is not more then 0,010 mg/l.

Note: When carrying out verification through alcohol breath test simulators, additionally consider requirements to the operating range of ambient air temperature specified in their OM.

- 4.2 When operating under 6.2.3 and 6.3 paragraphs of the Metrological Verification Procedure, it is not allowed to supply into analyzer the GM from alcohol breath test simulator and the GM ethanol/nitrogen from cylinders under pressure (table 2) alternately.
- 4.3 When carrying out verification through alcohol breath test simulator, the following conditions should be met for standard samples of composition of water solutions:
 - the bottle with solution is opened just before use;
 - the solution is used for single filling in the simulator;
- the solution should be replaced, if the maximum number of generated samples without replacement indicated in the simulator OM is exceeded; or if the maximal residence time of the solution in the generator, specified in the Technical Datasheet of the standard samples, is exceeded:— after use, the solution should not be stored or reused.

5 PREPARING FOR VERIFICATION

- 5.1 The analyzer should be prepared for use following the paragraph "Preparing the Analyzer for Operation" of its OM, including checking and, if necessary, adjusting the current preset date and time, if necessary.
- 5.2 The metrological verification facilities should be prepared to operating following the requirements of their operational documentation.
- 5.3 GM in cylinders under pressure and standard samples of composition of water solutions of ethanol should be checked for the presence of technical data sheet and expiration date validity. The availability and integrity of protective labels on the bottles with standard samples of the composition of water solutions of ethanol also should be checked.
- 5.4 The cylinders with GM should be kept in the verification room not less than 24 h, the analyzer should be kept in the verification room not less than 2 h. It is not allowed to supply into analyzer breath or GM for at least 1 hour.

75

6 VERIFICATION

6.1 Visual inspection

During the visual inspection, the compliance the following requirements should be checked:

- analyzer is presented for verification with its OM, technical datasheet, and a certificate of previous metrological verification.
 - there are no external damages affecting performance and safety;
 - controls and connectors are serviceable;
- inscriptions and markings on the analyzer body are clear; they are corresponding with OM:
 - preset date and time are correct .

Note: If, during the previous verification of the analyzer, instead of issuing a verification certificate, a verification mark (verification stamp) was put in the technical datasheet, then instead of having a verification certificate, the presence of a verification mark in the analyzer's technical datasheet is checked.

The results of the visual inspection are positive if the analyzer meets the listed requirements.

6.2 Assay

6.2.1 The general functioning checking.

- 6.2.1.1 Checking the general functioning of the analyzer is carried out by switching on following the manual, while the analyzer performs automatic diagnostics of operability.
- 6.2.1.2 During the initial verification, prior to commissioning, the operation of the printer is checked according to the OM, the measurement protocol is printed on paper in order to assess the print quality and check the displayed information.

Note: A printout of the measurement protocol in order to assess the print quality and check the displayed information is allowed to be performed after measurements have been carried out following 6.2.2.2 list item d).

- 6.2.1.3 The results of checking the general functioning of the analyzer are recognized positive if all tests of automatic diagnostics of the analyzer's operability have been completed successfully according to OM and, if the operation following 6.2.1.2 was performed, the following was established:
- the analyzer's printer is operational, the prints of the characters in the measurement protocol on paper are clear, easy to read;
- the information displayed in the paper measurement protocol corresponds to the OM of the analyzers; analyzer serial number, date, and time of measurement are correct.

6.2.2 Checking of functioning of the automatic mode of breath sampling.

6.2.2.1 Checking of the general functioning is carried out by sequential air supplying from a cylinder under pressure into analyzer input under different volumetric flow and checking of automatic GM sampling. The air supplying is carried out through the mouthpiece from the analyzer set.

Note: when performing the verification operation following 6.2.2, measurements by means of an analyzer should be carried out in the measuring mode in the automatic mode of breath sampling.

6.2.2.2 Checking is carried out in the following sequence:

- a) open the cylinder with air and set the volumetric flow 16 l/min by the precise control valve, under control of rotameter; disconnect the rotameter;
- b) Turn the analyzer on following OM and, when it will be ready for sampling, supply into it an air from the cylinder under pressure; while the analyzer should not perform automatic sampling of the GM:
- c) open the cylinder with air and set the volumetric flow 24 l/min by the precise control valve, under control of rotameter; disconnect the rotameter;
- d) Turn the analyzer on following OM, and, when it will be ready for sampling, supply into it the air from the cylinder under pressure; while the analyzer should perform automatic sampling of the GM.
- 6.2.2.3 The results of the checking of functioning of the automatic mode of breath sampling are positive, the analyzer meets the requirements indicated in 6.2.2.2 of the procedure.

⁴ This verification condition is recognized meeting when the verification room has supply and exhaust ventilation according to 3.1 of the procedure.

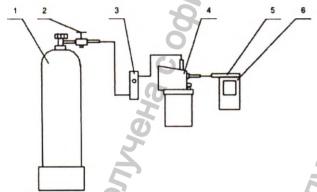
6.2.3 Readings adjustment

6.2.3.1 Before correcting the readings, three measurement cycles are carried out by feeding the GM No. 3 to the analyzer input (Table B.1 of Appendix B) and registering the analyzer readings.

Measurements are performed following 6.2.3.2 or 6.2.3.3 of this procedure, depending on the selected verification facility.

6.2.3.2 Measurements through the alcohol breath test simulators:

a) Assemble gas system which scheme is represented in figure 1. At assembly of the gas system, it is necessary to locate the simulator so that direct sunshine did not fall on it, and close there were no sources of cooling or heating. The length of breath alcohol simulator output tube should be no longer 5 cm. Before pouring the solution into the simulator, check the absence of moisture and condensation on the inner surfaces of the simulator, connecting tubes and mouthpieces; if moisture or condensation is present, it is necessary to dry all the elements of the simulator, connecting tubes and mouthpieces.

1 – gas air in a cylinder under pressure; 2 – valve; 3 – rotameter; 4 – alcohol breath test simulator; 5 – mouthpiece of analyzer set; 6 – analyzer

Figure 1 – The gas system scheme; the gas mix is supplied to the analyzer from the breath alcohol test GUTH model 10-4D.

- b) GMs are prepared following the OM of the simulator, using an appropriate aqueous ethanol solution according to Table B.1 of Appendix B.
- c) Calculate the actual value of the GM ethanol mass concentration on the simulator output \mathbb{C}^A , mg/l, by the formula:

$$C_i^A = 0.38866 \cdot C_p^A$$

where C_p^A - the certified ethanol mass concentration value of in the used standard samples of composition of water solutions of ethanol, specified in the passport, mg/cm³.

d) Measurements are carried out following the scheme (for each ith GM, three measurement cycles are carried out):

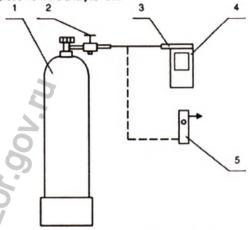
turn the analyzer on following its OM;

- select the measuring mode without input data following the OM; press the "START" virtual key to go to the state of readiness for sampling;
- when the analyzer is disconnected, open the cylinder and set the GM volumetric flow 6 l/min by the precise control valve, under the control of rotameter PM-0,63 GUZ;
- when the analyzer will ready for sampling, connect the analyzer to the simulator and supply GM into it;
- in 3-4 sec, carry out the manual sampling of GM by pressing the virtual key "Manual sampling" on the touchscreen;
 77

- in 1 sec after the sampling disconnect the analyzer and close valve on the cylinder;
- $-\,$ record the $\boldsymbol{C}_i\,$ analyzer reading (mg/l) and go to the next measurement according to the OM:
 - the interval between measurement cycles should not be less than 10 sec.

6.2.3.3 Measurements through the standard samples of composition of gas mix ethanol/nitrogen in cylinders under pressure:

a) Assemble gas system which scheme is represented in figure 2. The connecting tube should not be longer than 10 sm. The GM should be supplied into the analyzer input through the mouthoiece from the analyzer set.

1 - cylinder with GM; 2 - valve; 3 - a mouthpiece from an analyzer set; 4 - analyzer; 5 -rotameter

Figure 2 - The gas system scheme; the gas mix is supplied to the analyzer from the cylinder under pressure.

b) The measuring is carried out in the dry gas sample measuring mode (for each ith GM, three measurement cycles are carried out):

- turn the analyzer on following its OM;
- go into the "user settings" menu following the OM;
- when the analyzer is disconnected, open the cylinder and set the GM volumetric flow 1l/min by the precise control valve, under the control of rotameter PM-0,16 GUZ;
 - connect the analyzer, and supply GM into it;
- in 3-4 sec carry out the manual sampling of GM by pressing the virtual key "Checking" on the touchscreen;
 - after 1 sec after the sampling disconnect the analyzer and close valve on the cylinder;
 - record the \$\Pi\$, analyzer reading (mg/l);
- go to the next measurement by touching the virtual key "to the left" or "to the right" at the touchscreen bottom;
 - the interval between measurement cycles should not be less than 10 sec.
- calculate the measured value of the ethanol mass concentration in the i^{th} GM $\,C_{i}$, $\,\text{mg/l}$ (for every GM except GM No1) by the formula

$$C_i = \Pi_i \cdot \frac{P_0}{P} \,. \tag{2}$$

Where

 P_0 - is atmospheric pressure, equals 101,3 kPa;

P - is measured atmospheric pressure value, kPa

6.2.3.4 Checking the fulfillment of the admission condition for readings adjustment

According to the measurement results obtained when supplying GM No. 3 for each measurement cycle following 6.2.3.2 or 6.2.3.3 of this procedure, check the fulfillment of the following condition:

$$|C_i - C_i^{\alpha}| \le 0.07.$$

Where

C_i - is measured ethanol mass concentration value in the ith GM, mg/l;

C_A - is the actual ethanol mass concentration value in the ith GM, (during the verification procedure through simulators is calculated by formula (1); during the verification through the GM in cylinders under pressure is taken from the cylinder technical datasheet), mg/l;

If the condition 3 is met, the analyzer should be allowed to readings adjustment according to 6.2.3.5–6.2.3.7 of the procedure, and, further, to verification.

If the condition 3 is not met, the analyzer should not be allowed to readings adjustment according to 6.2.3.5–6.2.3.7 of the procedure; further verification is stopped.

The reason for the unsuitability is indicated in the analyzer's failure notice: "The analyzer does not meet the requirements 6.2 "Assay" of MP-242-1956-2015 "Breath alcohol analyzers AL-COTEKTOR PRO-100 touch-K. Metrological Verification Procedure".

The readings adjustment of the analyzer is allowed not to be performed if, according to the measurement results obtained when supplying GM No.3 for each measurement cycle by following per 6.2.3.2 or 6.2.3.3 of the procedure, the following condition is met:

$$|C_i - C_i^{\mathcal{A}}| \le 0.03 \tag{4}$$

6.2.3.5 Course of readings adjustment of analyzer

The readings adjustment of the analyzer is carried out following 6.2.3.6 or 6.2.3.7 of the procedure according to the metrological verification facility by supplying GM No3 (see table B.1, Appendix B) into the analyzer.

Note: It is prohibited to perform the readings adjustment procedure of the analyzers when the analyzer verified on behalf of the courts, prosecutors, arbitration courts, and federal executive bodies.

6.2.3.6 Procedure of readings adjustment of analyzer through the alcohol breath test simulators:

a) Complete the operations following 6.2.3.2, list itenms a) - c);

b) Turn the analyzer on following its OM and go to the engineering menu by password input (see table 1). Select the "Cal" virtual key; then select "Wet Gas" in the window of GM type selection; push the "Adjust" virtual key; enter the simulator output GM ethanol mass concentration value C_i^A calculated by formula (1), mg/l, in the "X.XXX" format; then press the virtual key "to the right" at the touchscreen bottom.

c) Check the temperature of the fuel cell sensor board, indicated in the second line of "AD-JUST: wet gas" window; it should be from 20 °C to 27 °C.

Note: If the temperature value of the fuel cell sensor board indicated in the second line of "ADJUST: wet gas" window is below 20 °C or above 27 °C, turn the analyzer off and keep it in the verification room for an hour.

d) Carry out the measurement following the further procedure:

 when the analyzer is disconnected, open the cylinder with air, and set the GM volumetric flow 6 l/min by the precise control valve, under the control of the rotameter PM-0.63 GUZ;

- connect the analyzer to the simulator and supply GM into it;

- in 3-4 sec, carry out the manual sampling of GM by pressing the "Cal" virtual key on the touchscreen;
 - in 1 sec after the sampling disconnect the analyzer and close valve on the cylinder;
- in 10-15 seconds, press the central virtual key at the touchscreen bottom to return to the main menu.
- 6.2.3.7 Procedure of readings adjustment of analyzer through GM in cylinders under pressure:
- a) Assemble gas system following figure 2. The connecting tube should not be longer than 10 sm. The GM should be supplied into the analyzer input through the mouthpiece from the analyzer set.
- b) Turn the analyzer on following its OM and go to the engineering menu by password input (see table 1). Select the "Cal" virtual key; then select "Dry Gas" in the window of GM type selection; push the "Adjust" virtual key; enter the simulator output GM ethanol mass concentration value calculated by the formula (5):

$$C_{i}^{p} = C_{i}^{p} \cdot \frac{P}{P_{0}}, \tag{5}$$

Where $C_i^{\mathcal{A}}$ - is certified ethanol mass concentration value of GM in cylinders under pressure, noted in technical datasheet, mg/l.

Input the ethanol mass concentration value in "X.XXX" format; then press the virtual key "to the right" at the touchscreen bottom".

c) Check the temperature of the fuel cell sensor board, indicated in the second line of "AD-JUST: dry gas" window; it should be from 20 °C to 27 °C.

Note: If the temperature value of the fuel cell sensor board indicated in the second line of "ADJUST: wet gas" window is below 20 °C or above 27 °C, turn the analyzer off and keep it in the verification room for an hour.

d) Carry out the measurement following the further procedure:

- when the analyzer is disconnected, open the cylinder with GM, and set the GM volumetric flow 1 l/min by the precise control valve, under the control of the rotameter PM-0.16 GUZ:

- connect the analyzer, and supply GM into it:

- in 3-4 sec, carry out the manual sampling of GM by pressing the "Cal" virtual key on the touchscreen;
 - in 1 sec after the sampling disconnect the analyzer and close valve on the cylinder;
- in 10-15 seconds, press the central virtual key at the touchscreen bottom to return to the main menu.

6.2.4 Confirmation of software compliance

6.2.4.1 Confirmation of compliance of the internal system-application software is carried out visually by number identification of the software version: turn the analyzer on and record the number identification of the software version brought to the display at turning on of analyzers.

6.3.2 Results of check of compliance of the software are recognized positive if version number of the firmware of analyzers not below RU V1.28.

6.3 Determination of metrological characteristics

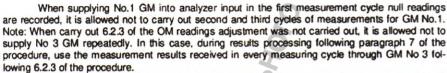
6.3.1 Determination of an error at 20 ± 5 °C temperature of ambient air

6.3.1.1 The determination of an error is carried out in four points of the specified measuring range (further – verification points) by the alternate GM supplying of into analyzer and recording of readings of the analyzer.

The base metrological characteristics are listed in the annex A.

6.3.1.2 The GM are supplied into the analyzer in the sequence NN 1-2-3-4-1 (table B1 of the annex B).

6.3.1.3 In every verification point carry out three measurement cycles by supplying into analyzer ith GM and recording readings of the analyzer according to 6.2.3.2 or 6.2.3.3 depending on the selected means of verification.

- 6.3.2 Determination of an error at ambient temperature corresponding to operating conditions
 - 6.3.2.1 Determination of an error is carried out in two stages:
 - on the first stage determine the error at (20 ± 5) °C temperature of the ambient air;
- on the second stage, determine the error at the temperature of ambient air, corresponding to the lower and upper levels of the operating conditions of the analyzer.
- 6.3.2.2 Determination of an error at the temperature of ambient air (20 ± 5) °C is carried out following 6.3.1 of the procedure.
- 6.3.2.3 Determination of an error at the temperature of ambient air corresponding to the lower and upper levels of the operating conditions of the analyzer is carried out by keeping the analyzer in the climatic chamber and supplying the GM No3 into the analyzer input (Table B.1 of Apendix B).

Measurements should be carried out in the following sequence:

- a) put the analyzer into the climatic chamber and set the temperature 48 °C; keep the analyzer in the chamber at this temperature not less than 2 hours;
- b) carry out three measurement cycles by supplying the GM No3 into the analyzer input and recording of analyzer readings following 6.2.3.2 or 6.2.3.3 (depending on selected verification facility); remove the analyzer from the climatic chamber only for the time of supplying GM - no more than 15 s; keep the analyzer between the measurement cycles in the climatic chamber for at least 5 minutes:
- c) put the analyzer into the climatic chamber and set the temperature -3 °C; keep the analyzer in the chamber at this temperature not less than 2 hour;
- d) carry out three measurement cycles by supplying the GM No3 into the analyzer input and recording of analyzer readings, following 6.2.3.2 or 6.2.3.3 (depending on selected verification facility); remove the analyzer from the climatic chamber only for the time of supplying GM - no more than 15 s; keep the analyzer between the measurement cycles in the climatic chamber for at least 5 minutes:.

Note: When the climatic chamber has stabilized, the rate of change of the air temperature in its working volume should not exceed 1° / min.

7 PROCESSING OF MEASUREMENT RESULTS

7.1 Base on the results obtained following 6.3.1 n 6.3.2 of the procedure in every verification point for every measurement cycle, the absolute error value or the relative error value is calculated depending on the normalized type of error for this verification point.

The analyzer absolute error value Δ_i , mg/l, under supplying i^{th} GM is calculated by the formula

$$\Delta_{i} = C_{i} - C_{i}^{A}, \tag{6}$$

where C_i - is measured alcohol mass concentration in supplying i^{th} GM, mg/l;

 C_1^A – is actual value of alcohol mass concentration in ith GM (when simulators are used, calculated by formula (1), when GM ethanol/nitrogen in cylinders under pressure are used, indicated in passport), mg/l.

The analyzer relative error value δ_i , %, at supplying i^m GM, is calculated by the formula

$$\delta_i = \frac{C_i - C_i^A}{C_i^A} \cdot 100 \tag{7}$$

7.2 The results of analyzer error determination are recognized positive if the obtained values of an error of the analyzer in each point of checking on each cycle of measurements do not 81

8 REGISTRATION OF VERIFICATION RESULTS

8.1 If the results of the verification are positive, the analyzer is recognized suitable for use; a verification certificate is issued for it in the established form according to the actual regulatory legal acts of the Russian Federation. The recommended verification protocol form is given in Appendix B. The form of the verification certificate backside is given in Appendix G.

The verification mark is put in the verification certificate or analyzer technical datasheet (when primary verification is carried out).

If the results of the verification are positive, to input the verification date into the analyzer memory, go to the engineering menu by password input (see table 1). Then select the "Verification" item; in the "Verification" window press the "Verification" virtual key; in the following window input verification date (the verification date should be input in xxyyzz format, where xx – date, yy – month, zz – year), and press the virtual key "to the right" up to exit into the main menu.

8.2 If the results of the verification are negative, the analyzer is recognize unsuitable for use; the verification certificate is canceled; the notice of unsuitability is Issued for it in the established form according to the actual regulatory legal acts of the Russian Federation, indicating the reasons for unsuitability.

APPENDIX A (obligatory) Base metrological characteristics of analyzers

Table A.1 – Specified measuring range and allowable limits of intrinsic error of analyzer at

Allowable limits of Intrinsic Error at 20 ± 5℃ temperature range:		
absolute error	relative error	
± 0,020 mg/l	- (2)	
_	± 10 %	
	at 20 ± 5℃ tem absolute error	

Table A.2 - Allowable limits of error under the ambient air .temperature

9	Allowable limits of Error 1)			
Temperature range of ambient air	absolute error (specified measuring range from 0,000 to 0,200 mg/l)	relative error (specified measuring range from 0,200 to 1,200 mg/l)		
from minus 5,0 °C to 5,0 °C inclusive	± 0,040 mg/l	± 20 %		
above 5,0 °C to 15,0 °C вкл.	± 0,030 mg/l	±15%		
above 15,0 °C to 25,0 °C inclusive	± 0,020 mg/l ²⁾	± 10 % ²⁾		
above 25,0 °C to 50,0 °C inclusive	± 0,030 mg/l	±15%		

In the table are given allowable limits of error under the operating conditions.

2) See table A.1.

APPENDIX B (obligatory) Metrological characteristics of gas mix used in the metrilogy verification procedure

Table B.1 - Metrological characteristics of gas mix used in the metrilogy verification procedure

GM Number	Nominal value of GM mass ethanol concentration supplied into the analyzer, limits of an allowable deviation, mg/l	Nominal value of ethanol mass concentration in composition of water solutions ¹ , limits of an allowable deviation, mg/cm ³	Nominal value of GM ethanol mass concentration in cylinders under pressure ²⁹ , limits of an allowable deviation, mg/l
No1 GM	0	distillated water	air
No2 GM	0,150±0,015	0,386±0,039	0,150±0,015
No3 GM	0,475±0,048	1,22±0,12	0,475±0,048 ³
No4 GM	0.850±0.085	2,19±0,22	0.850±0.085

When carried out the metrological verification of the analyzer through Alcohol Breath Test Simulators with standard samples of composition of water solutions of ethanol VRE-2: GSO 8789-2006. Borders of relative error under P=0.95 are ± 1 %.

33333 19.11.2021

When carried out the metrological verification of the analyzer through standard samples of composition of gas mix C_2H_5OH/N_2 in cylinders under pressure: GSO 10338–2013. Borders of relative error under P=0.95 are ± 2 %.

When carried out the metrological verification of the analyzer through standard samples of composition of gas mix C2H5OH/N2 in cylinders under pressure it is allowed to use GM in cylinder under pressure with mass concentration of ethanol from 0,33 to 0,52 mg/l instead of GM No3

APPENDIX C (recommended) Form of the protocol of metrological verification of analyzers

METROLOGICAL VERIFICATION PROTOCOL

No on
1) Name of the analyzer, type
2) Serial number
3) Owner
4) Date of issue
5) Registered number in Federal information fund for support of unity of measurements.
6) Name of the normative document on verification
7) Metrological verification facilities ⁵
- unit standard
(specify the standard name and registration number 3)
(simulator type and serial number, verification certificate date and number)
- standard samples of composition of water solutions of ethanol
(specify the registration number and serial numbers of standard samples)
(specify the registration number and serial numbers of standard samples)
- standard samples of composition of gas mix cyhanol/nitrogen in cylinders under pressure
(specify registration number3, serial numbers of used cylinders, technical datasheet numbers and validity
periods)
- climatic chamber
- Clariatic Charitoci
(specify type, serial number, number and validity period of the certificate on attestation)
(specify type, serial number, number and valuely period of die certificate on attestation)
8) Verification type (primary/periodic)
(Underline whatever applicable)
9) Verification conditions:
- ambient air temperature
- relative humidity of an ambient air
- atmospheric pressure
10) Verification results
Visual inspection
Assay
Checking of the general functioning
Checking of functioning of the automatic mode of breath sampling
Confirmation of software compliance

83333 19 11 20

Determination of metrological characteristics

Tempera- ture of an ambient air, °C	Cpecified Measu- ring range, мг/л	Allowable limits of Error		Actual value of the GM ethanol mass	Measured value of the GM ethanol mass	Error value obtained during verification	
		Absolute	Relative	concentration, mg/l	concentration, mg/l	Absolute mg/l	Relative mg/l

	3						
			-		-		-

Conclusion	, serial No	
MANAGO MA	(Name, type, modification of mea	asuring instrument)
complied (does not con (unsuitable for use).	nplied) to the requirements and is recog	mize suitable for use
Verificator	(Signature)	(Initials, surname)

⁵ Specify used metrological verification facilities.

Specify registration number in Federal information fund for support of unity of measurements.

85

APPENDIX D

(OBLIGATORY)

Form of the back side of certificate of verification

Visual inspection re Assay results Results of determin Error determination	ation of metrologic	minimum management			
Specified measur- ing range,	Allowable limits of at 20 ± 5°C temper		Maximum error value obtained during verification		
mg/l	absolute error	relative error	absolute error	relative error	
0 - 0,200	± 0,020 MF/л	6			
bove 0,200 - 1,200		± 10 %	-	(fD)	
Note: - Allowable temperature values a	re given in the Op	perating Manual and	d Technical Data Sh	eet of the analyzer.	
4.2 Error determination level of the operating c		aperature of ambient a	ir corresponding the	lower and the upper	
		Day abla limits of	Maximun	n value of abso-	

Ambient air temperature	Allowable limits of absolute/relative error	Maximum value of abso- lute/relative error obtained during verification
minus 3 °C		
cluc 49 oc		

Verificator		
	(Signature)	(Initials, surname)

 $^{^{7)}\,}$ The verification certificate contains this paragraph if the operation following 6.3.2 of the procedure is carried out.

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China Council for the Promotion of International Trade China Chamber of International Commerce

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Technical Data Sheet

Director

Shenzhen Well Electric Company Limited

_ Chan Yung

21 October 2021

83333 19.11.2021

Breath Alcohol Analyzers ALCOTEKTOR PRO-100 touch-K

Technical Data Sheet

1 General

The Breath alcohol analyzer ALCOTEKTOR PRO-100 touch-K (hereinafter Analyzer) is intended to be used as a device for fast measurements of breath alcohol mass concentration.

There are no contraindications and side effects.

The registration certificate of Federal Service for Surveillance in Healthcare No. Φ C3

The number in the Federal information fund for support of unity of measurements is No. 63415-16.

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The manufacturer's authorized representative: OOO ALCOTEKTOR.

Legal address: 10 A, 2-N, 1-st Sovetskaya St., St. Petersburg, 191036, Russian Federation.

Address for correspondence: 5-7, Naberezhnaya reki Smolenki, St-Petersburg, 199178, Russian Federation, p.o. box 256, ph. (812) 320-22-97, www.alcotector.ru.

2 Metrological and technical characteristics

2.1 The specified measuring range and allowable limits of intrinsic error of analyzer under ambient temperature range from +15 to +25°C inclusive are given in table 1.

lable

Specified measuring range of breath alcohol mass con-	Allowable limits of Intrinsic Error under +15 to +20 °C inclusive temperature range:		
centration, mg/l	Absolute error	Relative error	
from 0 to 0,200 inclusive	± 0,020 mg/l	-	
above 0,200 to 1,200	- (/)	± 10 %	

Note: In the analyzers is programmed the minimal result interval which is indicated and printed as null readings: from 0.00 to 0.020 mg/l.

2.2 Readings range:

from 0,000 to 2,500 mg/l.

Note:

I In case of the readings of the analyzer exceeding the upper limit of measurements, the mass concentration of breath ethanol is more than 1,000 mg/l.

2 The readings of the analyzer exceeding upper limit of indication are indicated and printed as «>2.500 mg/l».

2.3 The scale interval is: 0,001 mg/l.

2.4 The allowable limits of intrinsic error under the operating temperature range of ambient air are given in table 2.

Table 2

Temperature range of ambient air	Breath alcohol mass concentration range,	Allowable limits of Error 1)		
of amoient air	mg/l	Absolute error	Relative error	
	0 - 0.200 incl.	±0,040 mg/l		
from -5,0 °C to +5,0 °C inclusive	above 0,200 - 1,200	-	± 20 %	
	0 - 0,200 incl.	±0,030 mg/l	_	
above +5,0 °C to +15,0 °C incl.	above 0,200 - 1,200	-	± 15 %	
-1	0 - 0.200 incl.	±0,020 mg/l	,—	
above +15,0 °C to +25,0 °C incl.	above 0,200 - 1,200	_	± 10 %2)	
-b	0 - 0,200 incl.	± 0,030 mg/l		
above +25,0 °C to +50,0 °C incl.	0 - 0,200 incl. above 0,200 - 1,200 0 - 0,200 incl. above 0,200 - 1,200 0 - 0,200 incl. above 0,200 - 1,200		± 15 %	

2.19).

2.5 There are no the additional errors from existence of unmeasured components in an analyzed gas mix.

2.6 Parameters of the analyzed gas mix when supplying a sample to the analyzer input (automatic breath sampling mode):

gas mix volumetric flow is:

at least 20 l\min;

analyzed gas mix volume is:

at least 1.21.

2.7 The time of preparing for operation after turning on is: no more than 5 sec.

2.8 The measurement time after breath sampling is: no more than 15 sec.

2.9 The time of preparing for operation after 0,50 mg/l gas mix

concentration measurement is:

no more than 30 sec.

2.10 An analyzer operating time interval

without readings adjustment is: no more than 12 months. Readings adjustment is carried out before metrological verification procedure or by need.

2.11 The Analyzer has the firmware TouchK, RU V1.28 version or more.

2.12 In the printer paper compartment the thermal paper roll with the next dimensions is installed:

the paper width is (56 ± 2) mm, the external diameter is (24 ± 2) mm.

2.13 Modes of analyzer operating are:

- the measuring mode with data input;
- the measuring mode without data input;
- the screening mode:
- the data transmitting to PC mode;
- the mode of readings adjustment and check;
- the mode of readings check through the dry gas sample;;
- the touchscreen calibration mode.

- 2.14 The Analyzer has two breath sampling modes: automatic and manual.
- 2.15 The memory of the analyzer can hold not less than 16000 results of the last measurements.
- 2.16 Overall analyzer dimensions (Length x Width x Height) are no more than 220 mm x 80 mm x 40 mm.
- 2.17 The mass of the analyzer (with rechargeable power unit) is no more than 600 g.
- 2.18 An electrical power of the analyzer can be carried out from AC mains 230±23V/50±1 Hz through the AC power adapter, from DC mains 12 (24) V through the vehicle power adapter, or from a rechargeable power unit 7,4 V, not less than 4300 mAh.
- 2.19 Operating conditions
- temperature range of ambient air:

from minus 5 to plus 50 °C;

- relative humidity range of ambient air; from 10 to 100% (without condensation);
- atmospheric pressure range: from 84,0 to 106,7 kPa.
- 2.20 Work term of the fuel cell sensor is not less than 2 years.
- 2.21 Average work term of the analyzer is 5 years.
- 2.22 Average time between failures is 8000 hours.

3 Options

3.1 Before selling the analyzer, the following fields of the protocol form are pre-set (according to Breath alcohol analyzers ALCOTEKTOR PRO-100 touch-K. Operation manual, 2020):

□ Opti	on 1	Other:
No.	Content	Saved*
1		
2		
3		
4		
5		
6		
7		
8		
9		
10		

* Data entered in the checked fields are saved in the analyzer memory.

3.2 The blocking after the metrological verification interval will end is installed:

4 The analyzer's supplied set

The analyzer's supplied set is given in table 3.

Table 3

Item	Name	QTY
1	Analyzer	1 pcs.
2	Mouthpiece	105 pcs.
3	Mouthpiece cup	1 pcs.
4	Rechargeable power unit	1 pcs.
5	AC power adapter	1 pes.
6	Vehicle power adapter	1 pcs.
7	USB cable	1 pes.
8	Thermal paper	6 pcs.
9	Pouch	1 pcs.
10	Case	1 pcs.
11	Operation Manual	1 exs.
12	Technical data sheet	1 exs.
13	Metrological Verification Procedure MII-242-1956-2015	1 exs.

Notes:

1 The analyzer's supplied set includes mouthpieces supplied by the manufacturer and / or mouthpieces «Мундштук по ТУ 22.29.29-001-82139963-2017 (идентичны ТУ 2291-001-82139963-2015), исполнение «Мундштук АЛКОТЕКТОР с двумя отверстиями» [Mundshtuk po TU 22.29.29-001-82139963-2017 (identichny TU 2291-001-82139963-2015), ispolnenie «Mundshtuk ALCOTEKTOR s dvumya otverstiyami]. 2 One thermal paper roll is installed into a printer before supplying.

3 The Operation Manual and Metrological Verification Procedure may be stitched together.

5 Technical maintenance registration

- 5.1 On-site technical maintenance of analyzer includes:
- the charging of rechargeable power unit at low voltage;
- the refill the printer with paper;
- · lithium battery replacement, as needed;
- · the touchscreen calibration, as needed;
- analyzer cleaning and disinfection as it gets dirty (the frequency is set by the user).
- the visual inspection is recommended before the metrological verification
- 5.2 The periodic technical maintenance of the analyzer during the life of the device includes:
- technical condition control: it is recommended before the metrological verification;
- readings checking: it is recommended before the metrological verification (or more frequently according to frequency set by user);
- readings adjustment (under the negative results of readings checking or before the metrological verification if necessary).

5.3 It is recommended to register performed technical maintenance, repair, changes of options under 3.1 and 3.2 in the table 4.

Note: Only service centers do records about service carried out in the technical data sheet; it is recommended to the user to register service carried out by himself in the own maintenance register.

CALLE LINE	en a contract				
	Quality	acceptan	ce certificate	5	
6.1 Inp Breath	alcohol	analyzer	ALCOTEKTOR epted according to		
conside	red suitabl				
	re version ility of add	number _ litional acc	ess levels:		To the second second
□ No		es (numbe	r of levels	_)	
Year o	f manufa	cturing _			

Representative of the technical control department (TCD) (of manufacturer's authorized representative):

initials, sumane	signature

6.2	Primar	verification	procedure
-----	--------	--------------	-----------

Breath	alcohol	analyzer	ALCOT	EKTOR	PRO-	100	louch-K	serial	number
				accordi					
		nalyzer AL							
Procedu	ire», appro	oved by D.I.	Mendele	ev All-Ru	ssian I	nstitute	for Metro	ology 23	Octobe
2015.									

in							
	institution	registration	number	of the	certificate of	f accreditation	ŕ

and is recognized suitable for use in the fields of state regulation of ensuring the uniformity of measurements based on the results of the primary verification.

Verification date	
The next verification date is not later than	
Verificator:	initials, surname
Signature	HIII CALLS SOFFICIALISTS

Verification mark

Table 4

Table 4					
Initials, surname, signature					
Service centre mark Initials, surname, (stamp)	7/-				
Type of technical maintenance performed					1.
Date					

						0			
Initials, surname, signature					ang.	101			
Service centre mark (stamp)				77900					Hadao
Type of technical maintenance performed	Mood,	OU BULL	CHONICA			Hebankii	CALCA CALCA	190xc	
Date					8				

7 000 AL	COTEKTOR V	Varranty	
polity of the au pecified in 7. the Supplier to months from to een) months is anty is valid Datasheet, res implementation with the signa	nalyzer SN	within 12 al Datasheet. When the ana y for quality and serviceable ecified in 7.8 of the Technical in the Technical stamps of both Supplier and digations is possible only up (operating organization) in	consibility for the quality and servicea- (Twelve) months from the date of sale, lyzer is sold to the Buyer by the Seller, lity of the analyzer within 12 (Twelve) cal Datasheet, but not later than 15 (Fif-I Datasheet. In this case Supplier's ward Seller in 7.7 and 7.8 of the Technical on presenting this Technical Datasheet. date specified in 7.7 of the Technical
he address of Seller is entitl	the Supplier. If the led to warranty re	ne claim is found justified, the	anty period, the user should deliver it to the Buyer (operating organization) or the ficiencies or replacement with a similar.
COTEKTOR,	phone: (812) 320)-22-97. The reception of a i-7, office 148, Naberezh	e service centre of Supplier: OOO AL- malyzers for repair and service is car- naya reki Smolenki, 199178, Saint-
The answith valid wa	alyzers can be a arranty, undama	ged seals or brand marks	r and service includes: air if they have Technical Datasheets (if any are on the device), and a letter oplier does not take warranty obliga-
Recomm	nends to present t	he analyzer for warranty ser	vice in its entirety (excluding consuma-
For war functions cause violations (sed by the following the analyzer op	ng:	hout mechanical damage and other mal- product for other purposes or following
c) unqualified	tended power sup	ply, repairs carried out by unaut	horized persons,
e) accidents, r 7.5 The Buye defective anal	natural disasters, for coperating organizer to the address	s of the Supplier.	transportation costs for the delivery of a of the repaired analyzer to the address of
		tasheet is lost, it does not re lier (OOO ALCOTEKTO	
stamp	date of sale	signature of Supplier representative	initials, surname
7.8 The date	of sale by Selle	r:	
stamp	Seller	date of sale	signature of representative, initials, surname
7.9 I have re	ead the warranty		in the company of the first contract and the contract of the contract.

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